# GIMA

**LARINGOSCOPI A FIBRE OTTICHE**  
**FIBRE-OPTIC LARYNGOSCOPES**  
**LARYNGOSCOPES À FIBRES OPTIQUES**  
**LARYNGOSKOP MIT FIBER OPTIK LICHTLEITER**  
**LARINGOSCOPIOS DE FIBRAS ÓPTICAS**  
**LARINGOSCÓPIO A FIBRAS ÓPTICAS**  
**ΛΑΡΥΓΓΟΣΚΟΠΙΑ ΜΕ ΟΠΤΙΚΕΣ ΙΝΕΣ**  
**منظار الحنجرة العامل بالألياف البصريّة**

**MANUALE D'USO E MANUTENZIONE**  
**USE AND MAINTENANCE BOOK**  
**INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN**  
**BETRIEBS UND WARTUNGSANWEISUNGEN**  
**MANUAL DE USO Y MANTENIMIENTO**  
**MANUAL DE USO E MANUTENÇÃO**  
**ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**  
**دليل الإستعمال والرعاية**

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## FEATURES

The GIMA laryngoscopes are useful medical appliances to examine the larynx and to execute tracheal intubations.

The Green GIMA line is made by anti-magnetic stainless steel which is highly resistant to corrosion and uses fibre-optic blades and handles compatible with all major producers (certified ISO 7376-3). Light is transmitted from the handle to the tip of the blade through a fibre-optic bundle that can be taken apart to be sterilized independently or to be replaced in case of malfunction.

The handle uses two alkaline batteries "C" type 1,5 volt or rechargeable batteries, not supplied with the item.

The product or its components cannot be used for purposes different from the one specified in the present manual.

## PRESCRIPTIONS

*The product shall be used exclusively with alkaline batteries type "C" correctly installed. It shall not be dipped into water! When cleaning, make sure that the lamp socket or the battery compartment is not soaked with water.*

*Do not use the equipment in case it is damaged. Apply to your retailer. Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.*

*The tracheal intubations need a special training. The laryngoscopes must be used only by specialists who have been instructed about the specific technique.*

*The blade of laryngoscope can transmit infections from a patient to the other. Therefore proper measures must be taken in order to clean, disinfect or sterilize the blade after each operation.*

*Blades are to be kept in sterile conditions. The new blades are not sterile.*

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

### UNPACKING

*Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.*

*They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).*

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

### INSTALLATION

The user must check the following steps before starting an intubation process:

- checking that on the blades there are no sharp rims or imperfection due to mechanical damages
- the fibre-optic bundle must be properly fixed on the blade
- the bulb must be correctly screwed on the inner part of the handle.

The blade has to be fixed and turned to the “on” position. Please check that the bulb radiates enough light, a low light might mean that the batteries are almost flat.

Check that there is not too much “play” between the blade and the handle and that it does not cause an intermittent light (should it happen, the handle would be probably worn and need to be replaced).

If intubation process is often executed, for example in the operation theatre, it is better to have spare instruments because it is not possible to state precisely how long batteries last.

### OPERATION

First please choose the size and type of blade that fit better the required application.

For each model of handles, different sizes and types of blades are available.

Insert the fork of the blade in the proper pin on the top of the handle.

To turn on: the lamp of the laryngoscope will turn on automatically when the blade is fixed in the “on” position. The blade is fixed when, turning it to the top, a click is heard.

To turn off: to exit the “on” position please fold the blade towards the bottom.

### MAINTENANCE

*Periodically check the battery conditions, making sure that no sign of corrosion or oxidation is present, in case of necessity replace them with new alkaline ones. Carefully handle the batteries as the liquids they contain can irritate skin and eyes.*

Before being used, thoroughly check the product. The same operation shall be carried out after cleaning.

### HOW TO REPLACE BATTERIES

Unscrew the bottom of the handle and take the batteries out. Insert the new batteries keeping the right polarity as stated. Then screw the bottom of the handle on again. Check that the handle is properly screwed before using it.

The power of some kind of batteries (above all the rechargeable NiCd) decreases faster while using them, causing a quick decrease of the light. Consequently GIMA advises not to use NiCd batteries for safety reasons.

### HOW TO REPLACE BULB

Should the bulb stop working, it is possible to replace it with an original spare element. Unscrew the upper part of the handle. Unscrew the bulb container. Unscrew the black top. Unscrew the broken bulb and take it away. Screw the new bulb just to the end. Screw the black top on. Screw the container of the bulb on the head of the handle and the head onto the handle.

### HOW TO REMOVE FIBRE-OPTIC BUNDLE

Should it be necessary the fibre-optic bundle on the blade can be removed following these steps: take out the screw by a screwdriver. Pull out the green block and then take the bundle off.

Insert the new bundle, place the green block and screw it on. Make sure that the screw is properly tightened.

### BLADES- cleaning and disinfection

We advise to clean the instrument immediately after using it in order to avoid any residual formation. Rub the blade, better under running water or by a cloth and a proper detergent. Dry it carefully. Do not use abrasive materials.

*DO NOT scrape or use abrasive materials to clean the plated electric contacts.  
Avoid cleaning by ultrasounds! It might damage the instrument!*

To disinfect the instrument plunge it in a disinfecting solution or submit it to a thermo-chemical process to a maximum of 65° C.

When disinfection is over just wash it by sterile water and dry it by a sterile cloth.

*As far as concentration of solution are concerned please follow always the producer's instructions!*

### BLADES: Sterilization

*We advise to remove the blades out of the fibre-optic bundle before sterilization.  
A frequent sterilization can alter the optic fibres and decrease the light emission.*

Sterilization through gas.

Sterilization through gas with ethylene oxide to a maximum of 65° C and a pressure of 8 p.s.i. is better, above all when frequent sterilizations are foreseen.

Sterilization in autoclave:

The sterilization in autoclave can be used even if it is not advised because it reduces optic fibres life largely.

*Do not exceed the temperature of 134° C and the pressure of 28 p.s.i!  
Respect the generally advised sterilization time! (as stated by country rules)  
Quick sterilization in autoclave or by warm air emission are not advised because high temperature and frequent changes in temperature reduce life of the item!*

#### Sterilization by immersion

Plunge the item in the sterilized solution avoiding contact with the metallic parts. Follow advises of the producer concerning the solution concentration and time of immersion.

When the process is over, wash it in sterile water to remove any chemical residue and dry it with a sterile cloth.

*Use a proper product and respect time and measuring suggested by producer!  
GIMA advises not to use solutions which contain hypochlorite sodium or peroxide solutions.*

#### HANDLES:

To clean, disinfect and sterilize the handles, please follow the same instructions stated for blades. Anyway please remember always to remove batteries and bulb.

Before insert batteries and bulb, make sure that the handle is dry, inside and outside, better using warm air. The bulb can be cleaned using some cotton wool soaked in alcohol.

*DO NOT scrape or use abrasive material to clean the plated contacts on the handle because they might be damaged.  
DO NOT plunge the handles in cleansing agents or solutions which are not proper for this use.  
DO NOT put handles in autoclave without removing batteries or bulb!*

#### GUARANTEE

Thank you for having acquired our product. This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, etc. No compensation will be given for loss of use of the product.

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
Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

*This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.*

The guarantee will be invalidated if the registration number has been removed, cancelled or altered.

Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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---

**Disposal:** The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.

For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

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