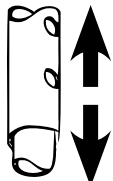
Ref					
250700-250800	70-94cm	Ø 20mm	Ø 17mm	100kg	0.38kg
243603-243604-243623-243624	70-94cm	Ø 22mm	Ø 19mm	120kg	0.36kg
243625-243626-243635-243636	84-94cm	Ø 22mm	Ø 19mm	120kg	0.36kg
250280-250281	82-92cm	Ø 20mm	Ø 17mm	100kg	0.4kg

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Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place ! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0° or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips...).
- ♥ Check regularly if all parts are tightened (clips, handle, rubber tip..).
- ♥ Hold firmly and completely the handle when using.
- ♥ In case of falling : let the product fall down on the side, so that you don't fall on it.
- ♥ Always move forward quietly and with short steps to ensure a good balance of your weight and to avoid losing your balance.
- ♥ Don't use on wet or icy surfaces. (snow, ice, water..)!!
- ♥ Don't use in stairs, severe risk of injuries!!
- ♥ Use only as a walking aid and in proper way : don't use as a lever or to move furniture.
- ♥ ALWAYS observe the weight limit on the labelling of your stick.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After assembling the walking aid, make sure that it is securely locked in OPEN position and level to the ground before using.
- ♥ ALWAYS test to see that the walking aid and attachments are properly and securely locked in place before using.
- ♥ DO NOT hang anything on the Walking aid.
- ♥ In case you didn't use your walking aid for a long time, let it checked by the technical staff, before using.
- ♥ Be careful when using: beware holes and deformations in the floor.
- ♥ The suction cap must not be too worn down. If the cap shows considerable wear, ask your supplier or nursing staff to replace it.

2. Installation

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient.

The device must be adjusted individually by the technical staff to suit your needs.

Ask the technical staff how to use this product and make sure that you understood clearly their explanation.

In case of doubt, don't hesitate to ask again.

References : 250700.250800

Height Adjustment : Take out the clip from the tubes using both thumbs, then adjust the interior pipe to the desired height and lock again with the clip. Before using the crutch, check if the metal pin of the clip is correctly inserted into the pipe. The sticks have 1 adjusting hole on the outside pipe and 5 holes on the inside pipe, what means 5 adjusting positions. The stick can be adjusted every 2 cm.

References : 243603.243604

Height Adjustment : Take out the clip from the tubes using both thumbs, then adjust the interior pipe to the desired height and lock again with the clip. Before using the crutch, check that the metal pin of the clip has gone through both pipes and exits from the outer pipe.

The sticks have 1 adjusting hole on the outside pipe and 5 holes on the inside pipe, it means 5 adjusting positions.

The stick can be adjusted every 2 cm.

References : 250280.250281

For height adjustment, loose locking ring, grip stick shaft with one hand whilst pushing the metal locking pin down with the other hand so the locking pin retracts fully into the shaft. Select required height by adjusting the top section of the shaft in an upwards or downwards motion and release the metal locking pin in front of the required hole position and tighten locking ring. Prior to use always ensure that the locking pin fully protrudes from the selected hole position and by applying downward pressure on the handle of the walking stick test that the walking stick has been assembled correctly.

The stick folds in 4 elements. Prior to use always ensure that the locking ring is tightly secured in place and that, following height adjustment of the stick, the metal locking pin is always fully engaged in the selected hole position.

To unfold, release one tube at a time, guide them onto the handle tube until all tubes are connected together. Check that all tubes are seated in place.

To fold, pull the bottom two tubes apart and fold onto each other. Repeat this operation with the remaining tubes. Place into the bag when folded.

Keep the sticks close to your body while going to avoid accidents.

Rental: this product can be rented under the condition that it was correctly cleaned. This user's manual belongs to that product and must be delivered to each new user.

3. Cleaning

The stick has to be cleaned regularly with a mild cleaning agent and a soft cloth.

NEVER use OILY substances!! Slip hazard!!

Ask your retailer or the technical staff to check the walking aid regularly. The rubber tips must be checked and changed regularly.

4. Materials

Tube in anodised aluminium and polypropylen handle.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

Check the table at the end of the user's manual.

This product has a 24 months warranty, from the date of purchase, against any manufacturing defect.

Standards: DIN-EN-12182: 1999, EN-ISO 11334-1, DIN-EN-ISO 10993-1. Biocompatible.

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