

User manual

BELT

REF 34010 - 34011

Size: 5x213 cm



This product is used in stretchers during patient transfer.
 Loading Capacity: More than 60kgs

Operation

Unfasten the safety strap, and fix the patient on a stretcher, then fasten the buckle, adjust the length of strap for securing the patient properly.

Product storage and maintenance:

3. After each use, please use clear water or neutral detergent to clean the dirt on the surface. Store it for subsequent use when it is dry.
4. If there are bloodstains or body fluids left, please disinfect the product with disinfectant, and clean up.



Symbols	Symbols description
	Please read instructions carefully
	Keep away from direct sunlight
	Store in a dry place and avoid humidity
LOT	Lot number
	Manufacturer
REF	Product code