## ITALIANO

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

## ENGLISH



Thank you! Have you bought an aid intended for use in toilets or bathrooms. Its purpose is to facilitate the users to use the disabled toilets. Any other use is prohibited. Use only in a seated position and do not stand on it; for mounting please refer to the explanatory images below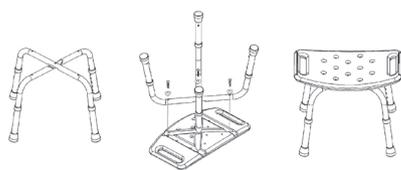




REF 42900

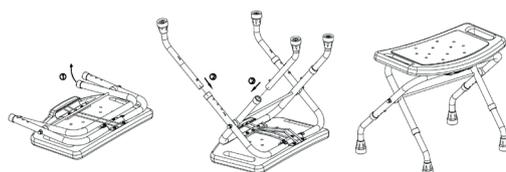REF 42901

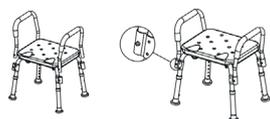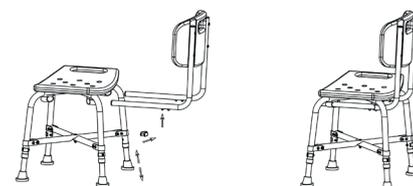REF 42907

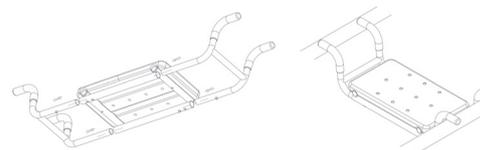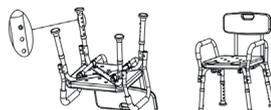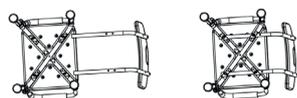REF 42910

## Cleaning

Use damp cloth with normal detergents. Do not submerge the seat in the bath water; drain after use and dry the seat. Don't use harsh abrasives or bleach based cleaners.

## Warning

- These products are recommended for people weighting kg. 100 maximum.
- The manufacturer and distributor decline all responsibility for any consequences resulting from an incorrect use of this product.
- Check that the product is in perfect condition before you use it.
- Do not store the products outdoors. Please check the stability of the product, if screws are properly tightened and the rubber tips are in good conditions before use.
- Please help people with movement difficulties to seat on the products.
- Avoid any hard movement during patient positioning on the chair. Pay attention on wet floors, can cause problems during patient positioning on the chair; do not stand on or use if there is on soap or mousse
- Avoid sudden movements during positioning. Be careful wet floors, wet or slippery in general, could make it difficult patient positioning.