# GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy  
Italia: tel. 199 400 401 - fax 199 400 403  
Export: tel. +39 02 953854209/221/225 - fax +39 02 95380056  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

## **CARROZZINA ESSEX ESSEX WHEELCHAIR FAUTEUIL ROULANT ESSEX SILLA DE RUEDAS ESSEX**

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## 1. Simple Structure Figure

### **Transfer Version with 12" Rear Wheels**

This wheelchair version is particularly suitable for the patient transfer.

1 Handle	2 Side plate
3 Rear wheels	4 Front wheels
5 Front forks	6 Foot rest
7 Leg strap	8 Seat and back upholstery
9 Cross bars	10 Footplate

## 2. General Safety Instructions

**2.1** Please read the instructions for use first! Before using the product, you should be familiar with the handling, function and use of the product. You may compromise your safety if you do not observe these instructions. However, it is impossible to protect against and avoid all unforeseeable circumstances and situations.

- 2.2** Do not ascend or descend stairs without the assistance of attendants. If devices such as ramps or elevators are available, they should be used. If wheelchair-friendly access is missing, two attendants must carry the wheelchair over the obstacle.
- 2.3** If possible, the footplates should be flipped up before getting into or out of the wheelchair.
- 2.4** Risk of accident when driving without experience. This can lead to falls and other dangerous situations. Therefore, familiarize yourself with the new wheelchair by practicing on even, straightforward terrain first. Learn how the wheelchair reacts when the center of gravity shifts; for example on slopes or inclines or when clearing obstacles such as steps and curbs. This should be done only with the assistance of another person. Using an anti-tipper is strongly recommended for inexperienced wheelchair users.
- 2.5** All brakes acting on the tires do not serve as service brake but are only designed as parking brake (wheel lock). The wheel locks must not be used as driving brakes for slowing down the wheelchair, as in extreme cases, the abrupt stopping of the wheelchair can lead to falls.
- 2.6** Risk of damage caused by overload. The maximum load capacity is 120 kg.
- 2.7** The drive wheels must be replaced in case of tire damage (tread becomes worn down to such an extent that there is only 5 mm of radius left on the outer edges of the tire) or damage on the rim. The 12" drive wheels of the transfer version is replaced in the same way.

### **3. Delivery and Preparation for Use**

*(Please Note: The photos here may show differences to the wheelchair you actually received).*

Your wheelchair is usually delivered fully assembled, folded, and with the rear wheels mounted.

A few simple steps are sufficient to prepare it for use:

- 1.** While standing next to the wheelchair (Figure 1), unfold the wheelchair using flat hands (Figure 2). Ensure that the telescoping bars (Figure 1-A) are completely collapsed and that the seat upholstery is flattened out.

fig. 1

fig. 2

2. If applicable, insert the footrests into the receivers from the side (Figure 3). Turn the footrests inward until locking to the right place. Finally, fold down the footplates (Figure 4).

fig. 3

fig. 4

## 4 Transportation

Grab the middle of seat upholstery at the front and the back and pull it upwards, the wheelchair will be folded (Figure 5).

For more manageable transportation, e.g. in cars, the footrests can be entirely removed in just a few easy steps. To move the footrest, press the release lever to the inside (Figure 6). The footrest with footplate can then be swung outward by 90° and be removed by pulling it upward.

fig. 5

fig. 6

## 5. Armrest usage

The side panels can be flipped-up to the rear for transfer user (Figure 7).

To raise the side panel, manipulate the release lever (Figure 8) behind the side panel tube. The height of the armrest is not adjustable.

fig. 7

fig. 8

## **6. Footrests usage**

The height of standard footrests can be changed with tools (Figure 9). To release the screws and then adjust the height to right place for user, finally tighten the screws.

fig. 9

fig. 10

## **7. Replacing the Seat and Back Upholstery**

Can be removed if necessary by loosening the cross screws (Figure 10).

## **8. Cleaning and Disinfection**

- 8.1** Clean the upholstery with warm water and a washing-up liquid. Remove spots with a sponge or a soft brush. Rinse with clear water and let the components dry.
- 8.2** Do not use any aggressive cleansing agents, solvents, or hard brushes etc.
- 8.3** No wet-cleaning. Components are not machine washable. For disinfection, water-based agents (e.g. Sagrotan original concentrate) should be used. The manufacturer's instructions for use must be observed.
- 8.4** Before disinfecting, the pads and handles must be cleaned. Plastic parts, frame parts as well as chassis and wheels can be cleaned with a mild detergent. Dry thoroughly afterwards.

## **9. Re-use**

### **Information on Re-Use**

The product is suitable for re-use. Similar to second-hand machines or cars, products that are being re-used are subject to increased strain. Features and functions must not change in a way that could endanger patients or other persons within the product's life cycle.

Based on market observations and the current state of technology, the manufacturer has calculated that the product can be used for a period of **4 years**, provided that it is used properly and that the service and maintenance instructions are observed. Periods during which the product is stored at the dealer or with the third-party payer are not included in this period. It should be clearly pointed out, however, that the product is reliable far beyond this defined period of time, provided that it is cared for and maintained appropriately.

In cases of re-use, the affected product must first be thoroughly cleaned and disinfected. Subsequently, the product must be examined by an authorised specialist to check the condition, wear and possible damage. Any worn and damaged components as well as components which do not fit or are not suitable for the user must be replaced.

## 10. Specifications and Parameters

Specifications	
<b>Style</b>	Transport
<b>Surface finishing</b>	Powder coating
<b>Frame</b>	Steel and foldable
<b>Armrest</b>	Flip-up desk length armrest
<b>Seat and backrest</b>	Nylon upholstery
<b>Cross bars</b>	Seat width 43cm with single Seat width 46cm with single Seat width 51cm with double
<b>Footrest</b>	Swing away footrest with heel strap and height adjusted clips
<b>Rear wheel</b>	12" Mag rear wheel with PU tire
<b>Front wheel</b>	8"x2"
<b>Handrim</b>	Without
<b>Front fork</b>	Steel with black powder coating
<b>Footplate</b>	Aluminum alloy
<b>Anti-tipper</b>	Without
<b>Brake</b>	With hand brake
Measurements	
<b>Total length</b>	1020 mm
<b>Total width</b>	630 mm - 680 mm
<b>Total height</b>	920 mm
<b>Folding width</b>	260 mm
<b>Seat width</b>	460 mm - 510 mm
<b>Seat depth</b>	400 mm
<b>Backrest height</b>	410 mm
<b>Back of seat to floor</b>	480 mm
<b>Front of seat to floor</b>	500 mm
<b>Seat to armrest</b>	275 mm
<b>Net weight</b>	18 kg
<b>Gross weight</b>	20 kg
<b>Weight capacity</b>	120 kg
<b>Carton size</b>	820x260x850 mm

**Disposal:** *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment. For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.*

## **GIMA WARRANTY CONDITIONS**

Congratulations for purchasing a GIMA product. This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA. During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included. All components subject to wear are not included in the warranty. The repair or replacement performed during the warranty period shall not extend the warranty. The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use. GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc. The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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