Istruzioni per l'uso
Instructions for use
Mode d'emploi
Bedienungsanleitung
Instrucciones para el uso

DISTILLER

INDICE – INDEX - INDEX

INFORMAZIONI AL DESTINATARIO
INFORMATION TO THE CONSIGNEE
INFORMATIONS POUR LE DESTINATAIRE

2-5

ITALIANO

6-13

ENGLISH

14-21

FRANÇAIS

22-29

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Tecno-Gaz S.p.A., seated in Strada Cavalli 4, 43038 Sala Baganza, Parma, Italy.

Declares under its responsibility, that the water distiller "DISTILLER"

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Imported by Tecno-Gaz S.p.A., is in conformity with the laws that comply with the 73/23/CEE Low Voltage Directive, with the 89/336/CEE Electromagnetic Compatibility Directive and their subsequent modifications and integrations.

The Tecno-Gaz S.p.A. Quality System is in conformity with the ISO 13485 e ISO 9001 regulations, approved by DNV (Det Norske Veritas Italia).

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The “DISTILLER” model has been inspected in accordance with the requirements of the directive **73/23/CEE e 89/336/CEE** and later modifications and additions.

The fact that this inspection has been made can be seen from the plate on the machine which bears the CE mark shown in the conformity declaration which accompanies this booklet.

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WARRANTY

“**DISTILLER**” is covered by twelve-months’ guarantee, except for parts of normal usury.

The guarantee starts from the date of the product’s delivery to the customer, proved by the guarantee’s coupon duly filled in, stamped and signed by the dealer.

In case of objection, the date indicated on the sale note is judged to be valid.

The reparation or the replacement under warranty is carried out after unquestionable judgment of TECNO-GAZ S.p.A and doesn’t include the staff’s transfer, the packaging cost and the transport.

The guarantee doesn’t cover damages caused by the bad maintenance, negligence, inexperience or other causes not due to the manufacturer, direct or indirect damages of any sort to people or things.

The replacement of the whole machine is not envisaged.

No compensation is envisaged for the machine’s stop.

The guarantee falls if the machine is repaired, modified or damaged by the customer or others not authorized.

For assistance, the customer has to address only to the dealer or to the after-sales service centers indicated by TECNO-GAZ S.p.A

ATTENTION:

Original packaging should be preserved during the whole endurance of the guarantee.

Don’t disperse the material of the packaging in the environment, recover recyclable products and give the rest of the products to a specialized company .

Packed machines must be preserved in a dry place and with a temperature between 10°C to 40°C.

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E-Mail : info@tecnogaz.com

<http://www.tecnogaz.com>

For technical service please apply to the dealer that invoiced the device, or directly to:

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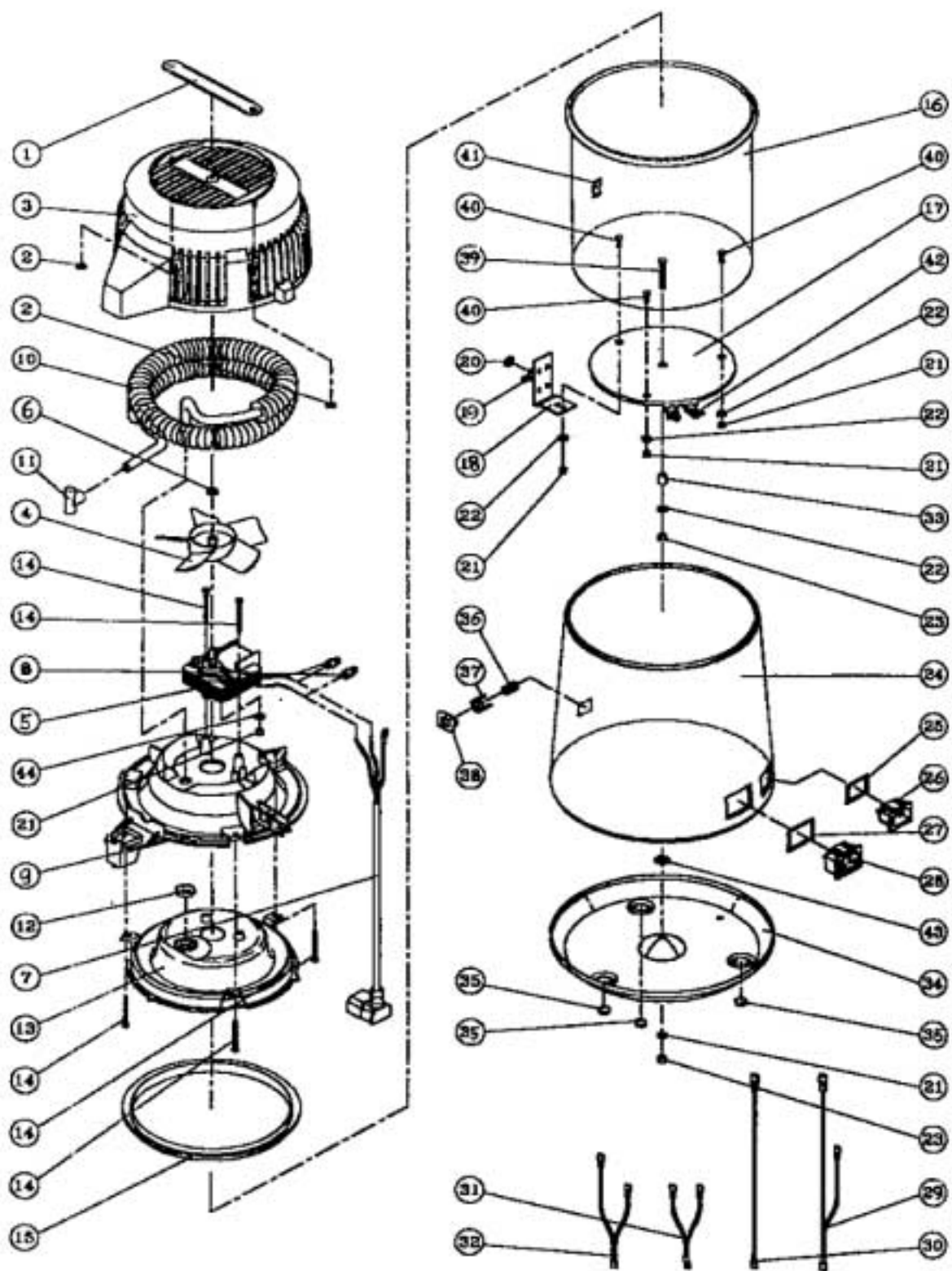
file:XDIM001

INDEX

PAR	DESCRIPTION	PAGE
01	INFORMATION TO THE CONSIGNEE	2
02	TECHNICAL FEATURES	15
03	DEMOLITION	15
04	WORKING AND USE	15
05	INSTRUCTIONS FOR USER	16
06	ORDINARY AND EXTRAORDINARY MAINTENANCE	18
07	ANALYSIS AND RESEARCH OF FAILURES	19
08	COMMERCIAL FEATURES, SPARE PARTS AND DOCUMENTATION	20
09	DIAGRAMS	21

PAR 02**TECHNICAL FEATURES****TECHNICAL FEATURES**

	AC 100V/60Hz	AC 220V/50Hz
Feeding available upon request	AC 110V/60Hz	AC 230V/50Hz
	AC 120V/60Hz	AC 240V/50Hz
Electric power absorbed (max)	580 Watt	
DIMENSIONS		
Diameter	200 mm	
Height	360 mm	
Weight	3.5 Kg	
CAPACITY	4 Lt	

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DISTILLER is made of plastic and ferrous materials recover recyclables products and give the rest of materials to a specialized company.

Don't abandon the machine in non-guarded places.

PAR 04**WORKING AND USE**

DISTILLER is a water distiller, it produces pure and clean water trough a natural and effective process.

The product is made of high quality materials and carefully manufactured.

Water is brought gradually to 100 °C and are eliminated:

the majority of virus, bacteria, organic and inorganic chiminal substances.

Chimical substances, heavy metals and contaminated agents settle in the bottom.

Impuritiесе smells, brought by the steam, are eliminated in the cooling coil.

Water is further purified by the active coal filters present in the filter holder collector.

PAR 05

INSTRUCTIONS FOR USER

INSTALLATION OF ACTIVATED COAL FILTER

Turn the cap (A) and remove it from the filter-holder container (B).

Fold the active coal filter (C- ref. SDIA042) and put it into the proper container (B)

Reassemble the container with the cap and place on the collection water tank.

Adjust the tank with the filter with the draining of the distiller.

A filter can be used for about a month

FILLING OF CHAMBER

Remove the connection from the upper cover (N) and lift the upper cover (G).

Fill the stainless steel making sure that it doesn't exceed the level (see internal mark) with cold water or hot water if you want to reduce the time necessary to obtain distilled water.

FIG 1

OPERATING

Verify that the upper cover is closed (G) and place the distiller on a flat surface, solid and safe, line up the tank with the draining of the distiller (D), putting the filter-holder container between them as shown (E). Connect the electrical feeding cable to the electric network (M) and the connection of the upper cover for the feeding of the engine (N).

The process is completely automatic, at the end of the distillation, water is finished, the machine switches off automatically.

In order to stop the machine before the distillation of all the water it is necessary to disconnect the electric feeding. After about 1 hour of working the process starts and after about 7 hours you'll have 4 liters of distilled water. The water obtained can be kept in a container preferably in refrigerator. Before consuming you'd better shake the container.

AT THE BEGINNING THE MACHINE IS READY TO DISTILL, THEN EVERY TIME YOU WANT DO START A PROCESS OF DISTILLATION IT IS NECESSARY TO REARM THE MACHINE PRESSING THE PROBER BUTTON (R)

FIG 2

FIG 2

**WARNING**

- BEFORE USING THE DEVICE READ THE OPERATING MANUAL
- PLACE THE DISTILLER ON A STABLE AND DRY SURFACE, AWAY FROM HEAT SOURCES
- KEEP AWAY FROM HEAT SOURCES, WE SUGGEST TO PLACE THE MACHINE IN A WELL-AIRED PLACE, BECAUSE THE FAN USES AIR FOR THE COOLING.
- DON'T OPEN THE MACHINE, FOR INTERVENTS ADDRESS TO ASSISTANCE CENTERS
- DON'T USE A PLASTIC WATER TO PRESERVE BOILING HOT WATER
- DISCONNECT THE PLUG FROM THE SOCKET IN ORDER TO STOP THE DISTILLATION
- DON'T REMOVE THE UPPER COVER WHILE THE MACHINE IS WORKING
- DISCONNECT THE ELECTRICAL FEEDING DURING THE FILLING AND THE CLEANING OF THE DISTILLER AND WHEN IT IS NOT WORKING
- DON'T PLUNGE ANY PART OF THE DISTILLER DURING THE CLEANING
- AVOID THE CONTACT WITH CHILDREN
- CHECK THE INTEGRITY OF THE MACHINE AND OF ITS PARTS, BRAKINGS, CRACKS OR OPENINGS CAN PREVENT THE USE
- CHECK THAT THE FEEDING CABLES ARE COMPLETE.
- DON'T USE PLUGS WITH WET OR HUMID HANDS.
- USE THE MACHINE ONLY FOR THE AIMS PROPOSED BY THE MANUFACTURER

THE APPLIANCE HAS NOT TO BE USED IN POTENTIAL EXPLOSIVE ATMOSPHERE.

PAR 06

ORDINARY AND EXTRAORDINARY MAINTENANCE

- As for the external cleaning of the DISTILLER, you can use a neutral detergent and then dry.
- Clean, if necessary, the gasket of the upper cover, but don't remove it, it's placed in a specific direction (with the lip upwards)
- The stainless steel internal chamber is delicate, don't use abrasive detergents for cleaning

CLEAN THE MACHINE AT THE END OF EVERY CYCLE

1. Put hot water in the chamber just to cover the residues
2. Put from 1 to 4 teaspoons of detergent supplied inside the chamber, the quantity depends on the residues.
3. With the chamber opened connect the feeding, bring to boiling the water and disconnect the connection to the network
4. After 20 minutes drain the water and rinse with clean water
5. Repeat this operation until the chamber is completely clean
6. Little quantities of residue in the bottom don't have to be considered harmful, the machine works in effective way.

COOLING COIL

If the cooling coil is obstructed, the effectiveness of the distiller could be compromised.

For the cleaning, unscrew the screws (G) in the part below the cover.

Remove the cover being careful during the extraction to the joint in position (H).

Remove the coil and brush the cooling commutators with care (I)

- Don't plunge the engine in the water during the cleaning of the cover.
- Add few drops of oil on the output shaft in homogenous way.

FIG 4

FIG 5

PAR 07
ANALYSIS AND RESEARCH OF FAILURES

N°	FAULT	CAUSE	INTERVENTION
1	DOESN'T PRODUCE WATER	FEEDING CABLE NOT CONNECTED TO THE SOCKET OR AT THE BASIS OF THE MACHINE	CORRECTLY CONNECT THE FEEDING CABLE
		RIARMO NON ESEGUITO	PREMERE IL TASTO DI RIARMO
3	DOESN'T PRODUCE ENOUGH DISTILLED WATER DURING THE CYCLE	YOU HAVEN'T PUT IN WATER RIGHT TO THE MAX LEVEL	FILL THE DISTILLER CORRECTLY
		THE GASKET OF THE COVER IS NOT CORRECTLY PLACED OR DOESN'T SEAL ENOUGH	PLACE THE GASKET CORRECTLY
		THE WATER CONTAINER IS NOT PLACED UNDER THE DRAINING OF THE DISTILLER	PLACE THE CONTAINER CORRECTLY
4	LEAK OF WATER AND STEAM FROM THE DRAIN	ACTIVATED COAL FILTERS ARE NOT PLACED IN THE SPOUT	REPLACE FILTERS CORRECTLY
		THE CONTAINER IS NOT PERFECTLY ALIGNED WITH THE DISTILLER	ALIGN THE CONTAINER

PAR 08
COMMERCIAL FEATURES, SPARE PARTS AND DOCUMENTATION

POS	QT	DESCRIZIONE	CODICE
1	1	HAND CARRY RIBBON	SDIA001
2	2	NUT	SDIA002
3	1	UPPER COVER	SDIA003
4	1	FAN	SDIA004
5	1	AC MOTOR	SDIA005
6	1	SPRING BUSHING	SDIA006
7	1	UPPER COVER POWER LINE	SDIA007
8	2	ALL PLASTIC CONNECTOR	SDIA008
9	1	SUPPORTINO CAP	SDIA009
10	1	COOLING DUCT	SDIA010
11	1	RUBBER BEND DUCT	SDIA011
12	1	RUBBER WASHER	SDIA012
13	1	LOWER COVER	SDIA013
14	5	STAINLESS STEEL SCREWS	SDIA014
15	1	CUSHION RUBBER RING	SDIA015
16	1	STAINLESS STEEL TANK	SDIA016
17	1	HEATER	SDIA017
18	1	ALUMINIUM BRACKET	SDIA018
19	1	THERMAL SWITCH	SDIA019
20	1	NYLON WASHER	SDIA020
21	5	SPRING WASHER	SDIA021
22	3	NUT	SDIA022
23	2	NUT	SDIA023
24	1	SHELL (TINPLATE / STAINLESS STEEL)	SDIA024
25	1	SEAL-A	SDIA025
26	1	AC SOCKET	SDIA026
27	1	SEAL-B	SDIA027
28	1	AC SOCKET	SDIA028
29	1	HEATING WIRE SETS - C	SDIA029
30	1	HEATING WIRE SETS - D	SDIA030
31	1	HEATING WIRE SETS - E	SDIA031
32	1	HEATING WIRE SETS - F	SDIA032
33	1	COPPER BUSHING	SDIA033
34	1	BASE COVER (TINPLATE / STAINLESS STEEL)	SDIA034
35	3	RUBBER PEDAL	SDIA035
36	1	SPRING	SDIA036
37	1	BUTTON	SDIA037
38	1	BUTTON BRACKET	SDIA038
39	1	STAINLESS STEEL SCREWS	SDIA039
40	1	STAINLESS STEEL SCREWS	SDIA040
41	1	STAINLESS STEEL PIECE	SDIA041
42	1	STAINLESS TERMINAL	SDIA042
43	1	STAR WASHER	SDIA043
44	1	SPRING WASHER	SDIA044

ACCESSORI


101	PC COLLECTION BOTTLE	SDIA101
102	PC COLLECTION BOTTLE LID (CHARCOAL FILTER)	SDIA102
103	ACTIVATED CHARCOAL (12 SACHEST IN A BOX)	SDIA103
104	CLEANER	SDIA104

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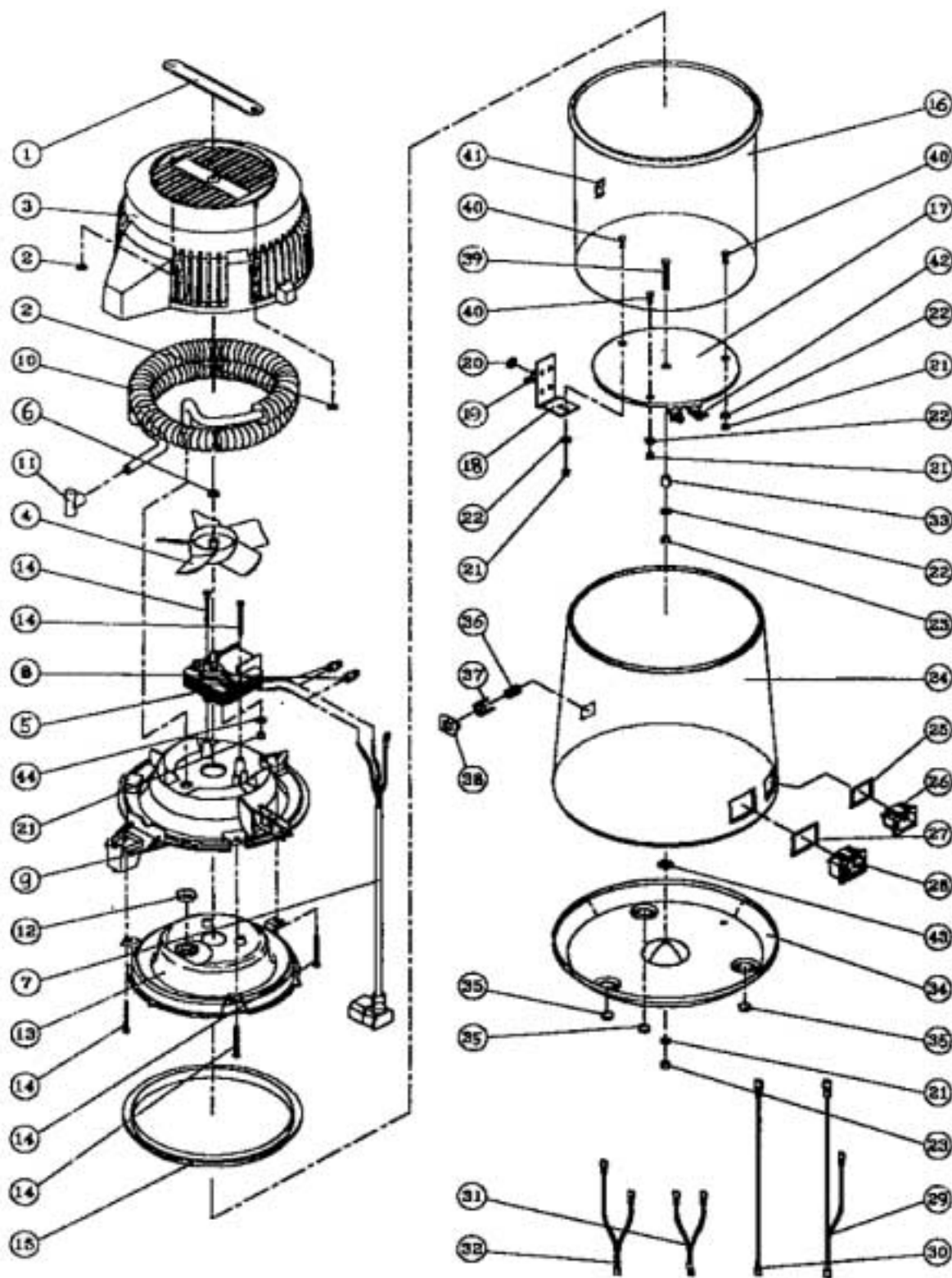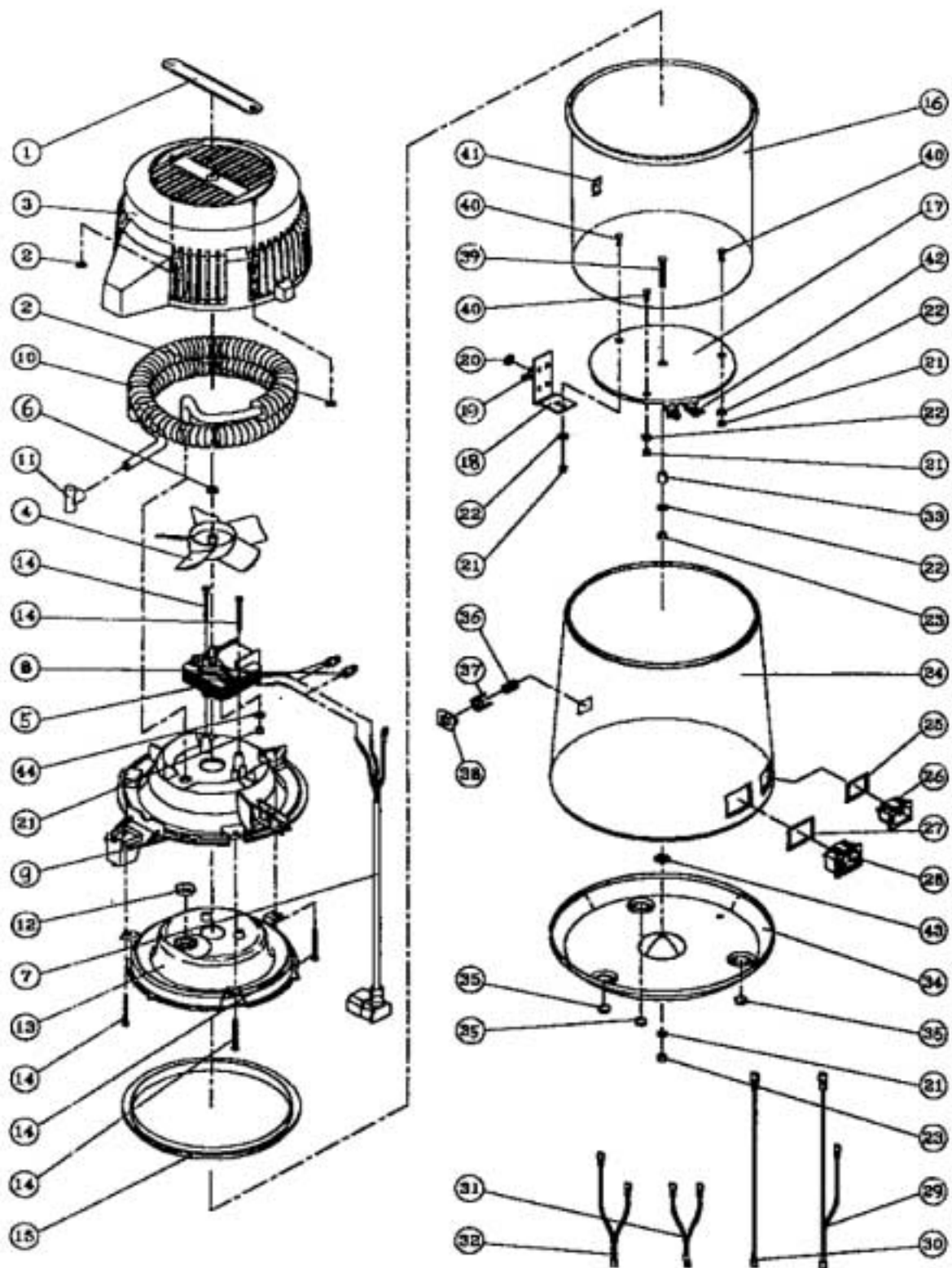
file:XDIM001

[illegible]

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

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GB

This manual must always be kept with the product, in complying with the Directives of European Community. TECNO-GAZ reserves the right to modify the enclosed document without notice. TECNO-GAZ reserves the property of the document and forbids others to use it or spread it without its approval.

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