

MISURATORE DI PRESSIONE DIGITALE DIGITAL BLOOD PRESSURE MONITOR TENSIOMÈTRE NUMÉRIQUE MONITOR DE PRESIÓN ARTERIAL DIGITAL MONITOR DIGITAL DA TENSÃO ARTERIAL DIGITALES BLUTDRUCKMESSGERÄT DIGITAL BLODTRYCKSMÄTARE CIŚNIENIOMIERZ CYFROWY MONITOR DIGITAL PENTRU MĂSURAREA TENSIUNII ARTERIALE

Digitalni monitor za krvni tlak

## ψηφιακο πιεσομετρο

يمقرل امدل طغض قبقارم زاهج



## CONTEC08E (GIMA 49880)




Contect Medical System Co., Ltd Adress: No 112 Qinhuang West Street, Economic&Technical Development Zone, Qinhuangdao, Hebei Province, People's Republic of China Made in China



Shanghai International Holding Corp. Gmbh (Europe) Address: Eiffestrasse 80, 20537, Hamburg, Germany



Gima S.p.A. Via Marconi, 1 - 20060 Gessate (MI) Italy gima@gimaitaly.com - export@gimaitaly.com www.gimaitaly.com

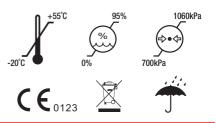

## FOREWORD

Please read the User Manual carefully before using this product. The User Manual which describes the operating procedures should be followed strictly. This manual detailed introduce the steps must be noted when using the product, operation which may result in abnormal, the risk may cause personal injury and product damage and other contents, refer to the chapters for details. Any anomalies or personal injury and device damage arising from use, maintain, store do not follow requirements of the User Manual, Our company is not responsible for the safety, reliability and performance guarantees! The manufacturer's warranty service does not cover such faults!

Our company has a factory record and user profile for each device, users enjoy free maintenance services for one year from the date of purchase. In order to facilitate us to provide you with a comprehensive and efficient maintenance service, please be sure to return the warranty card when you need repair service.

### ANote: Please read the User Manual carefully before using this product.

Described in this User Manual is in accordance with practical situation of the product. In case of modifications and software upgrades, the information contained in this document is subject to change without notice.

## The warning items

#### Before using this product, you should consider the safety and efficacy of the following described:

- Described each measurement results combined with clinical symptoms by qualified doctors.
- The reliability and operation of using this product whether meets the operation of this manual relate to the maintenance instructions.
- The intended operator of this product may be the patient.
- Do not perform maintenance and service while the device is in use.

AWarning: Replace accessories which not provided by our company may lead to the occurrence of errors. Replace adapters, cuffs at will may result in wrong measurement results. Without our company or other approved maintenance organizations trained service personnel should not try to maintain the product.

#### Responsibility of operator

- The operator must carefully read the User Manual before use this product, and strictly follow the operating procedure of the User Manual.
- Fully consider the security requirements during product design, but the operator should not ignore the observation for the patient and the state
  of machine.
- The operator has the responsibility to provide the use condition of the product to our company.

#### Responsibility for our company

- Our company have the responsibility to provide qualified product which conform to company standard of this product
- Our company will provide the circuit diagram, calibration method and other information at the request of the user to help the appropriate and qualified technicians to repair those parts designated by our company.
- Our company have the responsibility to complete product maintenance according to the contract.
- Our company have the responsibility to respond the requirements of user in time.
- In the following case, our company is responsible for the impact on the safety, reliability and performance of the device:
- Assembly, addition, debugging, modification or repair are carried out by personnel approved by our company.

The electrical facilities in the room are in compliance with the relevant requirements and the device is used in accordance with the User Manual.

## The User Manual is written by our company. All rights reserved.

## **CHAPTER 1 - FUNCTIONS AND PURPOSE**

**1.1 Main Functions** 

- Measure blood pressure and store the measurement results.
- Data storage function, up to 199 records can be stored.
- With data review interface which is convenient for reviewing blood pressure parameter.
- The screen will prompt message when the power is low.
- When the measurement result can not be obtained due to some factors during the measurement, the device will display the corresponding error information.
- Measurement units: mmHg and kPa, which can be switched by the button.
- With automatic shutdown function, if there is no operation, the device will automatically turn off.
- Voice broadcast(optional for devices with vioce function)

#### 1.2 Purpose

The device apply to measure the non-invasive blood pressure of human. Record parameter value of blood pressure to provide the reference for the health care professional.

## CHAPTER 2 - SAFETY PRECAUTIONS:

In order to use it correctly, please read the "Safety Precautions" carefully before using it.

Operators do not need professional training, but should use this product after fully understanding the requirements in this manual.

To prevent users from suffering damage or loss due to improper use, please refer to "Safety Precautions" and use this product properly.

#### For safety reasons, be sure to comply with safety precautions.

🛆 Note 🖄

## If not use correctly, it exists the possibility of damage for personnel and goods.

Good damage means the damage of house, property, domestic animal and pet.

 ${
m ar \Delta}$  Contraindication  ${
m ar \Delta}$ 

No.

#### 🗥 Warning 🗥

 You must not perform NIBP measurements on patients with sickle-cell disease or under any condition which the skin is damaged or expected to be damaged.

## For patients with severe disturbances of blood coagulation, whether automatically measure the blood pressure should be based on the clinical evaluation, because limb friction with the cuff may cause the risk of hematoma.

ENGLISH

For severe blood circulation disorder or arrhythmia patients, please use the device under the guidance of a doctor. If the arm is squeezed during
measurement, it may cause acute internal hemorrhage or inaccurate measurement results.

#### **Measurement Limitations**

To different patient conditions, the oscillometric measurement has certain limitations. The measurement is in search of regular arterial pressure pulse. In those circumstances when the patient's condition makes it difficult to detect, the measurement becomes unreliable and measuring time increases. The user should be aware that the following conditions could interfere with the measurement, making the measurement unreliable or longer to derive. In some cases, the batter's condition will make a measurement impossible.

#### Patient Movement

Measurements will be unreliable or can not perform if the patient is moving, shivering or having convulsions. These motions may interfere with the detection of the arterial pressure pulses. In addition, the measurement time will be prolonged.

#### Cardiac Arrhythmia's

Measurements will be unreliable and may not be possible if the patient's cardiac arrhythmia has caused an irregular heartbeat. The measuring time thus will be prolonged.

#### Heart-lung Machine

Measurements will not be possible if the patient is connected to a heart-lung machine.

#### **Pressure Changes**

Measurements will be unreliable and may not be possible if the patient's blood pressure is changing rapidly over the period of time during which the arterial pressure pulses are being analyzed to obtain the measurement.

#### Severe Shock

If the patient is in severe shock or hypothermia, measurements will be unreliable since reduced blood flow to the peripheries will cause reduced pulsation of the arteries.

#### **Heart Rate Extremes**

Measurements can not be made at a heart rate of less than 40 bpm and greater than 240 bpm.

#### **Round Patient**

The thick fat layer of body will reduce the measurement accuracy, because the fat that come from the shock of arteries can not access the cuffs due to the damping

#### 🗥 Warning 🗥

Self-diagnosis and treatment using measured results may be dangerous. Follow the instructions of your physician.

Please hand measurement results to the doctor who knows your health and accept diagnosis.

For Infant and the person who can't express oneself, please use the device under the guidance of a doctor.

Otherwise it may cause accident or dissension.

Please do not use for any other purpose except BP measurement.

Otherwise it may cause accident or holdback

Please use special cuff.

Otherwise it is possible that measurement result is incorrect.

Please do not keep the cuff in the over-inflated state for a long time.

Otherwise it may cause risk.

Do not use the device in the case of there are flammable anesthetic gasses mixing with the air or nitrous oxide.

Otherwise it may cause risk.

If liquid splashes on the device or accessories, especially when liquids may enter the pipe or device, stop using and contact the service department. Otherwise it may cause risk.

Dispose of the packaging material, observing the applicable waste control regulations and keeping it out of children's reach.

Otherwise it may cause harm to the environment or children.

Please use approved accessories for the device and check that the device and accessories are working properly and safely before use.

Otherwise the measurement result may be inaccurate or an accident may occur.

When the device is accidentally damp, it should be placed in a dry and ventilated place for a period of time to dissipate moisture.

Otherwise the device may be damaged due to moisture.

Do not store and transport the device outside the specified environment.

Otherwise it may cause measurement error.

It is recommended that you check if there is any damage on the device or the accessories regularly, if you find any damage, stop using it, and contact the biomedical engineer of the hospital or our Customer Service immediately. Do not disassemble, repair and modify the device without permission. Otherwise it cannot be accurately measured.

This device can not be used on mobile transport platforms.

Otherwise it may cause measurement error.

This device can not be used on a tilted tabletop.

Otherwise there is a risk of falling.

Dispose of packaging materials, waste batteries and end-of-life products in accordance with local laws and regulations. The end-of-life products and materials are properly disposed of by the user in accordance with the authority's decree.

Replace accessories which not provided by our company may lead to the occurrence of errors.

Without our company or other approved maintenance organizations trained service personnel should not try to maintain the product.

This device can only be used for one test object at a time.

If the small parts on the device are inhaled or swallowed, please consult a doctor promptly.

The device and accessories are processed with allergenic materials. If you are allergic to it, stop using this product.

After pressing the power button, if the device has display fault such as white screen, blurred screen or no display content, please contact our company.

The device shall comply with the standard IEC 80601-2-30:Particular requirements for basic safety and essential performance of automated noninvasive sphygmomanometers.

#### 2.1 Operation for power adapter(Separate Sale)

🗥 Note 🗥

The device can be powered by a power adapter that is a part of the medical electrical system. Be sure to use the dedicated medical grade power adapter of this device.

Otherwise it may cause trouble

Dedicated power adapter must use AC 100 V~240 V

Otherwise it may cause fire or electric shock.

When there is breakage of dedicated power adapter plug or wire, please do not use it.

Otherwise it may cause fire or electric shock.

Please do not plug or unplug the adapter on the socket with wet hands.

Otherwise it may cause electric shock or injury.

When using the power adapter to connect with the power socket, make sure the power socket is conveniently accessible, in order to timely disconnect from the power when emergency.

#### 2.2 Operation for Battery

🗥 Note 🖄

Please use 4 "AA" size manganese or alkaline batteries, do not use batteries of other types.

Otherwise it may cause fire.

Do not mix old and new batteries and batteries of different types

Otherwise it may cause battery leakage, heat, rupture, and damage to Electronic Sphygmomanometer.

Please don't put wrong the positive and negative of battery. When the batteries power exhausts, replace with four new batteries at the same time.

Please take out the batteries when you do not use the device for a long time(3 months or more).

Otherwise it may cause battery leakage, heat, rupture, and damage to Electronic Sphygmomanometer.

If electrolyte of the batteries immodestly get in your eyes, immediately rinse with plenty of clean water.

It will cause blindness or other hazards, should immediately go to the nearest hospital for treatment.

If electrolyte of the batteries immodestly glues on the skin or the clothes, immediately rinse with plenty of clean water.

Otherwise it may hurt the skin.

#### Advice

Do not strike or drop the device;


Do not inflate before the cuff wraps around the arm;

Do not inflect the cuff and the air tube forcibly.

The device applies measurement Blood Pressure (BP) and Pulse of adult .

## **CHAPTER 3 - MAIN UNIT**

All products are in the box. Open the box and confirm whether the product is whole.

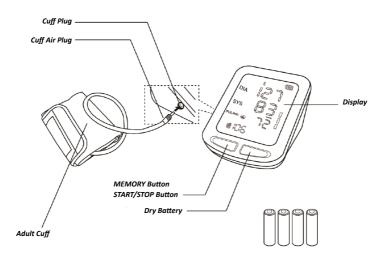



## AdultCuff: (Specification: limb circumference 22-32 cm (middle part of upper arm ), please choice suited cuff when measuring other.) User Manual Separate Sale: AC Adapter Input: voltage: AC 100 V~240 V

Extra large adult Cuff: the range of limb circumference is 32-43 cm (middle part of upper arm) Note:

frequency: 50 Hz/60 Hz Rated current: AC 150 mA Output:DC5.0 V±0.2 V 1.0 A**Cuff:** 

Ite cuff is a consumable. Calculate by measuring 6 times a day(3 times each morning and evening), the service life of the cuff is about 1 year. (using our experimental conditions);

O In order to correctly measure blood pressure, please replace the cuff in time;

© If the cuff leaks, please contact our company to buy a new one. The cuff purchased separately does not include the airway tube plug. When replacing, please do not throw the airway tube plug away, install it on the new cuff.

ANote A

When the product and accessories described in this manual are about to exceed the period of use, they must be disposed according to relevant product handling specification. If you want to know more information, please contact our company or representative organization.

## **CHAPTER 4 - EXTERNAL INTERFACES**

A Note A

When removing NIBP cuff, please take plug at the front of the windpipe to pull out.

 Cuff socket( is cuff identifier) Left side 2 Power adapter socket( G G G is power socket identifier) Back side

## 🛆 Note 🖄

All analog and digital equipment connected to this device must be certified to IEC standards(such as IEC60950: Information technology equipment-Safety and IEC60601-1: Medical electrical equipment-Safety), and all equipment should be connected to in accordance with the requirement of the valid version of the IEC60601-1-1 system standard. The person connecting the additional equipment to the signal input and output port is responsible for whether the system complies with the IEC60601-1 standard.

## CHAPTER 5 - BATTERY/AC ADAPTER INSTALLATION

- The production can use battery and AC adapter.
- 5.1 Battery Installation



Demount the battery cover in the direction of the arrow.

(2) Install "AA" batteries according to  $\oplus \ominus$  polarities.

③ Slide to close the battery cover.

Icon " \_\_\_\_": the batteries power will exhaust. Replace with four new batteries (the same sort) at the same time. Test while low power may cause data deviation and other problems.

Turn the unit off before replacing the batteries.

🛆 Note 🖄

When the battery reaches the end of its life, or if the battery is found to have odor, deformation, discoloration or distortion, stop using the battery and dispose of the used battery in accordance with local regulations, otherwise it will cause environmental pollution.

## 5.2 Usage of power adapter

1.Connect the sphygmomanometer and the power adapter. Plug the power adapter plug into the power adapter socket on the back of the device 2.Please insert the power plug of the adapter into the AC 100 V~240 V socket.

🛆 Note 🖄 🛛

The device can be disconnected from the power supply network by unplugging the adapter plug.

When cut off the power supply, first cut off the connection of power socket and the regulated power supply, then cut off the connection of regulated power supply and the sphygmomanometer.

Please be sure to use dedicated medical grade power adapter.

🖄 Note 🖄

When regulated power supply and batteries are both used at the same time, the battery power will not be consumed.

Switch regulated power supply and battery as power supply when the device is off, otherwise, the device may shutdown due to power failure. The device can be used normally after it is turned on ,without waiting for the device to be prepared.

## **CHAPTER 6 - BUTTON FUNCTIONS**

## 6.1 Description for button operation

All the operations to the Electronic Sphygmomanometer are through buttons. The names of the buttons are above them. They are:

- Enclose the section of the section
- Right button is "START/STOP" button, under "OFF" state, press this button to enter measurement mode, inflate the cuff to measure blood
  pressure, press this button again to turn off the device.

#### 6.2 Units setting

Under "OFF" state, press "M" button and "START/STOP" button simultaneously for 5 s to enter the setting interface, the default unit in this interface is "mmHg"; short press "M" button to switch the unit between "mmHg" and "kPa".

#### 6.3 Volume setting (optional for devices with voice function)

- Press "START/STOP" button again in the unit setting interface to enter the volume setting interface. Press "M" button to change the volume, the maximum volume is 4, and the minimum is 0 (silence).
- After completing the setting, repeatedly press the "START/STOP" button to turn the device off.

🛆 Note 🖄

- The default unit of the device when leaving factory is mmHg.
- In the volume setting interface, press "START/STOP" button to enter the factory setting interface, in which the "CAL" is the static pressure interface, and the "FAC" is the aging interface, which does not require user to operate. If you want to end the interface, press the "START/STOP" button twice to turn the device off.

### CHAPTER 7 - THE USAGE METHOD OF SPHYGMOMANOMETER

## 7.1 Accurate Measurement Way

#### Measurement in quiet and relaxing state.

- 1. Adopt a comfortable sitting position, use back and arms to support the body.
- 2. Place your elbow on a table, the palm faces up and the body is relaxed.
- 3. The cuff is level with your heart.
- 4. Feet flat on the floor, and do not cross your legs.

#### 🗥 Advice 🗥

Try to measure your blood pressure at the same time each day with the same arm and the same pose for consistency.

The high and low location of cuff will cause changes in measure results.

Do not touch the sphygmomanometer, cuff and windpipe during measure.

#### Measurements should be taken in a quiet place and the body relax.

Remain still 4~5 minutes before measurement.

Do not talk and movement during the measurement. Relax the body, do not let the muscle activity.

Wait 4~5 minutes between measurements

Do not use precision instrument near the Sphygmomanometer.

### 🗥 Warning 🗥

A Note A

When repeatedly measuring, the accurate blood pressure value may not be measured due to congestion in the arm. Please measure after the blood flow is smooth.

Repeated measurement for a long period of time, limbs rubbing with the cuff may be accompanied by purpura, ischemia and nerve damage. When measurement a patient, it is necessary to frequently check the color, warmth and sensitivity of the distal of the limb. Once any abnormalities are observed, place the cuff in another position or stop the blood pressure measurement immediately.

Please use the device at an environment of suitable temperature and humidity otherwise it will cause measurement error.

Do not twist or wrap the airway tube. It can cause constant pressure in the cuff which can block blood flow and cause serious damage to the patient.

Do not use the cuff on the injured area, which will cause more serious damage to the area.

Do not use the cuff in the area where the treatment is being performed inside blood vessel or the arteriovenous connection. This may cause

temporary blockage of blood flow and cause injury to the patient.

Do not use the cuff on the side of the mastectomy.

When using the cuff to pressurize, some of the body's functions may temporarily weaken. Do not use the measurement medical electrical equipment at the appropriate arm position.

Do not move during measurement, it will have a delayed effect on the patient's blood flow.

The device need to be placed for 2 hours from the minimum storage temperature to being ready for its intended use.

The device need to be placed for 4 hours from the highest storage temperature to being ready for its intended use.

#### The following conditions may also cause changes in the blood pressure measurement value.

Take the measurement in one hour after meal or after drinking alcohol, coffee or after smoking, exercise, bathing;

Using incorrect posture such as standing or lying down, etc;

The patient speak or move his body during measurement;

When measuring, the patient is nervous, excited, emotional instability;



ENGLISH

The room temperature rise or fall sharply, or the environment of measurement often changes;

Measuring in a moving vehicle;

The high and low location of cuff will cause changes in measurement results;

Continuous measurement for a long time.

## 7.2 Applying the Cuff

Both left and right arm can be measured.

Bare your arm or cloth close-fitting clothing during measurement.

Carry out the operation in a room with comfortable temperature.


#### When measuring, take the thick clothes off instead of rolling up the sleeves.

#### In order to measure accurately, pay attention to applying the cuff properly (left arm).

1 Insert the arm cuff air plug in the cuff socket of sphygmomanometer.

#### ② Stretch cuff into a barrel for the arm can conformable enter into the barrel

- ③ Left arm penetrate through the cuff, the air tube of the cuff will pass the top of your palm.
- ④ Wrap the cuff to your upper arm. Make the air tube inside the forearm and aligned with your middle finger.
- (5) The bottom of the cuff should be approximately 2cm~3cm above your elbow.
- 6 Be fixed with cloths, and wrapped tight cuff, the arm and the cuff should not have gaps.

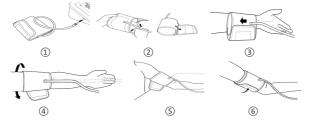

#### 7.3 Measurement BP

①Under "OFF" state, press "START/STOP" button to start measuring.

During measurement, please keep correct pose and quiet state, the body could not move. The "Movement" icon appears if patient moves, and continue measuring may lead to inaccurate measurement.

#### If you want to abort the measurement

Press [START/STOP] button, the device will stop inflating, and release the air from the cuff.

(2) Display the measurement results after finishing measuring. The pressure bar on the right side visually demonstrates the pressure level.

#### ③Confirm the Measurement Value

The World Health Organization has established globally accepted standards for the assessment of high blood pressure readings. The SYS higher than 135mmHg or the DIA lower than 85mmHg are used as the criteria of hypertension, and the pressure bar at the right side lights up in red. \*Self-diagnosis and treatment using measured results may be dangerous. Follow the instructions of your physician.

#### A Note A Wait at least 4-5 minutes between measurements.

- When repeatedly measuring, the accurate blood pressure value may not be measured due to congestion in the arm. Please measure after the blood flow is smooth.
- When the screen displays Err, the measure can't be carried out correctly.
- Irregular pulse icon is displayed in the measurement results if the pulse internal is irregular during measuring, which may cause it is unable to take
  measurement correctly. Please keep quiet and remeasure. If the irregular pulse icon appears frequently, please consult a doctor.
- The minimum value of the patient's physiological signal is the minimum limit that the device can measure. The device may obtain inaccurate
  measurement results when it is operated below the minimum amplitude or minimum value of the patient's physiological signal.

\*The device will automatically turn off after five minutes in which there is no operation to the device, even if you forget to turn the power off.

### **CHAPTER 8 - MEMORY FUNCTION**

The device can store NIBP values automatically, display up to 199 set of measurement results.

If 199 set of measurement data have been stored in current device, when saving the 200th set of data, the earliest set of data will be overwritten. If no measurement values, the memory values can be not numerated.

Memory function can not be used during measuring.

When there is no measurement values, "---" will display on the review interface.

#### 8.1 Review the Memory Value

1. Under "OFF" state, press "M" button to display the average value of the latest three set of data, when the number of measurement data is less than three groups, it will supplement automatically. Continue to press "M" button in current interface to view all measurement records.

#### 8.2 Delete Memory Values

1. Users can delete all memory values of the current user instead of separately delete one memory value

2. Under the memory interface, press "M" button and "START/STOP" button simultaneously for more than 5 s, after "DEL" appears on the screen, all memory values will be deleted.

#### A Caution A

When querying the measurement records, please press "M" button continuously to query one by one.

#### CHAPTER 9 - KEY AND SYMBOLS



Your device may not contain all the following symbols.

Signal	Description	Signal	Description
$\land$	Caution: read instructions (warnings) carefully		Follow instructions for use
SYS	Systolic pressure	DIA	Diastolic pressure
МАР	Mean blood pressure	PUL (Pressione del polso)	Pulse rate (bpm)
IPXX	Enclosure protection grade	EMC	Electromagnetic compatibility
0	Recyclable	P/N	Material code of manufacturer
LOT	Lot number	2	Use by date
<u>†</u> †	This way up	Ţ	Fragile, handle with care
Ť	Keep in a cool, dry place	\$•\$	Atmospheric pressure limit
1	Temperature limit	Ĩ	Humidity limit
<u> </u>	Manufacturer	M	Date of manufacture
	Batteries Power		Serial number
*	Flating	**	Deflating
X	WEEE disposal	Ŕ	Type BF applied part
SN	Serial number	CE	Medical Device complies with Directive 93/42/EEC
EC REP	Authorized representative in the European community		Class II applied
<b>~~</b>	Socket for power adapter		Interface for connecting cuff
٦×	Voice deactivation	()	Voice activation
<i>"</i> దొ	Great movement when measuring	(ок))	Cuff tied correctly
$\rightarrow$	Artery indicator label		Imported by

## CHAPTER 10 - ERROR MESSAGE

When the high pressure position appears "Err" and the low pressure position appears the error number, the measurement is not normal. (Error number are 02,04,06-16,19)

Error Mark	Causes	Solutions
Err02 Err15	Function abnormal	Please contact us
Err04	Low battery	Please replace the battery or link adapter
Err06	The cuff is not wrapped correctly.	Wrap the cuff correctly (refer to Chapter 10)
Err07	Cuff plug fall off	Make sure the cuff plug is securely inserted in the windpipe (refer to Chapter 10)
Err08	Air pressure error	Keep arm, body still, measure again

Err09	The pulse signal is too weak or the cuff is loose.	Wrap the cuff correctly (refer to Chapter 10)	
Err10	Cuff is blocked or squeezed	Wrap the cuff correctly (refer to Chapter 10)	
Err14	Cuff leakage	Replace with a new cuff	
Err11 Err12 Err13	The signal amplitude is too big owing to the arm or body moving or other reasons when measuring	Keep arm, body still, measure again	
Err16 Err19	It takes too much time		

## **CHAPTER 11 - TROUBLESHOOTING**

Abnormal Phenomenons	Causes	Solutions	
BP measurement	Cuff is not connected correctly.	Correctly connect cuff.	
values too high or	Talk or move arm in measurement	Keep quiet and restart a measurement.	
too low.	The turnup close oppress the arm	Take off the clothes, and restart a measurement	
	Cuff leakage	Buy a new cuff.	
No pressure	The cuff windpipe is not correctly connected with cuff	Correctly connect.	
	Cuff not inflate	Contact us.	
Cuff deflate in short time	Loose cuff Correctly tangle cuff.		
It can not carry on measurement ,even if press the measurement button		Return on the power and restart a measurement.	
Abruptly turn the power off in adding pressure	No use for a long time, the batteries can be exhausted owing to the changed temperature	Replace all four batteries with new ones.	
Hold the on/off	Batteries are worn	Replace all four batteries with new ones.	
button but can not start the device	The battery polarities is reversed	Check the battery installation for proper placement of the battery polarities.	
Cuff inflation start be	fore press the measurement button	Stop using the device and contact us.	
Cuff never deflation		Stop using the device and contact us.	
	Deflation error	Pull out the cuff to deflate. Stop using the device and contact us.	
Air pressure error	Others	Keep arm, body still, measure again.	
No press value displayed or the value unaltered when cuff inflating		Pull out the cuff to deflate. Stop using the device and contact us.	
Other phenomenon		Switch on the power once again and restart an operation. Replace the batteries. If no, please contact us.	

## **CHAPTER 12 - MAINTENANCE, CLEANING AND KEEPING**

\*Please do obey the precautions and correct operating methods in this user manual. Otherwise, we will not responsible for any fault.  $\hat{\Delta}$  Warning  $\hat{\Delta}$ 

## Remove the batteries before cleaning. The accessories and main unit must be separated for cleaning.

#### Maintenance is not allowed during device using.

Do not squeeze the rubber tube on the cuff.

## $\triangle$ Caution $\triangle$

- High pressure disinfection to the device and accessories is not allowed.
- Do not let water or cleaning agent flow into the socket to avoid device damage.
- Do not soak the device and accessories in liquid.
- If any damage or deterioration of the device and accessories is found, please do not use it.

#### Maintenance:

Clean the device and accessories regularly. It is recommended to clean them every one month. When the device or accessory gets dirty, use a dry
and soft cloth to wipe. If they are very dirty, it is available to dip the soft cloth into water or mild detergent, and wring out, then use the cloth for
cleaning.

The device shall be inspected and calibrated regularly (or according to inspection standard of hospital). The inspection can be carried out in

appointed institutions, or by professional personnel or contact us for inspection. Under the setting interface, Press the "START/STOP" button once, after "CAL" appears on the screen, press "M" button for more than 15 s to enter the static pressure interface.

## 🖄 Advice 🖄

- Do not use gasoline, volatile oil, diluent, etc. to wipe the device.
- Do not clean or wet the cuff.

Storage:

 $\triangle$  Advice  $\triangle$ 

- Do not expose the device in direct sunlight for long time, otherwise the display screen maybe damaged.
- The basic performance and safety of the device are not affected by the dust or cotton wool in home environment, while the device shall not be placed where with high temperature, humidity or dusty.
- Aged cuff may result in inaccurate measurement, please replace the cuff periodically according to the user manual.
- To avoid device damage, keep the device out the reach of children and pets.
- Avoid the device close to extreme high temperature such as fireplace, otherwise the device performance may be affected.
- Do not store the device with chemical medicine or corrosive gas.
- Do not place the device where there is water.
- Do not place the device where with slope, vibration or impact
- Take the batteries out if the device is not to be used for three months or longer.

### CHAPTER 13 - NIBP SPECIFICATION

Name	Electronic Sphygmomanometer		
The degree of protection against ingress of water	IPX1		
Display	LED		
NIBP Specifications	l		
Measurement Method	Oscillometric meth	od	
Working mode	Automatic		
Operation mode	Continuous operat	ion	
Pressure Range	0~297 mmHg(0~39	9.6 kPa)	
Measurement range	Pressure	SYS: 30∼270 mmHg(4∼36 kPa) DIA: 10∼220 mmHg(1.3∼29.3 kPa)	
	Pulse: 40~240/min		
Inflation	160±5 mmHg(21.3	3±0.67 kPa)	
Overpressure protect	297±3 mmHg(39.6	±0.4 kPa)	
Resolution	Pressure: 1 mmHg	(0.133 kPa)	
Accuracy	Static pressure: ±3	mmHg(±0.4 kPa)	
Error	The BP value measured by the device is equivalent with the measurement value of Stethoscopy, perform clinical verification in accordance with the requirements in ISO 81060-2: 2013, whose error meets the followings: Maximum mean error: ±5 mmHg Maximum Standard deviation: 8 mmHg		
Operating Temperature/ Humidity	+5ºC~40 ºC . 15%RH~85%RH(no condensation)		
Transport	Transport by general vehicle or according to the order contract, avoid pounded, shake and splash by rain and snow in transportation.		
Storage	Temperature: -20 ºC~+55 ºC; Relative humidity: ≤95 %; No corrosive gas and drafty.		
Atmospheric pressure	700 hPa~1060 hPa		
Power supply	4 "AA" alkaline batteries, AC Adapter(AC, 100 V-240 V, optional)		
Rated current	≤ 600 mA		
Battery life	When the temperature is 23 °C, limb circumference is 270 mm, the measured blood pressure is normal, 4 "AA" alkaline batteries cab be used about 300 times.		
Main Unit Dimensions	129*101*72 mm		
Main Unit Weight	281 gram(without batteries)		
Safety classification	Class II equipment (power supplied by power adapter)/Internally powered equipment (power supplied by batteries) Type BF applied part		
Service life	The service life of the device is five years or 10000 times of BP measurement.		
Date of manufacturer See the label			



Accessories	Standard Configure: Adult Cuff: link of counterence 22-32 cm (upper arm center)	
	User Manual, four "AA" alkaline batteries AC Adapter:	
	Input: voltage: AC 100 V~240 V frequency: 50 Hz/60 H Rated current: AC 150 mA Output: DC 5.0 V $\pm$ 0.2 V 1.0 A	

#### APPENDIX

#### Guidance and manufacturer's declaration – electromagnetic emissions-for all EQUIPMENT and SYSTEMS

#### Guidance and manufacturer's declaration – electromagnetic emission

The device is intended for use in the electromagnetic environment specified below. The customer of the user of the device should assure that it is used in such and environment.

Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B The device is suitable for use in all establishments,	
Harmonic emissions IEC61000-3-2	Class A	domestic establishments and those directly connected to public low-voltage power supply network that supplies buildi
Voltage fluctuations/ flicker emissions IEC61000-3-3	Complies	used for domestic purposes.

## Guidance and manufacturer's declaration - electromagnetic immunity -for all EQUIPMENT and SYSTEMS

#### Guidance and manufacturer's declaration - electromagnetic immunity The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment. Electromagnetic environment -Immunity test IEC 60601 test level **Compliance level** guidance Floors should be wood, concrete Electrostatic discharge or ceramic tile. If floor are covered ±6 kV contact ±6 kV contact with synthetic material, the (ESD) ±8 kV air ±8 kV air relative humidity should be at IEC 61000-4-2 least 30% Mains power quality should be Electrical fast transient/ that of a typical commercial or burst ±2 kV for power supply lines ±2kV for power supply lines hospital environment. IEC 61000-4-4 Mains power quality should be ±1 kV differential mode ±1 kV differential mode Surge that of a typical commercial or IEC 61000-4-5 ±2 kV common mode ±2 kV common mode hospital environment. <5% U\_ (>95% dip in U\_)for <5% U, (>95% dip in U,)for 0.5 cycle 0.5 cycle Mains power quality should be 40% U\_(60% dip in U\_)for 5 40% U\_(60% dip in U\_)for Voltage dips, short that of a typical commercial or interruptions and voltage 5 cycles cycles hospital environment. The device variations on power supply can continue the operation during 70% U\_(30% dip in U\_)for 25 70% U\_(30% dip in U\_)for input lines power mains interruptions due to IEC 61000-4-11 cycles 25 cycles the usage of battery. <5% U\_ (>95% dip in U\_)for <5% U\_ (>95% dip in U\_) 5 sec for 5 sec Mains power quality should be Power frequency (50/60Hz) that of a typical commercial or magnetic field 3A/m 3A/m hospital environment. IEC61000-4-8 NOTE: U<sub>r</sub> is the a.c. mains voltage prior to application of the test level.

Guidance and manufacturer's declaration - electromagnetic immunity -for EQUIPMENT and SYSTEMS that are not LIFE-SUPPORTING

Guidance and manufacturer's declaration – electromagnetic immunity				
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.				
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance	
			Portable and mobile RF communications equipment should be used no closer to any part of the <i>device</i> , including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.	
Conducted RF IEC61000-4-6 Radiated RF IEC61000-4-3	3 V/m 150 kHz to 80 MHz 3 V/m 80 MHz to 2.5 GHz	3 V/m 3 V/m	<b>Recommended separation distance</b> $d = \left[\frac{3.5}{E_i}\right]\sqrt{P}$ 80 MHz to 800 MHz $d = \left[\frac{3.5}{F_i}\right]\sqrt{P}$ 80 MHz to 2.5 GHz $d = \left[\frac{7}{E_i}\right]\sqrt{P}$ 800 MHz to 2.5 GHz Where <i>P</i> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <i>d</i> is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup> Interference may occur in the vicinity of equipment marked with the following symbol: (((*))	
NOTE 1     At 80 MHz and 800 MHz, the higher frequency range applies.       NOTE 2     These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.				
A Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device. B Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.				

# Recommended separation distances between portable and mobile RF communications equipment and the EQUIPMENT or SYSTEM – for EQUIPMENT or SYSTEM that are not LIFE-SUPPORTING

Recommended separation distances between portable and mobile RF communications equipment and the device

The device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum	Separation distance according to frequency of transmitter (m)			
output power of transmitter (W)	150 kHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2.5 GHz	
0.01	0.12	0.12	0.23	
0.1	0.37	0.37	0.74	
1	1.17	1.17	2.33	
10	3.69	3.69	7.38	
100	11.67	11.67	23.33	

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

ENGLISH

At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. NOTE 1

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

## A Warning

- Active medical devices are subject to special EMC precautions and they must be installed and used in accordance with these guidelines.
- Electromagnetic fields can affect the performance of the device, so other equipment used near the equipment must meet the appropriate EMC requirements. Mobile phones, X-rays, or MRI devices are possible interference sources, as they emit high-intensity electromagnetic radiation.
- The use of ACCESSORIES, transducers and cables other than those specified, with the exception of transducers and cables sold by the MANUFACTURER of the the device as replacement parts for internal components, may result in increased EMISSIONS or decreased IMMUNITY of the ME EQUIPMENT or ME SYSTEM.
- The device should not be used when they are close to or stacked with other equipment, if necessary, please observe and verify that they can operate normally in the configurations.
- Devices or systems may still be interfered by other equipment, even if other equipment meets the requirements of the corresponding national standard
- The device requires special precautions for electromagnetic compatibility (EMC) and requires gualified personnel to install and use in accordance with the EMC information provided below.
- The device should not contact the pins of connectors marked with an ESD warning symbol, unless electrostatic discharge precautions are used, the device should not connect to these connectors.
- In order to avoid the accumulation of electrostatic charge, it is recommended to store, maintain and use the equipment at a relative humidity of 30 % or more. The floor should be covered with ESD dissipated carpets or similar materials. In the use of the components, non-synthetic clothing should be wore.
- In order to prevent electrostatic discharging to the ESD-sensitive parts of the device, the personnel should contact the metal frame of the components or the large metal objects near the device. When using the device, especially when it is possible to contact the ESD-sensitive parts of the device, the operator should wear a grounded bracelet designed for ESD-sensitive devices. For more information on proper use, please refer to the instructions provided with the bracelet.
- All potential users are advised to understand the ESD warning symbols and receive training on ESD precautions.
- The most basic content of the ESD precautionary procedure training should include an introduction to electrostatic charge physics, voltage level in the conventional case, and damage to the electronic components when the operator with electrostatic charge contacts them. In addition, the methods for preventing electrostatic buildup, and the manner and reasons for the release of human body static electricity to the ground or equipment frame or the use of a bracelet to connect the human body to the equipment or the ground before establishing the connection should he described

The following cable types must be used to ensure that they comply with interference radiation and immunity standards:

Name	Length (m)	
Power adapter cable	1.5	

### GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty



Disposal: The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment