ES	EJERCITADOR RESPIRATORIO	TRI-BALL®
EN	RESPIRATORY EXERCISER	TRI-BALL®
FR	EXERCISEUR RESPIROAIRE	TRI-BALL®
DE	ATEMTRAINER	TRI-BALL®
IT	APPARECCHIO RESPIRATORIO	TRI-BALL®
PT	EXERCITANTE RESPIRATÓRIO	TRI-BALL®
NL	ADEMHALINGSTRAINER	TRI-BALL®
CS	TRENAŽÉR DÝCHÁNÍ	TRI-BALL®
HR	APARAT ZA VJEŽBE DISANJA	TRI-BALL®
HU	LÉGZÉSTRÉNER	TRI-BALL®
PL	URZĄDZENIE DO ĆWICZEŃ ODDECHOWYCH	TRI-BALL®
SL	VADBENI RESPIRATOR	TRI-BALL®
LT	KVĖPAVIMO TRENIUKOLIS	TRI-BALL®
AR	ممرن الجهاز التنفسي	TRI-BALL®
MS	ALAT SENAMAN PERNAFASAN	TRI-BALL®

					
<p>PECMED Lot 14, Industrial Zone No. 3 Djebel Ouest 1141 Bir Mchergua Zaghuan, Tunisia</p>					
					
<p>Patient Guard Ltd: Lancaster House, Amy Johnson Way, Blackpool, Lancashire FY4 2RP, United Kingdom</p>		<p>37 Jeanette Pontelle Street 69550 Amplepuis France</p>			

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DESCRIPTION.
The TRI-BALL® respiratory exerciser is a medical device that comes into contact with the mucous membrane of the body surface area (mouth). The system consists of a mouthpiece, an inhalation tube and a casing with three independent chambers and their respective inspiration flows, with a different coloured ball in each chamber. The casing also features a holder in which to place the mouthpiece.

INDICATIONS FOR USE.
The TRI-BALL® respiratory exerciser is designed to assist patients who have undergone abdominal or thoracic intervention to regain normal breathing. Deep breathing exercises play an important role in the recovery of these patients. The device helps patients to regain their normal breathing patterns sooner and reduces the risk of respiratory complications. The product is indicated for both adult and paediatric patients of over 8 years of age.

CONTRAINDICATIONS.
The TRI-BALL® respiratory exerciser is not indicated for:

- Patients who cannot be trained to use the device correctly or monitored.
- Patients with a physical disability or patients who are unable to understand or demonstrate the correct use of the device.
- Patients who cannot breathe deeply and effectively due to pain, diaphragm dysfunction or opioid analgesia.
- Patients with respiratory muscular fatigue who are unable to generate an adequate inspiration with a vital capacity of <10 ml/kg or an inspiratory capacity of <33% of normal predicted.

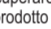
INSTRUCTIONS FOR USE.
Warnings:
A1. Do not use this device in the event the product is damaged or any of its parts are broken, as this may result in the product malfunctioning in the worst case scenario.
A2. This use of this product is restricted to a single patient.
A3. The product is valid for 30 days or 1,000 cycles of use.
Assembling the product:
1. Remove TRI-BALL® from its bag and the tube from its case (A).
2. Connect the open end of tube to the hole (B).
Instructions for use:
1. Prior to beginning your exercise, breathe out normally and then place the mouthpiece in your mouth, wrapping your lips tightly around it.
2. To achieve a low inspiratory flow rate, inhale until the first ball rises. Maintain this position for as long as possible. The second and third balls should remain in their initial position.
3. To obtain a medium inspiratory flow rate, inhale until the first and second balls have risen to the top of their chambers. The third ball should remain in its initial position.
4. To obtain a maximum inspiratory flow rate, inhale deeply until all three balls have risen to the top of their chambers.
5. Release the mouthpiece and exhale.
6. Rest for a moment breathing normally.
7. Repeat the exercise as instructed by the physician.

Precautions:
P1. Wash the mouthpiece prior to use.
P2. The patient should place the TRI-BALL® device in an upright position at face level whenever using the respiratory exerciser to ensure it functions correctly.
P3. If the respiratory exerciser is dropped on the floor accidentally or intentionally, do not reuse the product as it may be broken, which could result in the product malfunctioning in the worst case scenario.
P4. If you have any questions on how the device works or if you are feeling unwell, talk to your doctor or therapist.

CLEANING AND STORAGE INSTRUCTIONS.
Warnings:
A1. Do not use this device in the event the product is damaged or any of its parts are broken, as this may result in the product malfunctioning in the worst case scenario.
Precautions:
P1. Do not store in moist environments.

SAFE DISPOSAL OF THE PRODUCT.
1. Dispose of the product in accordance with the healthcare facility's protocol or healthcare personnel instructions. Any serious product-related incident should be reported to the manufacturer and the competent authority.

CLARIFICATION ON SYMBOLS.
The symbols used on the TRI-BALL® label comply with ISO15223-1: 2021. The following additional symbols have been used:

	Room number.
---	--------------

Additional copies of the instructions for use may be requested to PECMED free of charge.

IT	APPARECCHIO RESPIRATORIO	TRI-BALL®
-----------	--------------------------	------------------

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