GARANZIA

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WARRANTY

This tripod has **2-YEAR** warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In the case you need to get your tripod repaired and you fulfil the conditions of warranty, send the tripod to the nearest retailer together with the purchase receipt.

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Acquistato in data / Purchasing date _____

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• RP720
Tripode manico
aperto

• RP721
Tripode manico
chiuso

I class medical device
according to ECC directive 93/42

Code RP715-719-720-721

Thank you for purchasing tripod large base, **MOPEDIA** by Moretti tree-leg cane to support people with movement difficulty. It has anodized, light weight aluminium structure. It is supplied with non-slip soft rubber tips. It is height adjustable every 2.5 cm:

- **RP715** from 66 to 73 cm
- **RP719** from 76 to 98 cm
- **RP720** from 75 to 97 cm
- **RP721** from 74 to 97 cm

To adjust leg height:

To adjust the height, press the push button and slide the cane to the desired height. Be sure the push button is fully protruded through the desired hole when finished. Maximum endurable weight 130kg.

To adjust tripod for right or left hand use:

(Note: flat side of the base to outside of foot)

Press the push button and rotate handle by 180 degrees. Be sure the push button is fully protruded through the desired hole on the other side of the cane.

ATTENTION: USE EXTRA CAUTION ON WET AND SLIPPERY SURFACES.

Avoid to leave the tripod in contact with water for long periods of time and control its sturdiness periodically. Keep it away from children to avoid any possible risks. Replace rubber tips every time they are worn out.

• RP715
Three-leg cane
child type

• RP719
Three-leg cane with
brachium rest

• RP720
Three-leg cane
open handle

• RP721
Three-leg cane
closed handle