
HEINE BETA 200® / K180® Otoskop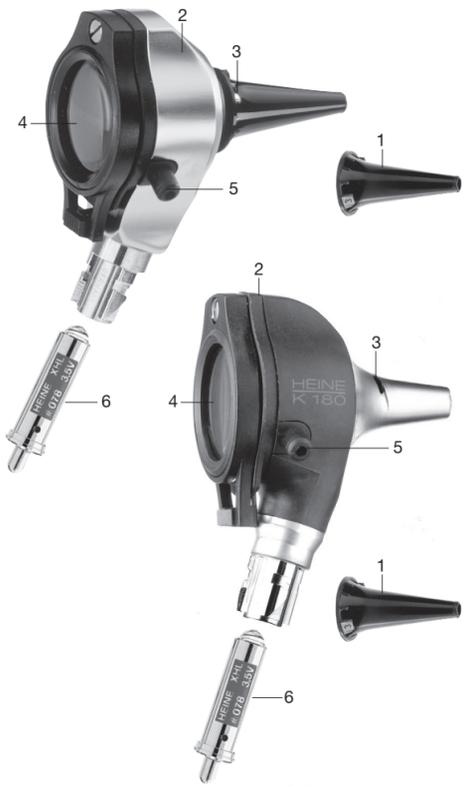

CE 93/42/EWG/CEE/EEC

HEINE
 HEINE Optotechnik GmbH & Co. KG
 Kientalstr. 7 · 82211 Herrsching · Germany
 Tel. +49(0)81 52/38-0 · Fax +49(0)81 52/38-202
 E-Mail: info@heine.com · www.heine.com
 med 9396 1/1.07

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DEUTSCH

HEINE BETA 200® / K 180® Otoskop

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ENGLISH

HEINE BETA 200® / K 180® Otokosope

Before using the Otokosope read these instructions carefully and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety information

Range of applications: The HEINE BETA 200® and HEINE K 180® Otokosope is used for the examination of the external ear and for general non-invasive examination.

Safety in use: Use only handles or power sources which are specifically designed for medical use. The Otokosope should only be inserted into the ear when a tip or speculum is fitted. Pressure should be applied with great care when doing pneumatic testing.

Please note: The performance of this instrument can only be guaranteed if genuine HEINE rechargeable batteries or alkali manganese batteries are used.

Using the instrument

The otoscope should only be introduced into the ear when a SANALON S or AllSpec tip is fitted.

- Push the tip (1) onto the instrument head (2) with a rotary movement so that the projection inside the tip engages in the slit (3) and rotate clockwise until it fits firmly.
- With the lens window (4) closed, a magnification of ca. 3x is available for routine examinations.
- A bellows can be attached to the insufflation nipple (5) for pneumatic testing.
- The lens window (4) can be swivelled to facilitate instrumentation under magnification.

General illumination: The HEINE BETA 200® and K 180® otoscope can be used with or without speculum or tip for general illumination of e.g. the oral cavity or skin. Avoid contact with sensitive or damaged skin.

Changing the bulb

Please note: The performance of this unit can only be guaranteed if genuine HEINE XHL® Xenon Halogen spare bulbs are used.

- Simply detach the otoscope from the handle and pull out the bulb (6).
- Push in the new bulb as far as it will go.

Cleaning and Sterilization

Otokosope head: The outside can be cleaned with a soft cloth, the inside with a cotton-wool bud moistened with alcohol. Gas sterilization is permitted. The instrument should not be soaked.

Reusable SANALON S tips can be cleaned with commercially – available solutions, disinfected or boiled. Sterilization at up to 170° C is permitted.

AllSpec tips and disposable specula are for single use only. Do not attempt to clean them as the tip/specula may be damaged and could injure the patient.

Accessories and spare parts

Bellows	B-00.11.240
Reusable SANALON S tips	
1 Set (4 pcs.)	B-00.11.111
AllSpec disposable tips	
Pack of 1000 pcs.	
distal dia 2,5 mm	B-00.11.128
same, dia 4 mm	B-00.11.127
AllSpec Tip-Dispenser	B-00.11.148
HEINE XHL® Xenon Halogen spare bulb 2,5 V	X-01.88.077
HEINE XHL® Xenon Halogen spare bulb 3,5 V	X-02.88.078

Warranty

We guarantee this device (except bulbs) for five years from date of purchase. We guarantee no warranty if the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product, the buyer must prove that any such defect was present at the time the product was delivered. This guarantee excludes any defects which occur because of wear and tear, incorrect use, the use of non-original HEINE parts, (especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance). Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

Bought from	
date	

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