UnionTest

LE1xx_{rev.001} - LE2xx_{rev.001} - LE3xx_{rev.001} TEST COUCHES / LETTINI TEST

Examination, diagnostic and treatment couches for use in physical therapy
Lettini da visita diagnosi e trattamenti per terapia fisica

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ENGLISH VERSION..... 4

1 GENERAL INFORMATIONS..... 4

1.1 PURPOSE, CONTENT AND TARGET AUDIENCE..... 4

1.2 MANUFACTURER 4

1.3 INTENDED USE AND EQUIPMENT DESCRIPTION 4

1.4 APPLICATION ENVIRONMENT 4

1.5 PRODUCTS INSIGHTS..... 4

1.6 STORAGE 4

1.7 SYMBOLS..... 4

2 SPECIAL WARNINGS AND USE RESTRICTIONS..... 5

2.1 COLLATERAL EFFECTS AND CONTRAINDICATIONS 5

3 CONFIGURATED ITEM CODE 6

4 USE..... 6

4.1 UNPACKING 6

4.2 STARTING 6

4.3 HEIGHT ADJUSTMENT 6

4.3.1 Configuration A (electric with foot switch) 6

4.3.2 Configuration C (hydraulic, pedal-operated). 7

4.3.3 Configuration D (electric, w/ hand control) 7

4.3.4 Configuration E (electric, w/ foot switch for 2 motors) 7

4.3.5 Configuration F (electric, w/ hand-control, 2 motors)..... 7

4.3.6 Configuration G (Ergonomic multipurpose control) 7

4.4 EQUIPMENT 8

4.4.1 With wheel kit 8

4.5 TRANSFER WHEELS FOR MODEL MOBIL LE211 8

4.6 GAS-SPRING ASSISTED COUCH SECTION ADJUSTMENTS 8

4.6.1 Back section and footrest section 8

5 MAINTENANCE 10

5.1 ROUTINE MAINTENANCE..... 10

5.2 NON ROUTINE MAINTENANCE..... 10

5.3 PREVENTIVE OR PLANNED MAINTENANCE 10

5.4 SPARE PARTS 10

6 TROUBLESHOOTING 11

7 CLEANING AND DISINFECTION 12

7.1 WASHING..... 12

7.2 DISINFECTION 12

7.2.1 Surface disinfection 12

8 REUSE OF THE PRODUCT 12

9 TECHNICAL SHEET 13

10 ELECTRIC DIAGRAM..... 13

11 RELATED ITEMS AND ACCESSORIES..... 14

12 WARRANTY 15

13 DISPOSAL OF WASTE ELECTRICAL OR ELECTRONIC EQUIPMENT AND PACKAGING 15

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ENGLISH VERSION

1 GENERAL INFORMATIONS

1.1 Purpose, content and target audience

This manual is intended to provide the necessary information to permit a proper use of the equipment and to be able to use it independently and safely.

This manual contains information relating to the technical, operation, maintenance, spare parts and safety. This book is also intended to technicians in charge of maintenance equipment.

This Instruction Booklet should always follow the equipment and must be carefully preserved in order not to compromise its readability.

In case of transfer to third parties this manual has to be transmitted with other documents as required by European Directive 93/42 EEC.

1.2 Manufacturer

CHINESPORT S.P.A, Via Croazia, 2-33100 Udine - Italy

tel. +39 0432621621–fax+39 0432621620 – website: www.chinesport.com.

The company produces according a quality system by standard UNI EN ISO 13485:2012.

1.3 Intended use and equipment description

Examination, diagnostic and treatment couches. Height adjustable, by means of a low-voltage controlled electric actuator, or by means of an hydraulic piston (except models with 2 electric actuators). Gas spring assisted back and footrest section. Simultaneous electrical adjustment of seat/back sections on models LE3xx to assume Trendelenburg or "chair style" positions.

1.4 Application Environment

Medical devices covered by this manual are in conformity according to attached VII of the directive 93/42/EEC amended by 2007/47/CE

In particular according to UNI CEI EN 60601-2-52:

ENVIRONMENT OF APPLICATION 5 - outpatient care administered in a hospital or other medical center under medical supervision, in which the EM devices are provided for the needs of sick people who have suffered injuries or are carriers of disability, for the treatment, diagnosis and monitoring.

1.5 Products insights

Further documentation can be found on our website www.chinesport.com.

Specially, you can reading the QR code on the cover of this manual to quickly connect to the website.

Where provided QR code can also be found on the product.

You can download declaration of conformity and other product documents in their latest version and in color format directly from website.

1.6 Storage

The following environmental conditions must be ensured when the equipment is to be stored:


1. relative humidity 10% ÷ 90%;
2. temperature -10°C ÷ +50°C.

1.7 Symbols

The symbols found in this manual and on the machine are used to draw the user's attention to important safety warnings and useful information. Observe all warning, caution and danger messages.

	<p>WARNING</p> <p>This symbol identifies operating/maintenance instructions or procedures which, if not observed, could result in injury or danger to the user.</p>
	<p>ELECTRIC SHOCK HAZARD</p> <p>This symbol identifies operating/maintenance instructions or procedures which, if not observed, could result in a risk of electric shock.</p>
	<p>CAUTION</p> <p>This symbol is used to warn the user that not to observe a certain operating or maintenance procedure, general rule or condition, could result in damage to the equipment.</p>

2 SPECIAL WARNINGS AND USE RESTRICTIONS

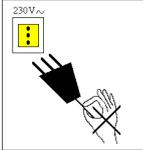	<p>This equipment must not be used:</p> <ul style="list-style-type: none"> • By inadequately trained operators. • When having been tampered with and/or if the original product has undergone changes. • On unstable and/or uneven surfaces (e.g.: stairway, steps, etc.). • With loads greater than those declared to be safe by the manufacturer. • In an improper configuration or when it is thought to be harmful for the patient. • In association with other devices that may interfere with the equipment. • In an explosive and/or corrosive atmosphere or exposed weather conditions. <p>In addition:</p> <ul style="list-style-type: none"> • Keep body parts away from moving parts and/or adjustments. These areas are marked by labels. • Three, four or more people are required to carry the equipment. • Set the bed in its lower and horizontal position when it is not supervised.
	<ul style="list-style-type: none"> • Do not connect to an electric system that does not comply with the current safety regulations. • Do not spill liquids on the product. • Disconnect power supply before moving and carrying out any service on the equipment circuits. • Do not damage electrical cables: preventing crushing; not disconnect the connections pulling on the cable directly, etc. 
	<ul style="list-style-type: none"> • Do not use the lifting system continuously, comply with the duty-cycle indicated in chapter "Technical sheet". • The foam rubber and plastic parts of the equipment can catch fire if brought into contact with naked flames. • The retractable wheels can be used to move the couch without any added weight. • Radiating therapy lamps can damage the upholstery. • Corrosive and/or abrasive detergents can damage the upholstery. • Make sure that the patient cannot accidentally operate the controls. • Make sure that during equipment operation any moving parts do not knock against adjacent masses. • Keep a proper distance from moving mechanical parts. • Periodically disinfect the table's leatherette. • Record any significant safety-related intervention/defect/damage in the attached form. • For operations not covered in the maintenance list, please contact Chinesport Service Centre. • Only the manufacturer can repair the actuators, gearcases, controls and battery chargers, which on no account must be opened or tampered with. Otherwise the manufacturer will be no longer responsible for the equipment, whose warranty will cease to be valid. • When safety is clearly in danger, immediately switch off the equipment and do not use it until the danger has been eliminated.

2.1 Collateral effects and contraindications

There is no collateral effects or contraindications during proper use, is recommended knowledge of this instruction manual.

3 CONFIGURATED ITEM CODE

Product code		L	E	*	*	*	*	*	*	
Variable code position		1	2	3	4	5	6	7	8	9
Position	Meaning variable code	Possible values								
1-2		Fixed value								
3	MODEL	1 Test-1 motor 2 Test Mobil -1 motor with transfer wheels 3 Test Plus – 2 motors								
4	TYPE AND NUMBER OF SECTIONS	1 model 3 sections								
		2 model 3 sections wide								
		3 3-section model for echo doppler								
		4 4-sections model specific for podiatry								
		5 3-sections model for gynecology and urology								
5		Fixed value								
6	HEIGHT ADJUSTMENT	A Electric with pedals								
		C Hydraulic								
		D Electric with hand control								
		E Pedal control 2 motors								
		F Electric with hand control motors								
		G Ergonomic multipurpose control								
7	WHEELS	1 No options 2 Transfer wheels								
8 - 9	TYPE AND COLOR COATING	REFERENCE TO CATALOGUE								

4 USE

4.1 Unpacking

The product is delivered already assembled. Then simply remove it from the package and proceed as provided in the following paragraphs.

4.2 Starting

The bed is delivered packaged to protect it from possible damage that may occur during transport. **Never use cutters to open box to prevent padding damages.**

After unpacking, remove the protective packaging materials. See chapter “Disposal of waste electrical or electronic equipment and packaging” for disposal instructions.

The carrier and the supplier must be immediately notified of any damage that may have occurred during transport. Damaged equipment cannot be used.

WARNING

LE131 TEST ECODOP – LE151 UROGYN – LE351 PLUS UROGYN: for packaging purposes, the equipment is supplied with the back support arm and the leg supports under the frame or disassembled. Position and fasten them for normal use.

4.3 Height adjustment

Depending on the configuration you choose you may submit the following types of control up-down of the bed.

4.3.1 Configuration A (electric with foot switch)

To raise the couch, press your foot on the side of the foot switch marked by the “up” arrow. Release at desired height. To lower, press the other side.

4.3.2 Configuration C (hydraulic, pedal-operated).

Press either of the two pedals to pump and raise the couch up to desired height; to lower, lift the pedal with the back of your foot.

4.3.3 Configuration D (electric, w/ hand control)

To raise, press the pushbutton indicated by the arrow and release at desired height; press the other button to lower.

4.3.4 Configuration E (electric, w/ foot switch for 2 motors)

Standard configuration on models LE311- LE341 -LE351: using your foot, press the switch indicated by the arrow in the foot switch section that controls the movement desired (raising-lowering of board or tilt –see photo-). Release at desired height. Press the other switch to reverse the movement.

4.3.5 Configuration F (electric, w/ hand-control, 2 motors)

To raise, press the pushbutton indicated by the arrow and release at desired height; press the other button to lower.
To tilt, press the pushbutton indicated by the arrow and release at desired tilt; press the other button to lower.

4.3.6 Configuration G (Ergonomic multipurpose control)

Use with foot when fixed on the bracket or with hand and attached to frame with magnet. Press on the side indicated by + to control the raising and release the desired height is reached; Using your hand or your foot, press on the side indicated by - to control the lowering and release the desired height is reached.

WARNING

WHILE MAKING UP/DOWN/TILT ADJUSTMENTS, LOOK OUT FOR ADJACENT OBSTACLES THAT MAY INTERFERE WITH PANELS

4.4 Equipment

4.4.1 With wheel kit


To engage the retractable casters, use your foot sole to push the lower PEDAL of the device until it stops; to disengage, lift the small upper bar of the pedal with the back of your foot.

WARNING

THE RETRACTABLE CASTERS MAKE IT POSSIBLE TO MOVE THE COUCH ONLY, WITHOUT ADDITIONAL LOADS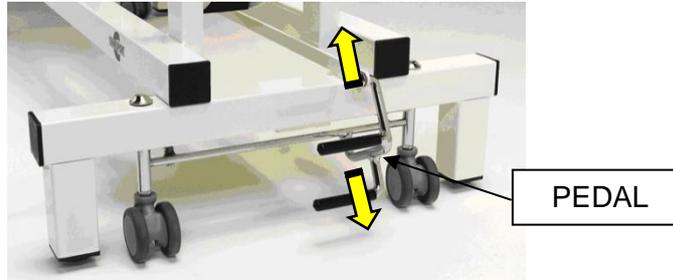

4.5 Transfer wheels for model Mobil LE211

The Mobil model is fitted with four casters: one with directional lock and three with total-lock brake (rotational and swivel lock).

BRAKES AND DIRECTIONAL LOCK

Caster with directional lock (blue) engaged

Caster with parking brake (grey) engaged

The caster brakes and lock are foot-operated.

- **Blue** lever down: directional lock ON. When the couch is moved, the wheel will position itself on the feed axis, automatically activating the directional lock. (The lever of the wheel can have different color from blue, but still different from that of the other 3 wheels)

- **Gray** lever down: parking brake ON. (It stops the rotation of the wheel and the swivel) - (The levers can have different colors from gray, but the same for all 3 wheels) ..

WARNING

- ALWAYS APPLY THE PARKING BRAKES AFTER COMPLETING THE MOVEMENT
- DIRECTIONAL LOCKING HAS NO BRAKING EFFECT: TO LOCK THE COUCH, USE PARKING BRAKES
- DO NOT PUSH THE COUCH SIDEWAYS WITH THE DIRECTIONAL LOCK ENGAGED

4.6 Gas-spring assisted couch section adjustments


4.6.1 Back section and footrest section

Press the gas spring specific control lever and adjust the section tilt as desired; release the lever to hold the position.

MODEL	DIMENSIONS OF INDIVIDUAL BOARD AND POSTURES	ADJUSTMENT OPERATION	FURTHER DEVICES
LE111 LE151 LE211			 LE151 leg supports are height- and tilt-adjustable; to hold them in position, tighten the lever of the locking devices.
LE121			
LE141			
LE131			The aperture suitable for doppler ultrasound tests can be opened by lifting the handle located under the removable tray. To close, rotate the removable tray until it safely clicks back in position.
LE311 LE351	 		 LE351 leg supports are height- and tilt-adjustable; to hold them in position, tighten the lever of the locking devices.
LE341	 		

⚠ WARNING ⚠

- DO NOT SIT AND DO NOT CONCENTRATE LOAD ON THE COUCH TERMINAL ENDS
- MAKE SURE TO SUPPORT THE ENTIRE LOAD AS YOU OPERATE THE ADJUSTMENT LEVERS WHILE THE PATIENT IS ON THE COUCH
- MOVEMENT OF THE ADJUSTABLE SECTIONS CAN CAUSE PINCHING IN THE VICINITY OF THE HINGES

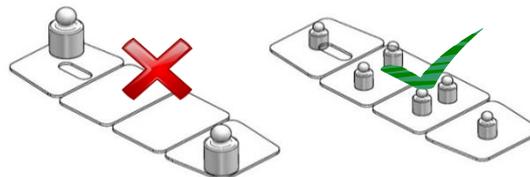5 MAINTENANCE

Only qualified personnel may carry out maintenance on the equipment. Technicians must be familiar with the model, use and maintenance of the equipment.

Check the equipment whenever it has undergone a "rough treatment" (e.g.: fall, liquid spill, etc.) and whenever any doubts arise concerning its safety.

5.1 Routine maintenance

Routine maintenance must be carried out at least once every 12 months, following the table below.

WARNING

- WHEN SAFETY IS AT RISK, IMMEDIATELY TURN OFF THE EQUIPMENT AND DO NOT USE IT UNTIL THE DANGER HAS BEEN ELIMINATED
- ANY SIGNIFICANT WORK CARRIED OUT TO ENSURE THE COUCH IS SAFE FOR USE MUST BE RECORDED IN THE RELEVANT FORM (ATTACHED)

FREQUENCY	OPERATION
yearly	General cleaning of equipment
	Visual and functional inspection of the equipment
	Frame and mechanical parts: check screws and bolts to ensure they are tight; pin and bushing clearance; check for any damage or deformation.
	Check that the adjustable sections are stable and do not collapse when the control lever is released
	Check spring control cables and respective sheaths
	Check the wheels are firmly secured and free to rotate; clean the tread
	Check control box, actuators and controls to see if connectors are fully plugged in
	Check insulation on electrical cables for damage
Make sure that during electric operation no unusual vibrations or sounds are produced that may signal defects in the actuators and/ or mechanisms.	
AFTER THE ABOVE OPERATIONS, COMPLETE INSPECTION BY CARRYING OUT A FULL LIFTING CYCLE USING THE MAXIMUM SAFETY WORKING LOAD.	

WARNING

- FOR OPERATIONS NOT COVERED IN THE MAINTENANCE LIST, PLEASE CONTACT CHINESPORT SERVICE CENTRE
- ONLY THE MANUFACTURER CAN REPAIR THE ACTUATORS, GEARCASES, CONTROLS AND BATTERY CHARGERS, WHICH ON NO ACCOUNT MUST BE OPENED OR TAMPERED WITH. OTHERWISE THE MANUFACTURER WILL BE NO LONGER RESPONSIBLE FOR THE EQUIPMENT, WHOSE WARRANTY WILL CEASE TO BE VALID

5.2 Non routine maintenance

Contact Chinesport service for all operations not mentioned in the maintenance list.

5.3 Preventive or planned maintenance

It's recommended a complete overhaul of the equipment every 3 years, performed by Chinesport or by an authorized service center.

5.4 Spare parts

Only use original spare parts. Contact CHINESPORT S.p.a. for the updated spare parts list.

6 TROUBLESHOOTING


The equipment has undergone heavy-duty tests; no significant breaks and/or failures were reported. If the equipment does not work, refer to the troubleshooting table below.

Problem	Cause	Solution
ELECTRICAL HEIGHT ADJUSTMENT		
Lifting does not work	Connectors either not connected or not properly plugged in	Check connections
	Overload	Observe safe working load
	No power supply	Check connection to mains power
	Safety fuse burned out	Contact the assistance service
	Control box, control or electric actuator is broken	Contact the assistance service
HYDRAULIC HEIGHT ADJUSTMENT - ADJUSTMENT OF SECTIONS		
Lift not working	Hydraulic piston broken	Contact the assistance service
Section adjustment not working	Gas spring broken or needs calibration	Contact the assistance service
	Transmission cable broken	

In case of troubleshooting table not solve your fail, you can open a service ticket means website www.chinesport.com customer service section.

ATTENTION:

Before calling our technical support, be sure to have your **SERIAL NUMBER** or the “Codice” and “Articolo” to hand: you can find it on the product label located on the equipment (as shown in the facsimile above); or the transport document / invoice number. This identification number helps reduce waiting times and improves service quality.

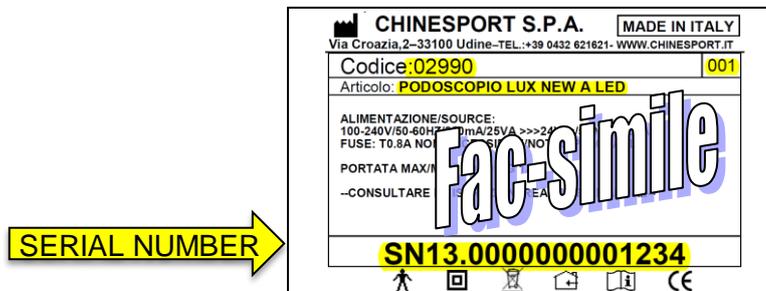NOTE:

The equipment in need of repair at Chinesport premises must carry the “AUTHORIZATION NUMBER” on the package. This identification number, to be required from the Technical Assistance Service, helps reduce waiting times and improves service quality.

Alternatively, you can contact the technical support by email (assistenza@chinesport.it), for users without internet connection it's possible to send a fax (0432 621 698) or phone (0432 621 699), but the timing of response could be longer.

7 CLEANING AND DISINFECTION

Electrical equipment must be disconnected from mains power before cleaning.

Sky is the flame retardant fabric made for residential and contract upholstery. Its resistant structure and the easy cleaning make sky particularly suitable for hospitals, nursing homes, spas, rehabilitation centers and gyms. Sky has a leather-effect surface with a brilliant finishing, in a wide range of colors all available on stock. Sky contains Vinyzene, an anti-microbial agent added in the first steps of production process. This creates a defensive barrier against mites, bacteria and fungus, avoiding smells and allergies.

7.1 Washing

Dust gently and in case of light stains pass the surface with a damp cloth or a neutral detergent, rinse with water without squeezing. Let the fabric dry avoiding direct exposure to the sunlight. The following instructions are recommended for small located and more obstinate stains (oil, fat, cosmetics, ink, coffee, liquor, chewing-gum etc.): dissolve immediately with clear ethylic alcohol diluted in water to 20%, dab and then treat with a neutral detergent solution. Rinse abundantly.

7.2 Disinfection

7.2.1 Surface disinfection

For the disinfection of surfaces such as chassis, sky, hand control, etc. we recommend the use of disinfectants such as AMUCHINA® 10% or equivalent disinfectant with sodium hypochlorite 0.1% at the following spectrum of action:

- bactericidal in the presence of interfering substances according to EN 1276:1997;
- fungicide in the presence of interfering substances according to EN 1650:1997;
- bactericidal test according EN 13697:2001;
- fungicide test according EN 13697:2001;
- active on Salmonella typhimurium according EN 13697:2001;
- active on HIV, HAV - HBV, HCV;
- virucidal according EN14476:2005;
- virucidal influenza H1N1 according EN14476:2005.

Before use is recommended to read specific flyer.

WARNING

- REGULARLY DISINFECT THE SYNTHETIC LEATHER AND USE DISPOSABLE PAPER ROLL TO KEEP THE COUCH CLEAN
- AGGRESSIVE PRODUCTS CAN DAMAGE SURFACES
- BEFORE USING THE EQUIPMENT CHECK THAT ALL COMPONENTS ARE COMPLETELY DRY
- TO AVOID PERMANENT STAINS, THE LIGHTER COLORS OF THE ARTICLE SHOULD NOT BE BROUGHT INTO CONTACT WITH TEXTILES AND CLOTHING COLORED WITH UNFIXED DYES. WE CANNOT GUARANTEE THE ABILITY TO REMOVE UNKNOWN DYES FROM LIGHT COLORED PRODUCT

8 REUSE OF THE PRODUCT

These products have been designed so as to be suitable for reuse by another user after execution of some works and the overcoming of controls for its release.

To prepare the products at re-use by another user must be done the following:


- cleaning and disinfection carefully with adequate means;
- maintenance according to the maintenance plan and control;
- granting permission for reuse, which must accompany the product.

The above transactions must be carried out by the manufacturer or authorized dealer.



9 TECHNICAL SHEET

CODE	LE1xx	LE2xx	LE3xx
Safe working load (Kg)	200		
Height adjustment (cm)	60 - 110		65 - 115
Min/max height adjustment time (sec) (electric models)	23		
Work surface dimensions (cm)	190x63 (LE111 – LE141 – LE151) 197x68 (LE121 – LE131)	190x63	
Electrical safety (electric models)	Class 2 - Part applied: typo B		
Use (electric models)	Temporary: 10% - min. USE 2 / min. PAUSE 18		
Power Supply (electric models)	220V 50/60Hz / 24V 70VA		
Fuses (electric models)	Internal parts are not accessible		
IP protection (electric models)	IP54		
Equipment weight (Kg)	82		88
Gas spring assisted sections (N° of)	2 (LE111 – LE121 – LE131 – LE151) 3 (LE141)	2	2 (LE311– LE351) 3 (LE341)
Electrically adjustable seat/backrest	No		Sì
GMDN code	34870 - a mains electricity (ac-powered) bed designed to be used as a general-purpose patient[...]		
Upholstery fabric	Class 1IM		
Upholstery padding	Foam, 40mm thick / 30kg/m ³ density		

10 ELECTRIC DIAGRAM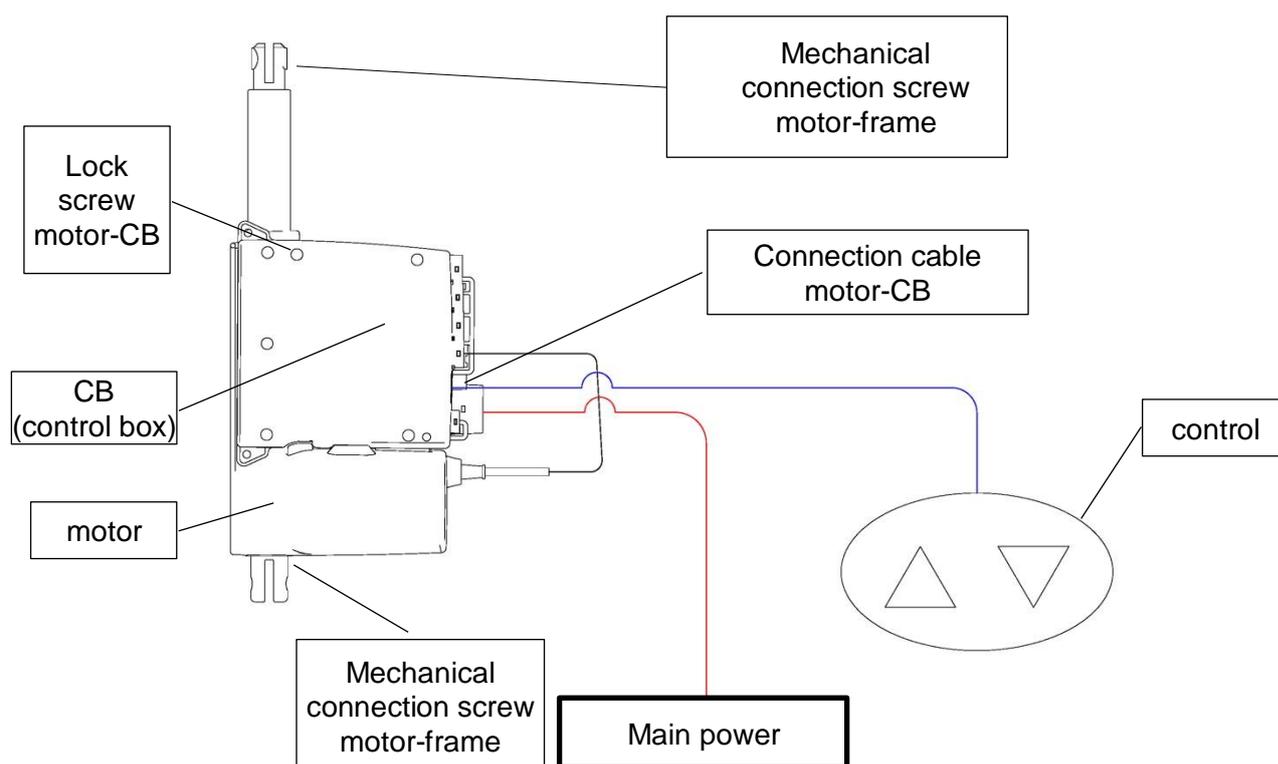

11 RELATED ITEMS AND ACCESSORIES
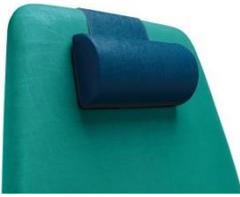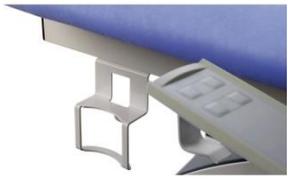
<p>AC0017 - AUXILIARY BATTERY</p> 	<p>AC0023 - CERVICAL PILLOW</p> 	<p>AC0026 - LOCKING DEVICE</p> 
<p>Battery to ensure the use in case of power failure</p>	<p>Cervical pillow for a more comfortable posture</p>	<p>Locking device for the use of the IV pole, the armrest for the chair, and the armrest for sampling</p>
<p>AC0027 - ASTA PORTAFLEBO</p> 	<p>AC0028 - ARM CHAIR</p> 	<p>AC0029 - ARM LEVY</p> 
<p>IV pole in chromed steel</p>	<p>Armrest for chair, polyurethane</p>	<p>Armrest for withdrawal, polyurethane</p>
<p>AC0031 - SHEET HOLDER</p> 	<p>AC0033 - VERTICAL SHEET HOLDER</p> 	<p>AC0034 - SHEET 2 VEILS</p> 
<p>Port lenzuolino adaptable to different roll widths</p>	<p>Stand to large rolls</p>	<p>Bedding 2-ply, pack of 6</p>
<p>AC0035 - BIG SHEET</p> 	<p>AC0036 - PROTECTIVE COVER</p> 	<p>AC0037 - ADDITIONAL FOOT SWITCH 1 MOTOR</p> 
<p>Big sheet, roll m 0,90 x m 100</p>	<p>Protective cover in synthetic fabric with elasticized edges</p>	<p>Command for height adjustment from both sides</p>
<p>AC0038 - ADDITIONAL FOOT SWITCH 2 MOTORS</p> 	<p>AC0040 - EXTRA HAND CONTROL 2 MOTORS</p> 	<p>AC0041 - HAND CONTROL SUPPORT</p> 
<p>Command for the height adjustment and the second movement from both sides</p>	<p>Command for the height adjustment and the second movement from both sides</p>	<p>Extra support for hand control model</p>

12 WARRANTY

GENERAL AFTER-SALES SERVICE CONDITIONS FOR CHINESPORT ITEMS

The warranty shall be valid for a period of 36 months from the date of delivery.

The warranty does not cover consumable items, rechargeable batteries, and generally all the material subject to wear, breakdowns caused by knocks, falls, misuse or improper use, accidental events and transport damage. This warranty shall be automatically invalidated if the equipment is tampered with. Repair of the equipment will be carried out in relation to the breakdown stated.

An estimate must be specifically requested.

Payment is cash on delivery, unless other agreements are made.

Any dispute will be dealt with solely by the Court of UDINE.

WARRANTY REPAIRS

Repairs under warranty must be specifically requested.

Repairs under warranty shall be carried out at our workshop and upon authorization, complete with the return authorization number.

The return shipment for products sent in their original package will be made free of carriage charges.

A fiscal document (bill of sale, purchase invoice or cash receipt) is required to make a warranty claim; the purchase date indicated must fall within the warranty time period.

If the product is found to be in good working order without defect, the fault-finding service shall be charged to the customer.

OUT-OF-WARRANTY REPAIRS

For out-of-warranty repairs carried out at our workshops upon authorization, transportation charges shall be paid by the customer.

The warranty of the parts replaced during the repair operations is 12 months from the moment the equipment is collected.

If the product is found to be in good working order without defect, the fault-finding service shall be charged to the customer.

REPAIRS MADE AT HOME

In the event repairs are made at the customer's home, a written request is required indicating the complete details of the person making the request, the type of machine and the breakdown.

The cost per kilometer for on-site technician visits will be agreed upon in relation to the urgency of the customer.

In the event the machine is under warranty, the customer will have to pay only the expenses incurred for the technician to reach the customer's house.

Time is calculated from the moment the technician leaves our workshop until he/she returns. The time of his/her return will be estimated based on the time required to get there.

SALE OF SPARE PARTS AND CONSUMABLE ITEMS

The orders must be in writing and specify the customer, the item and the shipping method as well as the precise fiscal data of the customer.

The order processing time can vary depending on the availability of the material.

Returned spare parts will not be accepted.

Payment will be cash on delivery, unless other agreements are made.

13 DISPOSAL OF WASTE ELECTRICAL OR ELECTRONIC EQUIPMENT AND PACKAGING

The symbol found on the equipment indicates that the waste must be "collected separately".

Therefore, the user shall take the waste to the separate waste collection points arranged by local authorities, or deliver it to the sales dealer against purchase of equivalent equipment (only in Europe). The separate collection of waste and the subsequent treatment, recovery and disposal operations promote the production of equipment using recycled materials and reduce the negative effects on the environment and health that improper waste management can cause.

Fraudulent disposal of the product will result in an administrative fine being imposed to the user by Authorities according to local Law.

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