
## HEINE BETA SLIM® Batteriegriff

## HEINE BETA SLIM® Ladegriff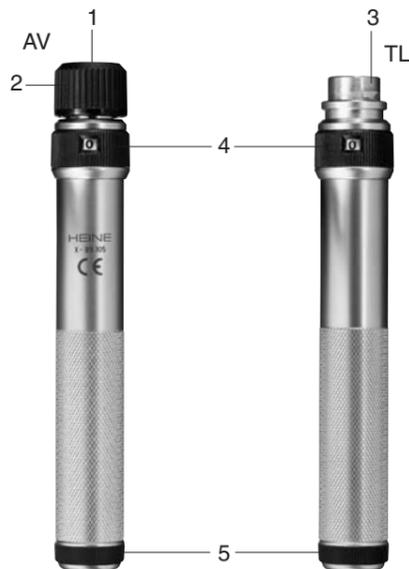

CE 93/42/EWG/CEE/EEC

### HEINE

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49(0)8152/38-0 · Fax +49(0)8152/38-202  
E-mail: info@heineopto.de · www.heine.com  
med 4100 1/1.06

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## Instructions for use

### HEINE BETA SLIM® battery handle

### HEINE BETA SLIM® rechargeable handle

Before using the HEINE handles read these instructions carefully und keep them in a safe place for future reference.

#### Safety Information

**Range of applications:** The HEINE BETA SLIM® battery and rechargeable handles are designed only for use with diagnostic instruments.

**Safety in use:** The HEINE BETA SLIM® battery handle takes two dry batteries LR6 (AA). The HEINE BETA SLIM® rechargeable handle can be fitted with a 2,5V (M2Z) or 3,5V (M3Z) rechargeable battery, which can be recharged in a HEINE NT 200 charger.

#### The automatic connector AV

**Inserting the instrument:** Hold the handle in one hand and use the other to insert the 3 male prongs of the connector into the guides of the handle head (1). Press the instrument into the handle until it clicks into place.

**Removing the instrument:** Hold the handle in one hand and rotate the knurled upper ring (2) anti-clockwise. With the other hand remove the instrument by pulling gently.

#### The twist-lock connector (TL)

**To connect an instrument head,** hold the handle in one hand, match up its mounting lugs with the recesses in the head, press in and turn the handle clockwise until securely attached.

**Removing the instrument:** To remove the head rotate it counter-clockwise and pull it off.

#### Brightness Control

To switch on, rotate the ring (4) to the left (clockwise) seen from above. In position "1" the light is brightest. Between 0 and 1 you can select any degree of brightness.

#### Changing the battery

Unscrew the bottom cap (5) and remove the used batteries or rechargeable battery.

#### Cleaning and Disinfection

Do not boil or autoclave the handle. To clean or disinfect the handle, wipe clean with soap solution or a mild disinfectant. Do not use surface disinfectants! Gas sterilization is permissible after the batteries have been removed.

#### Spare parts

M2Z rechargeable battery (2,5 V, NiCd)	<b>D-01.89.013</b>
M3Z rechargeable battery (3,5 V, NiMH)	<b>X-02.99.106</b>

### Warranty

We guarantee this device (except bulbs) for five years from date of purchase. We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product, the buyer must prove that any such defect was present at the time the product was delivered. This guarantee excludes any defects which occur because of wear and tear, incorrect use, the use of non-original HEINE parts, (especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance). Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

Bought from

date