## GARANZIA

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## WARRANTY

This forearm crutche has **2-YEAR** warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In the case you need to get your forearm crutche repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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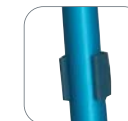
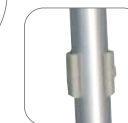
## FOREARM CRUTCHES INSTRUCTION MANUAL

Moretti S.p.A.  
Via Bruxelles 3 - Meleto  
52022 Cavriglia (Arezzo)  
Telefono +39 055 96 21 11

Fax +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)

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• Rosso  
RP701R

• Nero  
RP701N

I class medical device according  
to EEC directive 93/42

## Code RP701 (Available colours: silver, blue, red, green, black)

Thank you for purchasing forearm crutch, **MOPEDIA** by Moretti manufactured to support people with movement difficulty. It consists of lightweight anodised aluminium tubing, anatomic non-slip handle to give you comfort and security, and non-slip soft rubber tips. It is height adjustable every 2.5 cm:

- RP701 (all models) ..... from 74 to 97 cm

### To adjust height:

To adjust the height press the push button and slide the cane to the desired height. Be sure the push button is fully protruded through the desired hole when finished. Maximum endurable weight 130kg.

### ATTENTION: USE EXTRA CAUTION ON WET AND SLIPPERY SURFACES.

Avoid to leave the forearm crutch in contact with water for long periods and control its sturdiness periodically. Keep it away from children to avoid any possible risks. Replace rubber tips every time they are worn out.

• Blue  
RP701B

• Green  
RP701V

• Silver  
RP701A

• Red  
RP701R

• Black  
RP701N