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EN
GENERAL INFORMATION

**FASTERJET** is a Medical Device (Risk Class IIa - MDD 93/42/EEC) intended to be used for inhaling medication for respiratory disorders. It is highly efficient in medical therapies administered by means of aerosol nebulization.

**FASTERJET** is equipped with a double-valve system(on top of the nebulizer and mouthpiece), regulating the flow and to dramatically reduce the waste of medication during therapy.

IMPORTANT WARNINGS

• A single-patient use is recommended
• The device should be used only with compressor nebulizers for aerosoltherapy that comply with standards in force and whose use is indicated in the instruction manual
• Device not suitable for anaesthesia and lung ventilation
• The device should be used only with original accessories shown in this manual
• Information concerning connection and use with devices for aerosol-therapy are shown in the instruction manual of the devices. Always observe instructions and safety warnings as indicated in the instruction manuals of the devices for aerosoltherapy
• Use the device and its accessories as per your doctor’s recommendations
• Use medications prescribed by your doctor according to his indications
• Do not use the device for other use than Nebulizer for Aerosoltherapy. Manufacturer will not be held responsible for inappropriate use of the device
• The device does not come in sterile pack; always carry out cleaning and sterilization operations before and after each treatment. When using the device follow ordinary hygiene and cleaning precautions, (wash your hands and carry out cleaning operations as described in the specific section of this manual)
• The device has small components which might be removed and easily swallowed. Use by minors and disabled people require presence of an adult with his faculties. Do not leave the device unattended in places easily accessible by minors and disabled people
• Always store the device in a dry and clean place far from light, heat and weather
• Dispose of device as provided by standards in force
• In presence of pathologies with microbial contamination and infection hazard a single patient use of accessories and nebulizer is recommended (always consult your doctor)
• In view of its length, the connection hose could constitute a strangulation hazard.

HOW TO PREPARE AND TO OPERATE THE DEVICE

The device **is not sterile**. Before use carry out cleaning and disinfection operations as described in the specific section. To pour the medication rotate the nebulizer top counter clockwise and remove it. Pour the quantity of medication prescribed by your doctor into the nebulizer tank. **IMPORTANT: IN CASE YOU DO SO, EMPTY THE NEBULIZER, CLEAN IT THOROUGHLY AND PROCEED WITH NEW FILLING.**

When medication has been poured, connect the nebulizer top onto the tank, rotate clockwise and insert the accessory prescribed by your doctor (mouthpiece or nosepiece or mask) into the nebulizer top. Firmly connect one end of the air tube to the nebulizer and the other end to the air outlet on the compressor nebulizer for aerosoltherapy. Start the compressor for treatment to begin.

**IMPORTANT: NEVER INHALE IN HORIZONTAL POSITION. NEVER BEND THE NEBULIZER OVER 60°.**

CLEANING INSTRUCTION

After each treatment clean thoroughly each component of the nebulizer removing medication residuals and possible impurities. Clean each component as described here below except the air tube which -in case of treatment to a new patient or in case of impurities should be replaced. Wash all components (except air tube) under tap warm water (max 60° C) for about five minutes adding if necessary a small quantity of detergent following dosage and use limitations as provided by detergent manufacturer. Rinse thoroughly making sure that all deposits are washed away and let dry. All nebulizer components can be disinfected (except air tube) with chemical disinfectants following dosage and use limitations as provided by disinfectant manufacturer. They are usually available at pharmacy shops (Milton®, Amuchina® etc.). All nebulizer components can be heat steam sterilized (except air tube and masks) to 121° C (20 min.) or 134° C (7 min.) (EN554/ISO11134). The Sterilisation packaging must be conform to EN868/ISO11607 and be suitable for steam sterilisation. After sterilization always let all nebulizer components cool down to ambient temperature before use. Do not repeat sterilization cycle when nebulizer components are still warm.

<span></span> It is compulsory to carefully read the instructions before using this device.
--

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