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## **COMPOSITION**

Neoprene - Acrylic Velcro fasteners  
The device materials are latex free.

## **INDICATIONS AND INTENDED USE**

Support, protection and immobility of the limb or of the area to which it is applied. In case of long-term care / use, it is recommended to remove or loosen the device for short periods in order to aerate the affected part.

## **INSTRUCTIONS**

Apply the device and secure it with Velcro, ensure that the application has been correctly performed (the end with the Velcro of a different colour should be up), the metal core could be folded or removed depending on the requirement. The device should not be used in direct contact with broken skin.

### **Warnings**

Use extreme caution when applying.

Application by medical specialists or according to medical personnel's indications is recommended.

Before applying, inform the doctor, who prescribes the device, about possible disorders or health problems.

If you should experience any allergic reaction, that may be expected to result from the use of this product, consult a doctor.

Not sterile device.

## **CARE AND CLEANING**

Store in a cool, dry and clean place, away from direct sunlight and heat sources.

Reusable device, washing not above 30°.

Lot number and manufacture date are on the label.

Class I Medical Device

- REF 34680** Leg splint
- REF 34681** Arm splint
- REF 34682** Forearm splint
- REF 34683** Wrist splint
- REF 34684** Ankle-elbow splint
- REF 34685** Set with 5 splints in a bag

---

### **GIMA WARRANTY CONDITIONS**

Congratulations for purchasing a GIMA product. This product meets high qualitative standards both as regards the material and the production.

The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons.

Labor costs and personnel traveling expenses and packaging

not included. All components subject to wear are not included

in the warranty. The repair or replacement performed during the warranty

period shall not extend the warranty. The warranty is void in the following

cases: repairs performed by unauthorized personnel or with non-original

spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic

devices or software due to outside agents such as: voltage changes,

electro-magnetic fields, radio interferences, etc. The warranty is void

if the above regulations are not observed and if the serial code

(if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product

was purchased from. Products sent to GIMA will be rejected.

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### **DECLARATION OF CONFORMITY:**

*according to European Directive 93/42/EEC concerning medical devices*

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*We declare under our own responsibility that the following products:*

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*Meet the essential provisions of the **European Directive 93/42/EEC** (transposed into Italian legislation with Order in Council no. 46/97 and following amendments), as amended by the **European Directive 2007/47/EC** (transposed into Italian legislation with Order in Council no. 35/10)*

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Handwritten signature of Giulio Manzoni in black ink.

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Handwritten signature of Nicola Manzoni in black ink.

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GIMA S.p.A.