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0 1 2 3

Instructions for use

Hydrocoll is a self-adhesive absorbent hydrocolloid wound dressing with a gas-permeable polyurethane cover which is impermeable to liquids and bacteria.

Properties and mode of action

As the hydrocolloid takes up the wound exudate, a gel is formed, ensuring a moist wound environment. The gel layer prevents Hydrocoll from sticking to the wound. The dressing change does not irritate granulation tissue or epithelial tissue and causes minimal discomfort for the patient.

Hydrocoll is self-adhesive. Due to the thin flexible material Hydrocoll moulds easily to body contours. The surface of Hydrocoll is dirt-repellent and water-repellent, facilitating daily personal hygiene.

Indications

Hydrocoll is suitable for the treatment of wounds with light to moderate exudate.

Mode of application

Select the correct size and shape of Hydrocoll. The dressing should extend at least a centimetre beyond the edge of the wound on all sides to ensure that it stays firmly in place.

Hydrocoll/Hydrocoll thin: remove the protective paper backing, place Hydrocoll on the wound, adjust the edges to fit and press on.

Hydrocoll sacral: remove the protective paper backing and press the dressing into the anal crease. The tip of the dressing can point up- or downwards depending on the position and depth of the ulcer. Press the edges on firmly.

Hydrocoll concave for the heel and elbow: remove the largest part of the protective paper backing, place the middle part of Hydrocoll dressing over the wound, remove the rest of the protective paper backing, then overlap the wings to fit.

Due to the absorption of exudate a whitish gel is formed that is visible on the top as discoloration and a blister. Hydrocoll should be changed if the blister has almost reached the edge of the dressing. Should exudate leak from the dressing, it must however be changed immediately. To change the dressing, lift Hydrocoll at the edges and remove the whole dressing carefully. A layer of hydrocolloid gel remains on the wound which should not be confused with pus. It will not impair the healing of the wound and can be rinsed off if necessary.

In uncomplicated wounds, Hydrocoll can be left in place for intervals of up to 7 days until the wound has completely healed.

Contraindications

Do not use Hydrocoll on sores extending into muscle, tendon or bone, on clinically infected wounds or third-degree burns.

Special precautions

Hydrocoll can only be effective when a holistic assessment of the patient and the wound has taken place. Treatment with Hydrocoll cannot replace treatment of the cause of the impaired healing.

General instructions

Store below 30 °C. Contents sterile unless peel packs are damaged. Do not resterilise. Keep out of reach of children.

AU – PAUL HARTMANN Pty. Ltd. • Homebush NSW 2140
GB – PAUL HARTMANN Ltd. • Heywood/Lancashire OL10 2TT
ZA – HARTMANN-Vitamed (Pty) Ltd. • Sandton 2128

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