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#### REFERENCES

- Rubinsky, Boris. *CryoSurgery, Annual Review Biomedical Engineering*. 02:157-187. 2000
- Andrews, Mark. *Cryosurgery for Common Skin Conditions*. *American Family Physician*. 69:10. 2365-2372. 2004
- Gage, Andrew. *What Temperature is Lethal for Cells?* *J Dermatol Surg Oncol*. 5-6. 1979.
- Zitelli, John. *Wound Healing for the Clinician*. *Adv Dermatol*. 2:243-266. 1987.
- Kulrik EG. Gage AA. *Cryosurgical Treatment for Skin Cancer*. *IGAKU-SHOIN Medical Pub*. 113-141. 1990.
- Prendergast PM, Shiffman M(eds). *Aesthetic Medicine. Aesthetic Cryotherapy*. Springer-Verlag, 151-155. 2011.
- Fitzpatrick, T.B. *The validity and practicality of sun reactive skin types I through VI*. *Arch Dermatol*. 124:869-871. 1988.
- Golsen BJ. *Wound Healing for the Dermatologic Surgeon-Review*. *J Dermatol Surg Oncol*. 14:9. 959-972. 1988
- Dawber R, Colver G, Jackson A. *Cutaneous Cryosurgery, Principles and Clinical Practice*. 2cd Ed. 1997
- Whitehouse, H. *Liquid Air in Dermatology: Its Indications and Limitations*. *JAMA*. 49. 371-377. 1907
- Brodthagen H. *Local Freezing of the Skin by Carbon Dioxide Snow*. *Munksgaard-Copenhagen Publisher*. 1961.
- Carpenter, CC. *Cryotherapy for Common Skin Diseases*. *Journal of the Medical Society of NJ*. 354-357. Sept. 1943.
- Morgan, A.J. *Cryotherapy, Medscape article 1125851, pp 1-9*, Oct 8, 2012.

TABLE OF SYMBOLS	
	INSTRUCTIONS FOR USE <p>MODE D'EMPLOI GEBRAUCHSANLEITUNG INSTRUCCIONES DE USO ISTRUZIONI PER L'USO GEBRUIKSINSTRUCTIES</p>
	CAUTION/ATTENTION PRUDENCE/ATTENTION VORSICHT/ACHTUNG PRECAUCIÓN/ATENCIÓN ATTENZIONE/AVVERTENZA VOORZICHTIG/OPGELET
	MANUFACTURER FABRICANT HERSTELLER FABRICANTE PRODUKTÖRE FABRIKANT
	THIS SYMBOL IDENTIFIES THE IMPORTER CE SYMBOLE PERMET D'IDENTIFIER L'IMPORTATEUR DIESES SYMBOL KENNZEICHNET DEN IMPORTEUR ESTE SIMBOLO IDENTIFICA AL IMPORTADOR QUESTO SIMBOLO IDENTIFICA L'IMPORTATORE DIT SYMBOOL IDENTICEERT DE IMPORTEUR
	PROTECT FROM SUNLIGHT TENIR À L'ABRI DE LA LUMIÈRE SOLAIRE VOR SONNENLICHT SCHÜTZEN PROTEGER DE LA LUZ SOLAR PROTEGGERE DALIA LUCE DIRETTA DEL SOLE BESCHERMEN TEGEN ZONLICHT
	DO NOT EXPOSE TO TEMPERATURES EXCEEDING 50°C NE PAS EXPOSER A DES TEMPÉRATURES SUPÉRIEURES À 50°C NIGHT TEMPERATUREN ÜBER 50°C AUSSETZEN NO EXPONER A TEMPERATURAS SUPERIORES A 50°C NON ESPORRE A TEMPERATURE SUPERIORI AI 50°C NIET BLOOTSTELLEN AAN TEMPERATUREN BOVEN 50°C
	MEDICAL DEVICE DISPOSITIF MÉDICAL MEDIZINISCHES GERÄT DISPOSITIVO MEDICO DISPOSITIVO MEDICO MEDISCH APPARAAT
	AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY DÉPÔSITAIRE AGRÉÉ DANS L'UNION EUROPÉENNE AUTORISIERTER VERTRETER IN DER EUROPÄISCHEN GEMEINSCHAFT REPRESENTANTE AUTORIZADO EN LA COMUNIDAD EUROPEA RAPPRESENTANTE AUTORIZZATO NELLA COMUNITÀ EUROPEA VERTEGENWOORDIGER IN EUROPA
	AUTHORIZED SWISS REPRESENTATIVE REPRÉSENTANT SUISSE AUTORISÉ AUTORISIERTER SCHWEIZER VERTRETER REPRESENTANTE SUIZO AUTORIZADO RAPPRESENTANTE SVIZZERO AUTORIZZATO GEAUTORISEERDE ZWITSERSE VERTEGENWOORDIGER

	<b>Manufacturer:</b> CryoConcepts LP 1100 Conroy Place Easton, PA 18040, USA www.cryoconcepts.com		<b>Emerge Europe</b> Westervortsedijk 60 6827 AT Arnhem The Netherlands
	<b>MedEnvoy Global BV</b> Prinses Margrietplantsoen 33 Suite 123 2595 AM The Hague The Netherlands		<b>MedEnvoy Switzerland</b> Gotthardstrasse 28 6302 Zug, Switzerland
	<b>Australian Sponsor</b> Emerge Australia Level 20 Tower II Darling Park 201 Sussex Street, Sydney, NSW 2000 Australia		
			

CryoClear® is a trademark of CryoConcepts LP.
U.S. Patent # 8,647,337 - International Patent # W02011005495