



www.wunder.it

BABY PORTABLE HEIGHT ROD

MOD. WH80

USER MANUAL



Read this manual carefully before using the instrument

INDEX

1. TECHNICAL SPECIFICATION AND INSTALLATION.....	2
2. INSTALLATION OF MECHANICAL ROD	2
3. HOW TO MESURE A BABY.....	3



WARNINGS!

Operators must read this manual carefully, comply with the instructions it contains and become familiar with the correct use and maintenance procedures of the instrument.

- Retain this manual for consultation and as a help in staff training
- Do not apply loads abruptly.
- Do not try to open the instrument.
- Do not remove seals from the instrument.
- Do not short-circuit the battery terminals
- Regularly check the integrity of the instrument's power cord and make sure it does not come in contact with hot appliances
- Do not place the instrument in water or other liquids

1. TECHNICAL SPECIFICATION AND INSTALLATION

TECHNICAL SPECIFICATION


MEASUREMENT RANGE	35~80cm
DIVISION	1mm
SIZE	(w)87 x (L)30 x (h)7cm
WEIGHT	700g

2. INSTALLATION OF MECHANICAL ROD


Per l'installazione dello statimetro misura altezza digitale procedere come segue:

A) Apply the support of height rod to the scale's tray through the two screws supplied. (Figure .A).	Fig. A	
B) Insert the mechanical height rod into the support previously applied to the scale's tray. (Figure .B)	Fig. B	
C) The installation is now complete. (Figure .C)	Fig. C	

3. HOW TO MESURE A BABY

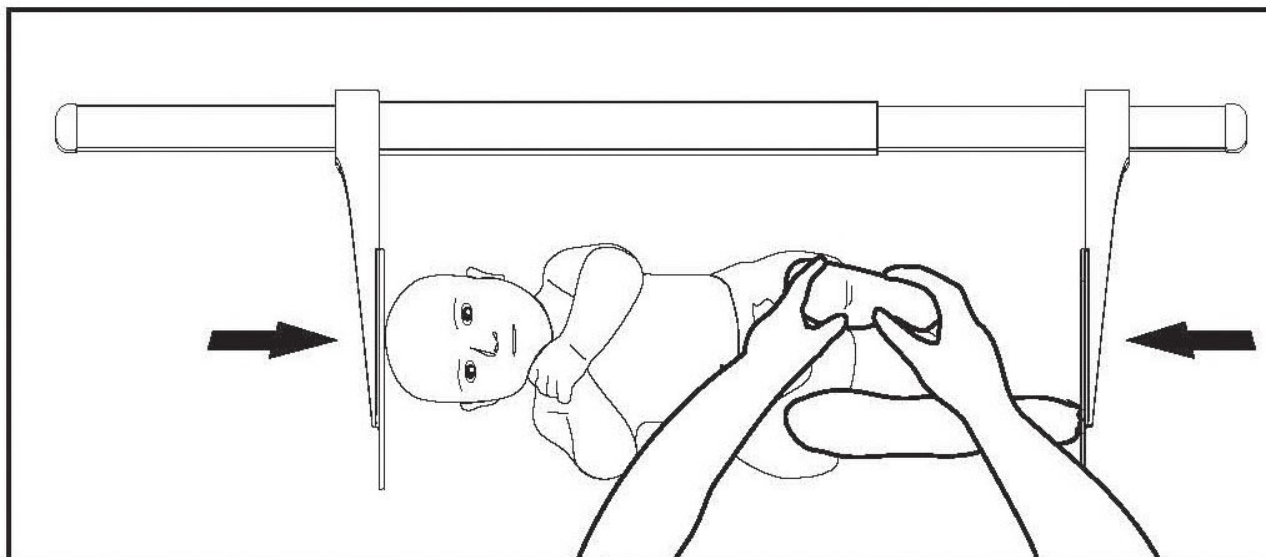

1. Lightly lean baby's head on the left shovel
2. Stretch baby's feet

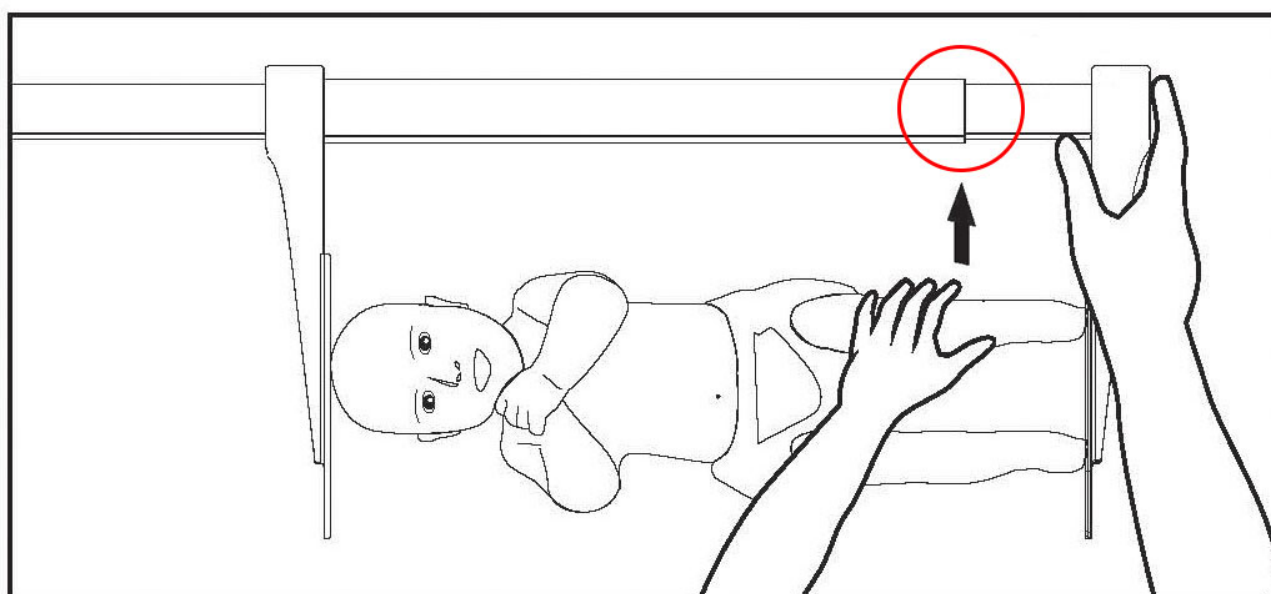

3. Move the right hand shovel until it touches baby's feet
4. The height of the child is read at the red line on the statimeter