meso-relle®

Aghi ipodermici

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INSTRUCTIONS FOR USE: The hypodermic needles meso-relle are intended for the intradermic injection of fluids.

WARNINGS: The sterility is guaranteed if the package is intact. Do not use if the package is open or damaged.

Handle carefully to avoid bites. Suitable for use by health professionals in environment adequate for medical practice.

DONT REUSE OR RSTERILIZE. Any reuse or re-sterilization of the device can lead to the transmission of serious diseases, cross-infections and microbiological contaminations.

INSTRUCTIONS FOR USE:

1. Open the package and take out the device. Do not remove the cap from the needle.
2. Steadily connect the needle to the syringe or injection device. Lock the needle firmly by pressing and turning
3. Remove the cap from the needle by pulling it horizontally.
4. Perform the injection according to the desired technique.
5. At the end of the operation, dispose the device in a special container for medical waste. Do not attempt to re-insert the cap after use.

Dispose of in accordance with the current local and national regulations on medical waste.

PRESERVATION TIME (if the packaging is intact): 5 years.

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İMALATÇI FIRMA: BIOTEKNE Srl, Via della Bastia, 9 - 40033 Casalecchio di Reno (BO), ITALIA