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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

The blades of bandage scissors are made of 2Cr13 stainless steel, the handles of PP.
The blades of these scissors have been milled in order to cut clothes, gauzes and bandages.

Bandage scissors are suitable for:

- Cutting medical gauze
- Cutting bandages away from skin

Warning

Remember that scissors may cause harm if not handled with care.
Please pay due attention and care during use.
Choose a safe place for storing the scissors, out of reach of non-authorised personnel.
Clean the scissors after use and eliminate any residue.
Keep the scissors closed when not in use. Any equipment showing signs of wear should be discarded immediately.

Instructions for use

Firmly grip the scissors, cut starting from the front of the body. Do not cut laterally, pay attention in order not to damage the patient's skin.

Sterilisation: autoclave at 138°C

	Keep in a cool, dry place
	Keep away from sunlight
	Consult instructions for use
	Product code
	Lot number
	Non-sterile
	Manufacturer
	Medical Device compliant with Regulation (EU) 2017/745
	Caution: read instructions (warnings) carefully
	Medical Device

	Date of manufacture
	Authorized representative in the European community
	Imported by
	Unique device identifier

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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EC REP **SUNGO Cert GmbH**
**Gima S.p.A.**