
Particle Filtering Mask - Earloop - black
Masque filtrant contre les particules - Avec élastique - noir
Mascherina filtrante contro particelle - Con elastico - nera
Mascarilla filtrante contra partículas - Con elástico - negra
Máscara de filtragem de partículas - Com elástico - preta

1 **Piece**
Pièce
Pezzo
Pieza
Peça

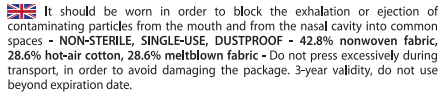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EU: 2016/425 - EN 149:2001 + A1:2009

 Wearing instructions

1. Place the nose piece on your nose and the mask on your chin.
2. Pull the elastic loops behind your ears and adjust the mask.
3. Adjust the nose piece to your nose.
4. Make sure that the mask fits close to your face.

⚠ WARNING

- a. Risk may be expected if the product is used in a surgical setting or in a setting with significant exposure to bodily or other hazardous liquids or fluids; or used in a clinical setting where the risk of infection through inhalation is high; or used in the presence of a high intensity heat source or flammable gas.
- b. The product is not designed for use in an anoxic or an underwater environment.
- c. Check for integrity before use.
- d. This product is not a respirator.
- e. The mask does not eliminate the risk of contracting any disease or infection.
- f. Failure to properly use and maintain this product could result in illness.
- g. Do not use beyond expiration date.
- h. Do not use if the package is damaged.
- i. Do not use with oxygen and use only in ventilated areas.

with oxygen >19% and **only** for a single work shift. The device is **less** effective if it is **altered** or **damaged** or if hair and beard are present. **If you feel sick, leave the working area and move to an aerated area.** Do not use in **highly contaminated or potentially explosive areas.**

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Gima S.p.A. Via Marconi, 1 - 20060 Gessate (MI) Italy