PROFESSIONAL MEDICAL PRODUCTS

TELO PORTAFERITI

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CARRYING SHEET

8 lifting handles version

CIVIÈRE DE TRANSPORT

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CAMILLA DE TRANSPORTE

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MACA DE LONA PARA TRANSPORTAR FERIDOS

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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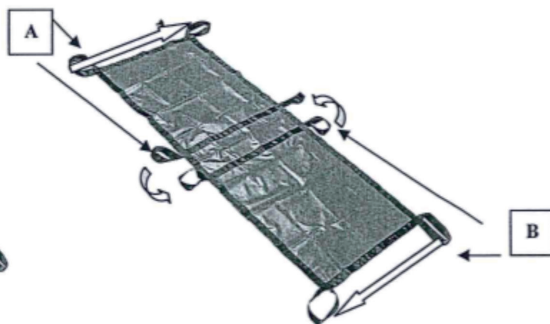
Fig. 2

ENGLISH

INDICATIONS: the flexible carrying sheet moves patient securely in restricted areas where standard stretchers have difficulties to be used. **It should not be used if there is a spine, backbone or pelvis suspected injury.**

SPECIFICATIONS: strong polyester fabric, PVC coated, with four secured handles per side to enable a comfortable patient carriage. The materials are waterproof and are easily washable and disinfected, bacterial and fungus proof, non putrescent. Transparent to X rays.

INSTRUCTIONS: should be used by skilled staff. After laying the patient on the carrying sheet, four rescuers (as per fig. 1) should pull up contemporary on the lifting handles and bring the patient to destination. However if four are not available, two rescuers (as per fig. 2).

CLEANING INSTRUCTIONS: after using the carrying sheet, it should be accurately washed with water and soap. It can also be disinfected by a bacterial liquid solution.

STORAGE: no special instructions are requested. Better if kept in a dry place, out of dust and direct sunlight.

MAXIMUM LOAD: 150 kg.

Made according EN 1865-1:2010+A1:2015

REF	IT - Codice prodotto GB - Product code FR - Code produit ES - Código producto PT - Código produto DE - Erzeugniscode	LOT	IT - Numero di lotto GB - Lot number FR - Numéro de lot ES - Número de lote PT - Número de lote DE - Chargennummer
	IT - Conservare al riparo dalla luce solare GB - Keep away from sunlight FR - À conserver à l'abri de la lumière du soleil ES - Conservar al amparo de la luz solar PT - Guardar ao abrigo da luz solar DE - Vor Sonneneinstrahlung geschützt lagern		IT - Conservare in luogo fresco ed asciutto GB - Keep in a cool, dry place FR - À conserver dans un endroit frais et sec ES - Conservar en un lugar fresco y seco PT - Armazenar em local fresco e seco DE - An einem kühlen und trockenen Ort lagern
CE	IT - Dispositivo medico conforme al regolamento (UE) 2017/745 GB - Medical Device compliant with Regulation (EU) 2017/745 FR - Dispositif médical conforme au règlement (UE) 2017/745 ES - Producto sanitario conforme con el reglamento (UE) 2017/745 PT - Dispositivo médico em conformidade com a regulamento (UE) 2017/745 DE - Medizinprodukt im Sinne der Verordnung (EU) 2017/745		IT - Leggere le istruzioni per l' uso GB - Consult instructions for use FR - Consulter les instructions d'utilisation ES - Consultar las instrucciones de uso PT - Consulte as instruções de uso DE - Gebrauchsanweisung beachten
	IT - Fabbricante GB - Manufacturer FR - Fabricant ES - Fabricante PT - Fabricante DE - Hersteller	MD	IT - Dispositivo medico GB - Medical Device FR - Dispositif médical ES - Producto sanitario PT - Dispositivo médico DE - Medizinprodukt
	IT - Data di fabbricazione GB - Date of manufacture FR - Date de fabrication ES - Fecha de fabricación PT - Data de fabrico DE - Herstellungsdatum		IT - Non sterile GB - Non-sterile FR - Pas stérile ES - No estéril PT - Não estéril DE - Nicht steril
UDI	IT - Identificatore univoco del dispositivo GB - Unique device identifier FR - Identifiant unique de l'appareil ES - Identificador de dispositivo único PT - Identificador exclusivo do dispositivo DE - Eindeutige Kennung des Geräts		IT - Attenzione: Leggere e seguire attentamente le istruzioni (avvertenze) per l'uso GB - Caution: read instructions (warnings) carefully FR - Attention: lisez attentivement les instructions (avertissements) ES - Precaución: lea las instrucciones (advertencias) cuidadosamente PT - Cuidado: leia as instruções (avisos) cuidadosamente DE - Achtung: Anweisungen (Warnungen) sorgfältig lesen
	IT - Non utilizzare se l'imballaggio è danneggiato GB - Don't use if package is damaged FR - Ne pas utiliser si le colis est endommagé ES - No usar si el paquete está dañado PT - Não use se o pacote estiver danificado DE - Nicht verwenden, wenn das Paket beschädigt ist		

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Gima S.p.A.