# HEINE BETA 200® VET

## HEINE

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. + 49(0)81 52/38-0 · Fax + 49(0)81 52/38-202  
E-mail: info@heine.com · [www.heine.com](http://www.heine.com)  
med 9187 1 / 1.08

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### Warranty

We guarantee this device (except bulbs) for five years from date of purchase. We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product, the buyer must prove that any such defect was present at the time the product was delivered. This guarantee excludes any defects which occur because of wear and tear, incorrect use, the use of non-original HEINE parts, (especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance). Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded. For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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# Instructions for use

## HEINE BETA 200® VET

Read these instructions carefully before using the HEINE BETA 200® VET and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

A guarantee for the safety, reliability and performance of this unit can only be made provided it is used in accordance with the instructions below.

### Specula

The specula (1) are mounted on the instrument head (2) so that the internal lug locates in the slit (3) of the metal body of the instrument, and are then firmly fixed by a twist to the right.

### Magnification optics

The BETA 200 VET otoscope is fitted with a swivel viewing window (4). Pneumatic insufflation testing can be done with the window in position. The viewing window swivels to left or right to facilitate instrumentation. The viewing window gives approx. 2.5 x magnification.

### Changing the bulb

**Please note: The performance of this instrument can only be guaranteed if genuine original HEINE bulbs are used.**

- Remove the instrument head from its handle and pull the bulb (5) out of the head by gripping the narrow rim at its lower end.
- Before inserting the new bulb, ensure that the voltage rating printed on it corresponds to that of the handle or transformer power supply.
- Re-insert the bulb fully into the body of the instrument.

### Sterilisation and Cleaning

**Otoscope head:** The outside can be cleaned with a soft cloth, the inside with a cotton-wool bud moistened with alcohol. Gas sterilisation is permitted. The instrument should not be soaked.

**Reusable SANALON S specula** can be cleaned with all normal solutions and disinfectants and can also be boiled. They can be autoclaved at up to 134 °C as often as required

### Accessories and Replacement Parts

|                                            |                    |
|--------------------------------------------|--------------------|
| Insufflator                                | <b>B-00.11.240</b> |
| HEINE XHL® Xenon Halogen bulb 2,5 V        | <b>X-01.88.077</b> |
| HEINE XHL® Xenon Halogen bulb 3,5 V        | <b>X-02.88.078</b> |
| Specula in SANALON S, 61 x 4 mm            | <b>G-00.21.210</b> |
| same 61 x 6 mm                             | <b>G-00.21.211</b> |
| same 61 x 9 mm                             | <b>G-00.21.212</b> |
| 1 complete set = 3 pcs. veterinary specula | <b>G-00.21.213</b> |

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