# GIMA

**CAVI BIPOLARI**  
**BIPOLAR CABLES**  
**CABLES BIPOLAIRES**  
**ZWEIPOLIGE KABEL**  
**CABLES BIPOLARES**  
**FIOS ELÉTRICOS BIPOLARES**  
**ΔΙΠΟΛΙΚΑ ΚΑΛΩΔΙΑ**  
**أسلاك ثنائيّة القطب**

**MANUALE D'USO E MANUTENZIONE**  
**USE AND MAINTENANCE BOOK**  
**INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN**  
**BETRIEBS UND WARTUNGSANWEISUNGEN**  
**MANUAL DE USO Y MANTENIMIENTO**  
**MANUAL DE USO E MANUTENÇÃO**  
**ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**  
**دليل الإستعمال والرعاية**

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### **FEATURES**

The GIMA bipolar cables are devices used with bipolar electro-surgery devices. The cables are made of light and flexible silicone, they are 3 meters long and can be sterilized in an autoclave up to 137°. They have a universal connection that makes them easy to use with other GIMA devices and with most of the electric scalpels produced in Europe.

### **INFORMATION**

The product is designed to be used only by qualified doctors, specialized in electro-surgery techniques and the maintenance of this type of product.

*The incorrect use or poor or inappropriate maintenance can make the product deteriorate quickly.*

*These products can cause injuries if not used correctly!*

*Do not put in contact with devices that are not insulated!*

*Do not use electro-surgical devices on patients with Pacemakers or near inflammable liquids or anesthetics!*

The electric scalpels used together with these cables will destroy tissues and are extremely dangerous if used incorrectly. Carefully read the user instructions supplied by the manufacturers of the electric scalpels.

The product must be sterilized before the first use: we recommend sterilization in the autoclave up to 137°.

### UNPACKING

*Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.*

*They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).*

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

### USE

Before using the device make sure the cables are fully compatible with the electro-surgical devices.

Connect the connectors to the chosen surgical device and then the feeding device and verify that the connections are correct and that no metallic parts are left exposed.

Do not bending the cable or fix it in any way with clips or similar devices that could damage its functionality.

*Wrong connections and/or improper use could cause shocks: do not leave the cable attached to the feeding device without first positioning the surgical device.*

*Before replacing the surgical instrument disconnect the cable from the feeding device.*

### MAINTENANCE

It is very important to examine the product to identify any breaks, defects or worn parts. Pay special attention to the connectors. The cables must be kept in a dry and clean place.

If the product is defective or too worn, do not use or try to repair it.

After use the product must be cleaned thoroughly with a proper solution. Remove blood and any other impurities using a soft cloth and rinse properly.

Before sterilizing the device, run water out of the connectors and dry the cable.

*Do not use ultrasound or mechanical cleaning methods because they could damage the product or shorten its technical life.*

Be sure the product does not come into contact with acids or other disinfectants which could corrode it.

#### **Sterilization with ethylene oxide**

Roll up the cable without pressing or twisting and put it in the ethylene oxide container (ETO).

#### **Sterilization in the autoclave**

Make sure the cable does not come into contact with the metal parts of the sterilizing unit. Do not exceed the temperature of 137°C.

#### **Sterilization with hot air**

Hot air sterilization or other systems that may damage the cable are not recommended.

## GUARANTEE

Thank you for having acquired our product.

This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

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
During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, etc. No compensation will be given for loss of use of the product.

Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

*This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.*

The guarantee will be invalidated if the registration number has been removed, cancelled or altered. Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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## Tecno Instruments Pvt. Ltd.

316 - C Small Industrial Estate,  
Sialkot - 51340, Pakistan

has been assessed and certified as meeting the requirements of

### Directive 93/42/EEC

Annex II (excluding Section 2)

For the following products


**Reusable Non Sterile Bipolar Forceps complete with Electrosurgical Cables, Reusable Non Sterile Monopolar Forceps complete with Electrosurgical Cables and Reusable Non Sterile Finger Switch Pencils complete with Electrodes.**

**Sterile Single Use Bipolar Forceps complete with Electrosurgical Cables, Sterile Single Use Monopolar Forceps complete with Electrosurgical Cables and Sterile Single Use Finger Switch Pencils complete with Electrodes.**

This certificate is valid from 29 August 2009 until 29 August 2012 and remains valid subject to satisfactory surveillance audits.  
Re certification audit due before 29 August 2012  
Issue 4. Certified since 29 August 2000

Notified Body Number 0120

Authorised by

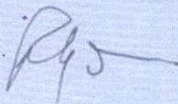CE  
0120

SGS United Kingdom Ltd Systems & Services Certification  
2028 World Parkway, Weston-super-Mare, BS22 6WA UK  
T +44 (0)1934 522917 F +44 (0)1934 522137 www.sgs.com

SGS CE 01 0308

Page 1 of 1