
# GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy  
Italia: tel. 199 400 401 - fax 199 400 403  
Export: tel. +39 02 953854209/221/225 fax +39 02 95380056  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

**Fermatesta**  
**Head immobilizer**  
**Immobilisateur de tête**  
**Kopfhaltebefestigung**  
**Inmovilizador de cabeza**  
**Imobilizador de cabeça**  
**Συσκευή σταθεροποίησης κεφαλής**  
**مثبت للرأس**

**MANUALE D'USO E MANUTENZIONE**  
**USE AND MAINTENANCE BOOK**  
**INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN**  
**BETRIEBS UND WARTUNGS ANWEISUNGEN**  
**MANUAL DE USO Y MANTENIMIENTO**  
**MANUAL DE USO E MANUTENÇÃO**  
**ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**  
**دليل الإستعمال والرعاية**

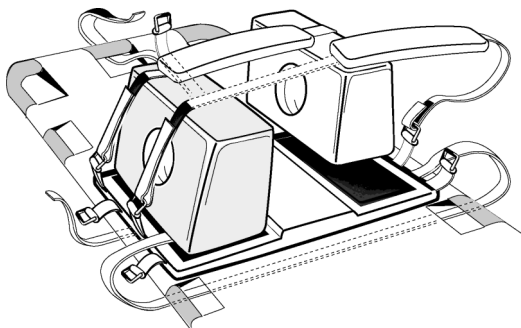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## FEATURES

The foreseen use is to block the head of the patient during transport on specific stretchers, in case of suspected injuries at the head or neck.

The product includes:

- a base that is fixed to the stretcher using side straps and a front one and sometimes there are underlying surfaces coated in Velcro;
- two side blocks, padded to hold body in place;
- two padded belts with Velcro closure to block forehead and chin.

For transportation in critical conditions or along rough routes, this piece can be used along with a rigid neck piece. The head piece is made of soft material covered with washable film that causes minimum interference with X-rays.

The product or parts of it, cannot be used in ways not specified in this manual.

## WARNINGS

*The device must be used by trained personnel and must be positioned by at least two operators. For the safety of the patient it is necessary to correctly fix the head immobilizer with the straps, verifying that they do not hinder transportation operations. Non-compliance with this instructions could cause injuries to people.*

*Do not use the product if damaged and contact our distributor. Avoid temporary repairs.*

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

## UNPACKING

*Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled. They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).*

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

## OPERATION

Always secure the patient to the stretcher so the patient does not fall due to jerks or accidental inclinations.

Fix the base to the stretcher using the specific belts and if present use the surfaces covered with Velcro.

If there are suspected injuries to the neck, before transferring the patient, apply a neck collar.

Transfer the patient to the stretcher and place the blocks on the head of the patient exercising a light pressure downwards to fix them to the Velcro-covered base.

The blocks must adhere to the sides of the head without generating excessive pressure and apply symmetrically. The inclined part of the blocks must face outwards.

Apply the padded belts on the chin and forehead according to the trauma and the morphology of

the patient: fixing them to the base using the nearest buckles or buttonholes.

*All emergency operations must be carried out by qualified personnel.*

### **MAINTENANCE**

Clean the head immobilizer without using abrasive products or solvents (we recommend using water and soap). After cleaning the immobilizer dry all parts thoroughly.

To disinfect the immobilizer, use a combination of cleaning products/disinfecting products. Dilute the disinfecting product following the manufacturer's instructions.

Periodically verify the wear of the product and if necessary replace the damaged parts with original spare parts.

### **GIMA WARRANTY CONDITIONS**

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production.

The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from.

Products sent to GIMA will be rejected.

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