# ASPEED PROFESSIONAL

GUARANTEED

3

YEARS

**HOSPITAL**

ISTRUZIONI D'USO

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INSTRUCTION MANUAL

MANUEL D'INSTRUCTIONS

MANUAL DE INSTRUCCIONES

**CARATTERISTICHE TECNICHE/TECHNISCHE MERKMALE/TECHNICAL CHARACTERISTICS/CARACTERISTIQUES TECHNIQUES/CARACTERÍSTICAS TÉCNICAS**

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ASPEED PROFESSIONAL aspirator is a professional unit for clinical use for secretion aspiration. It has a vacuum regulator (6), vacuum gauge (5) a 1000 cc. container (3) with protection device (10) to prevent the ingress of liquid into the suction pump. Lifetime lubrication, easy to handle, simple to use, reliable, strong and silent.

The ASPEED PROFESSIONAL aspirator is provided with the following 3A accessories: 1000 cc. container with protection device, power supply cord, short silicon connection tube that can be sterilised, long silicon connection tube that can be sterilised, **sterile, disposable** cannula, **sterile, disposable** manual regulator, **disposable** anti-bacterial filter, disposable bag.

**N.B.: Only use original 3A accessories.**

## **IMPORTANT WARNINGS**

**This is a medical device for use on medical prescription. It must be operated as indicated in this user instruction manual. It is important for patients to read and understand the information concerning using and maintaining the unit. If you have any questions, contact your stockist. MICROBIAL CONTAMINATION: in the case of illnesses with a risk of infection or microbial contamination, we advise each individual to use his own accessories. (Consult your doctor).**

The manufacturer has made every effort to ensure that all its products are of the highest quality and are reliable. Nevertheless, as for all electrical appliances, fundamental safety standards must be observed:

- Children and persons who are not self-sufficient may only use the unit under the strict supervision of a responsible adult who has read this manual.
  - Never use adapters for supply voltages different to the voltage shown on the data plate on the back of the unit. Keep the cord away from hot surfaces.
  - The unit should not be used in the presence of inflammable anaesthetic mixtures with air, oxygen or nitrous oxide.
  - Never handle the supply cord plug with wet hands or use the unit when taking a bath or a shower. Never leave the unit close to water, do not submerge it in any liquids. If by chance it falls into water, pull the plug out of the socket immediately before recovering it. Do not use the unit if the plug or supply cord are worn or wet (send it immediately to your stockist).
- 
- Unit casing not protected against liquid penetration.
  - Only authorised personnel may perform maintenance and/or repair work. Unauthorised repairs annul the guarantee.
  - Ensure that the connections and container closing are performed carefully to prevent suction losses.
  - Do not tip the container over while connected to the unit when operating, as liquid may be sucked inside the appliance causing pump damage. If this happens, switch the aspirator off immediately and empty and clean the container. Send the appliance to your stockist. The appliance is equipped with two safety fuses, easy to inspect in the event of a fault. They are located in the back of the appliance in the panel socket. Before performing this operation remove the power supply plug.
  - The cannula and the manual control of the aspirated flow are sterile, disposable products: They must be replaced after every application.
  - Check the use-by-date on the original packaging of the cannula and check the integrity of the sterile packaging.
  - The **disposable** antibacterial filter must be replaced after every application.
  - In view of their length, the power cord and connection hose could constitute a strangulation hazard.

## USE INSTRUCTIONS

Each time before use, ensure that all the accessories are perfectly clean according to the instructions indicated in the "CLEANING AND DISINFECTION OPERATIONS"

1. Connect the appliance as indicated in figure 1.
2. Using the vacuum regulator (6) we can set the desired vacuum value (bar). By turning the knob to "+" we obtain a higher vacuum and by turning it to "-" a lower one; these values are read on the vacuum gauge (5).
3. Start the appliance by placing the switch to the "I" (ON) position (4).
4. After the application, switch off the appliance, detach the supply cord from the current socket and perform the cleaning operations as illustrated in the "CLEANING AND DISINFECTION" paragraph.
5. **Secretion collection vessel - 1000cc. (Fig. 2)**

The 1000 cc. collection bottle supplied with the aspirator can be used in two ways: as a collection vessel which can be sterilised as shown in photograph 1A or as a collection vessel with disposable bag as illustrated in photograph 1B.

### **Sterilisable secretion collection vessel**

The antibacterial filter (6) should be inserted directly into the vessel top. Do not use the aspirator without the antibacterial filter, because from a bacteriological point of view, it becomes dangerous for the patient. The collection vessel is supplied with an overflow valve, polypropylene plug (2) and polycarbonate vessel (1). The plug (2) foresees the direct insertion of the antibacterial filter, which can only take place on the connector called "VACUUM". The antibacterial filter also performs the function of protecting the aspiration circuit from any contaminating agents sucked in during use. All the components of the vessel can be sterilised using a conventional system in an autoclave at a temperature of 121°C, or by boiling for 10 minutes. We recommend replacing the complete vessel at every 30 sterilisation cycles. Do not overturn the vessel during use, in order to prevent the intervention of the non-return valve (3-4-5); should this occur, switch the aspirator off and detach the tube connected to the antibacterial filter. Never use the aspirator without the secretion collection vessel and/or without the antibacterial filter.

Connection: connect one end of the short, sterilisable silicon tube (10) to the antibacterial filter connector (6) and insert this latter into the "VACUUM" hole of the blue top (2); connect the other end to the "INLET" connector of the aspirator. Connect one end of the long sterilisable silicon tube to the "PATIENT" connector of the blue top (2); to the other end, connect the disposable, sterile manual regulator (9) and connect the disposable, sterile cannula (8) to this latter.

**Secretion collection vessel with disposable bag.** The aspirator can be used with the re-usable 1000 cc. transparent secretion collection vessel (1) and with the disposable bag supplied. In this case, the antibacterial filter is integrated in the disposable bag and therefore the antibacterial filter (6) and blue top with valve (2) are not used. The filter integrated in the bag also performs the function of preventing the return of aspirated liquids towards the aspirator. In this case, to restore device operation, it is necessary to replace the disposable bag. For the cleaning and disinfecting operations of the tubes and vessel, sterilise the single parts in an autoclave at a maximum temperature of 121°C, or by boiling for 10 minutes. The bag is disposable and must always be replaced after every use. The bag must be completely inserted in the vessel in order to prevent any vacuum losses.

*N.B.: maximum disposable bag usage vacuum: -0.75 bar.*

Connection: connect one end of the short sterilisable silicon tube (10) to the yellow connector (VACUUM) of the top (photograph 1B) and the other end to the "INLET" connector of the aspirator. Connect one end of the long sterilisable silicon tube to the red connector (PATIENT) and connect the disposable sterile manual regulator (9) and the disposable sterile cannula (8) to the other. ***N.B.: only use Meditea 1 l. disposable bags - code M043002/A.***

## **CLEANING AND DISINFECTION OPERATIONS**

**P.S.: If using chemical disinfectants, follow the manufacturer's instructions exactly.**

- The cannula, the manual command of the aspirated flow are sterile, disposable products and must be replaced after every application.
- The antibacterial disposable filter must be replaced after every application.
- Never leave the appliance in water or submerged; clean the external casing of the appliance using only a damp cloth with detergent (non abrasive).

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**DISPOSAL PROCEDURE (Dir. 2012/19/Ue-WEEE)** The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment. At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same. **Caution!** The wrong disposal of electric and electronic equipment may involve sanctions.

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