

## ENDOTRACHEAL TUBE

### Directions:

#### CUFFED:

1. Prior to intubation, deflate the cuff completely.
2. After intubation, inflate the cuff using the minimum volume of air to provide an effective seal.
3. Immediately after cuff inflation, auscultate both lung fields. If breech sounds diminished over adjust the tube as required.
4. Endotracheal Tube placement should be confirmed by viewing the position of the tube tip with chest radiograph.

#### UNCUFFED:

1. Endotracheal Tube placement should be confirmed by viewing the position of the tube tip with chest radiograph.
2. If breath sounds diminished over one lung field or absent over one or both fields, adjust the tube as required.

### Warning:

1. Single use only.
2. Sterile if package is unopened or undamaged.
3. Do not re-sterilize.
4. Store in a dry, cool place.
5. Keep away from sunlight.



**CE 0123**



Well Lead Medical Co., Ltd.

C-4 Jinhu Industrial Estate, Hualong, 511434 Panyu, Guangzhou, P.R.C.



Shanghai International Holding Corp. GmbH (Europe)

Eiffestrasse 80, 20537 Hamburg, Germany