# GIMA

## SECURITY SYSTEM

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## FEATURES

### Product description

GIMA Security System or needle destroyer (Cod. 25820) is conceived to quickly (2-3 seconds) and easily melt needles with a diameter up to 1,2 mm. It is so powerful that it can reach a temperature of 1000°C and destroy any kind of virus and bacterium. It can also melt needles with larger diameters (up to 1,6 mm), however this operation takes a longer time (the melting system is patented).

GIMA needle destroyer is extremely solid; it weighs more than 4 Kg and has been built with high-quality materials, according to IEC 62-5 standard in force for electro-medical equipment, ensuring a very long lifetime. It is equipped with a withdrawable drawer for the collection of wastes and a practical handle for an easy transport.

### Technical features

Dimensions	22x14x119 cm
Weight	4 Kg
Power supply	220V ~ 50Hz. 400W
Melting temperature	1000 °C
Fuses	2x2,5 AT
Self-resetting thermal fuse	It is located in the transformer and switch the power supply off when a temperature of 100°C is reached inside the transformer.

### Field of application, improper use

GIMA needle destroyer is conceived to destroy hypodermic needles and it cannot be used for purposes not in compliance with its features.

## PRESCRIPTIONS

### Accident prevention

*Do not introduce your hand in the compartment inside which the melting waste collection drawer slides, according to warning sign applied to the product.*

The product shall be used by qualified personnel only.

*Do not use the equipment in case it is damaged. Apply to your retailer. Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.*

*Do not carry out maintenance or cleaning operations when the equipment is connected to the mains.*

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

## UNPACKING

### Unpacking

*Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.*

*They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).*

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

## INSTALLATION

### Installation and connections

The installation of GIMA needle destroyer is very simple. Lay the equipment on a stable base and plug in to a grounded 220 Volt ~ 50Hz socket.

Before switching on, make sure that the drawer for waste collection is inserted into the corresponding compartment.

## OPERATION

### Operating instructions

After plugging in, it is possible to switch the equipment on by means of **PUSH** button of the device.

Insert the needle into one of the two melting holes, keep the syringe in vertical position, press pushbutton **PUSH** and keep it pressed, the **START** led lights up and shows that the equipment is working and ready to melt the needle.

It is advisable to guide the needle into the melting hole by slightly rotating it. This rotation makes melting easier.

In case during melting the needle comes off the syringe and it is blocked in one of the two holes, make use of tweezers to recover the needle and repeat the operation.

### Stop and switching off

After melting has been concluded, release the **PUSH** button and the **START** led is switched off, removing voltage of device.

## MAINTENANCE

### General maintenance instructions

The product has been conceived to last forever; it is resistant to most part of chemicals except for methyl ethyl ketones (M.E.K.), diluents and nail polish removers.

### Routine maintenance

GIMA SECURITY SYSTEM is solid and easy to use, for this reason it does not need particular maintenance operations. It will be sufficient to remove dust and dirt by means of a soft cloth dampened with alcohol.

*It is recommended to carry out maintenance operations when the equipment is disconnected from the mains.*

### Scheduled maintenance

Every **4000 needles** it is necessary to replace the melting charcoals A. To replace them, proceed as follows:

- 1) Disconnect the mains plug
- 2) Unscrew the 4 screws located on the side and lift the cover.
- 3) Remove the two black plugs **B** on the drawer and introduce a small screwdriver into the holes to unscrew the screws holding the charcoal rollers.
- 4) Replace the charcoals A and screw down again. Replace the cover.

### DISPOSAL

#### Dismantling

In case of dismantling, the disposal of the parts composing the equipment shall be in compliance with the standards in force in the country of installation.

#### Environmental protection

After melting, the sterilized wastes present in the collection drawer at the base of the equipment can be disposed as any office waste.

### GUARANTEE

Thank you for having acquired our product. This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, etc. No compensation will be given for loss of use of the product.

Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

*This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.*

The guarantee will be invalidated if the registration number has been removed, cancelled or altered. Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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---

**Disposal:** The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.

For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

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