GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy
Italia: tel. 199 400 401 - fax 199 400 403
Export: tel. +39 02 953854209/221/225 fax +39 02 95380056
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

Pesaneonati elettronica BABY
BABY, electronic baby scale
Pèse-bébé électronique BABY
Elektronische Babywaage BABY
Pesa bebés electrónico BABY
Balança para recém-nascidos eletrônica BABY
Ηλεκτρονική ζυγαριά για νεογέννητα BABY
ميزان أطفال إلكتروني BABY

MANUALE D'USO E MANUTENZIONE
USE AND MAINTENANCE BOOK
INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN
BETRIEBS UND WARTUNGS ANWEISUNGEN
MANUAL DE USO Y MANTENIMIENTO
MANUAL DE USO E MANUTENÇÃO
ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ

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FEATURES

The GIMA electronic baby scale is a device suitable to weigh a newborn and measure its height. The scale is very precise and has an automatic zero setting feature. It switches off automatically after 10 seconds. A washable sheet is supplied along with the scale to cover the plate during measurements along with a metric wheel to measure the baby's height.

Technical Features

Maximum capacity: 20 Kg / 44 lbs

Precision: 10 g / 0.5 oz

Operating temperature: 5°C - 40°C

Power supply: 4 AA 1,5V batteries

Minimum capacity: 200g

Height measurement: 0-150 cm / 0-60 in

Humidity: <=85% RH

Product not certified class III

PRESCRIPTIONS

Do not use the equipment in case it is damaged. Apply to your retailer.

Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. We recommend not leaving the newborn baby on the scale unattended and do not leave the scale within the baby's reach.

UNPACKING

Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.

They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

INSTALLATION

The first operation after removing the scale from its box is to insert the batteries.

The battery box is on the base of the scale. Install 4 new batteries. Verify that the polarity is correct and close the lid.

The baby scale does not require calibration but it must be perfectly positioned on a flat surface.

OPERATION

Make sure there are no foreign objects under the baby scale and that the feet are touching the supporting surface.

Press the ON/OFF button and wait a few seconds without placing anything on the scale. When the display shows "0.00" the baby scale is ready for use.

The scale is generally set in kg. to select the unit of measure, press ON/OFF during the switching on phase (when the display shows "88:88.0"). Every time the button is pressed it is possible to choose between kg or lb, the display shows the selected measure.

To distract the newborn and make weighing operations easier it is possible to play a melody by pressing the "MUSIC" button. To deactivate the music press the same button again.

If the baby scale exceeds the maximum capacity the display shows "0-LD".

Do not weigh heavy loads and make sure the baby does not move on the plate, the loading cell may get damaged!

If the scale is not used, it automatically switches off after about 10 seconds. To switch off the unit by hand press the ON/OFF button after having removed the baby from the scale.

Remove batteries if the product is not used for a long time.

To measure the baby's height, lay the baby down on a sturdy surface and use the metric wheel supplied with the scale.

MAINTENANCE

When the display shows the "battery low" indicator it is time to replace the batteries. Dispose of the batteries according to the laws in force.

The product does not require any particular maintenance. The technical life of the scale depends on its use. To clean do not put in water or liquids. Use a soft cloth with detergents and/or disinfectants that will not ruin the product.

Disposal: *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.*

For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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