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### Eigenschaften

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## Instructions for use

### Properties

Hydrofilm plus is a transparent adhesive wound dressing with an absorbent pad that does not stick to the wound. Thanks to the transparent, extremely thin film, the area surrounding the wound and the secretion absorbed by the wound pad can be inspected at any time. Unnecessary changes of dressing are thus avoided and the wound is not disturbed. Since Hydrofilm plus is germproof and waterproof, the risk of secondary infection is minimised, and the patient can have a bath or a shower. The semi-permeable polyurethane film allows the free passage of water vapour and gases. The hypoallergenic adhesive layer ensures good adhesion, even on awkward areas of the body.

### Indications

Hydrofilm plus is suitable for the sterile post-operative care of slightly secreting wounds and as a wound covering after surgical procedures and injuries.

### Contraindications

Hydrofilm plus should not be used on clinically infected, bleeding or highly secreting wounds.

### Method of application (See illustrations)

Excessively hairy areas of the skin should be shaved before applying Hydrofilm plus.

### General instructions

Contents sterile unless peel packs are damaged. Store in cool and dry conditions. Single use only. Do not resterilize.

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