GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

TERMOMETRO A DISTANZA A INFRAROSSI INFRARED NON-CONTACT THERMOMETER THERMOMÈTRE À INFRAROUGES SANS CONTACT TERMÓMETRO INFRARROJO SIN CONTACTO

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Introduction

Congratulations on your purchase of our IR Thermometer. This thermometer is intended for scanning groups of individuals or monitoring an individual for elevated temperatures. Always use a clinical thermometer when high accuracy body temperature measurements are required

Considerations and Warnings:

- Read all instructions before using this product.
- For the most accurate results, make reading with an ambient (room) temperature of 16 to 35°C (60.8 to 95°F).

- Make sure the skin is dry and that no hair interferes with the measurement.
- Do not immerse the meter in water
- Avoid touching and/or scratching the infrared sensor lens.
- Always use a clinical thermometer to verify any abnormal temperature measurements.
- Clean the lens area by gently blowing with compressed air and use a damp swab to wipe the lens. Do not use any solvents to clean the lens.

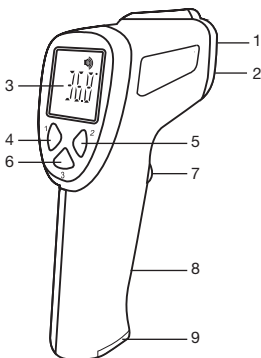
Measurement Notes

- If the meter has been stored in a cold or hot environment, allow it at least 30 minutes to acclimate to room temperature before making measurements.
- To prevent transmission of disease, avoid direct contact with the skin.
- After heavy exercise always wait at least 10 minutes before taking forehead temperature readings.

Meter Description


Meter Description

1. Laser sight
2. IR sensor
3. LCD display
4. Body/Object & “-” button
5. Voice On/Off & “+” button
6. Memory/Setting mode button
7. Measurement trigger
8. Handle
9. Battery compartment

Display

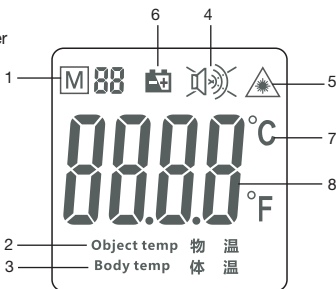1. Memory/Memory space number
2. Object temperature Mode
3. Body temperature Mode
4. Voice status
5. Laser
6. Low battery icon
7. Measurement unit
8. Temperature

Operation Instructions

Temperature Mode settings

- With the meter ON, press “3” button, for three seconds until to “F-1” shown to enter the long term calibration drift correction mode. On entering the mode, the previous temperature correction factor will appear on the display. To make a correction, measure a known, fixed temperature source. Enter the correction mode and press the + or - buttons to change the correction value and minimize the difference in readings. Repeat and adjust the correction value as needed until the measurement on the meter matches the known temperature.

The mode is applied to Body Temp.

- Press the “3” button a second time “F-2” shown to set the alarm temperature limit. Press the + or - buttons to change the value.

The mode is applied to Body Temp

- Press the “3” button a third time “F-3” shown to set the °C temperature units. The temperature units °C or °F will be shown. Press the + or - buttons to change the unit.
- Press the “3” button a fourth time “F-4” shown to set the backlit status. Press the + or - buttons to switch from ON to OFF.
- Press the “3” button a fifth time “F-5” shown to set the laser status. Press the + or - buttons to switch from ON to OFF.

Notice: Only when the laser head installed.

Mode	Function	“+”	“-”	Default	Remark
F-1	Setting Deviation Value	Increase 0,1°C	Decrease 0,1°C	0°C	Object mode is not available
F-2	Setting Alarm Value	Increase 0,1°C	Decrease 0,1°C	38.0°C	Effective Range: -3°C~3°C
F-3	Setting Measurement Unit	°F	°C	°C	
F-4	Backlit ON/OFF	On	Off	On	
F-5	Laser setting	On	Off	Off	Only when the laser head installed
Save	Save and automatic shut off				

Body Temperature Measurements

- Press the trigger to switch the meter on with one beeper. The device performs a brief self test: all elements of the display are shown for approx. 1 second. Following a successful self test with backlit (Red-Orange-Green),
- Press the “1 B/O” button for “Body temp” for accurate measurements in the range of 32.0 to 42.9°C (89.6 to 109.2°F).
- Hold the meter by its handle and point it toward the surface to be measured. Measurement distance should be 5 to 15cm (2 to 6”).
- Press the trigger to take a temperature reading. The temperature reading appears in the display.
If the F-4 MODE of backlit & “2” button voice ON & F-3 MODE set alarm temperature 38.0°C:

Temperature (°C)	Temperature (°F)	Fever Teller	Beep
$T \leq 32.0$	$T \leq 89.6$	Green	Three beeps
$32.1 \leq T \leq 37.5$	$89.8 \leq T \leq 99.5$	Green	One beep
$37.6 \leq T \leq 37.9$	$99.7 \leq T \leq 100.2$	Orange	One beep
$38.0 \leq T \leq 42.9$	$100.4 \leq T \leq 109.2$	Red	Ten Rapid beeps
$T \leq 43.0$	$T \leq 109.4$	Green	Three beeps

Notice: Red Backlit is followed alarm temperature limit (Fever alarm) .

- Release the trigger and the reading will hold for approximately 20 seconds after which the meter will automatically shut off
Note: Before shutting off, you can take further measurement.
- If the temperature goes above 109.2°F or 42.9°C, Hi will appear in the display. If the temperature goes below 89.6°F or 32°C, Lo will appear in the display.

Notice: When surrounding temperature is lower than 10.0°C or higher than 40.0°C, it's not allowed to measure of accuracy in not assured.

Object Temperature Measurements

- Press the trigger to switch the meter on with one beeper. The device performs a brief self test: all elements of the display are shown for approx. 1 second. Following a successful self test with backlit(Red-Orange-Green).
- Press the "1 B/O" button for "Object temp" for wide range surface measurements of most materials.
- Hold the meter by its handle and point it toward the surface to be measured.
- Press the trigger to take a temperature reading. The temperature reading appears in the display.
- Release the trigger and the reading will hold for approximately 20 seconds after which the meter will automatically shut off.

Note: Before shutting off, you can take further measurements.

- If the temperature goes above 212°F or 100°C, "Hi" will appear in the display.
- If the temperature goes below 32°F or 0°C, "Lo" will appear in the display.

Notice:

- The value under this Mode is object temperature instead of core temperature.
- The defaulted infrared from the real temperature according to different emissivity. For example, the reading on stainless steel will be obviously lower than the actual temperature. BE CAUTIONS FOR SCALDING.

Data memory

- To review the logged data points, press the "3 Memory" button for one second while the units is ON until "M" symbol shown on the display. Use the + or - buttons to scroll through the stored measurements and memory locations.

Notice:

Under "1 B/O" switch for "Body temp" Mode, the body temperature memories display;

Under "1 B/O" switch for "Object temp" Mode, the object temperature memories display;

Battery Replacement

- When the low battery symbol appears in the display, replace the meter's battery. The battery compartment is located on the bottom of the handle. Press down on the battery compartment lock and at the same time slide the battery compartment downwards. Replace the (2) "AA" batteries and close the battery compartment cover.

You, as the end user, are legally bound (Battery ordinance) to return all used batteries and accumulators; disposal in the household garbage is prohibited! You can hand over your used batteries / accumulators at collection points in your community or wherever batteries / accumulators are sold!

Disposal: Follow the valid legal stipulations in respect of the disposal of the device at the end of its lifecycle.

IR Measurement Notes

- Before measuring, be sure to clean surfaces that are covered with frost, oil, grime, etc.
- If an object's surface is highly reflective, apply masking tape or flat black paint to the surface before measuring. Allow time for the paint or tape to adjust to the temperature of the surface it is covering.
- Measurements through transparent surfaces such as glass may not be accurate.
- Steam, dust, smoke, etc. can obscure measurements.
- The meter automatically compensates for deviations in ambient temperature. However, it can take up to 30 minutes for the meter to adjust to extremely wide changes.

	Range	Accuracy
Body Temp	35.5 to 42.0°C (89.6 to 108.5°F)	±0.2°C / ±0.4°F
	In the remaining measurement range	±0.3°C / ± 0.5°F
Object Temp	0 to 100.0°C (32.0 to 212.0°F)	±0.8°C / ±1.5°F

Body Temp range	32.0 to 42.9°C (89.6 to 109.2°F)
Surface Temp range	0 to 100.0°C (32. 0 to 212.0°F)
Emissivity	0.95 fixed
Field of View	D/S = Approx. 5:1 ratio (D = distance; S = spot or target)
IR Spectral response	8 to 14 um (wavelength)
Display	Backlit LCD display with function indicators
Response time	500ms
Over range indication	"Hi" or "Lo"
Operating Temperature	10°C to 40°C (50°F to 104°F)
Operating Humidity	10% to 90% RH operating <80% RH storage
Storage Temperature	-25 to 55°C (-13 to 131°F)
Power Supply	(2) AA" Batteries
Automatic Power Off	20 seconds (approx.)
Weight	181g

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