



GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. – Via Marconi, 1 - 20060 Gessate (MI) Italy
Italia: tel 199 400 401 – fax 199 400 403 – gima@gimaitaly.com
Export: phone + 39 02 953854209/221/225 – fax + 39 02 95380056
- export@gimaitaly.com – www.gimaitaly.com

Code 27741 RAISED TOILET SEAT - with fixing system **Code 27743 RAISED TOILET SEAT - with fixing system and armrest**

Description:

Raised toilet seats for elderly and disabled people.

How to use the product:

In order to assemble the fasteners on the sides turn upside down the toilet seat and plug in the two brackets inside the slides. In order to fix brackets please screw the coupling pin clockwise until brackets will be firmly fastened.

Keep the two brackets as externally as possible to allow insertion of raised toilet seats on WC.

Lift the lid and the WC seat, place the raised toilet seat on WC making sure that the brackets are on the back of raised toilet seat.

Screw both brackets clockwise until you will notice a little resistance.

Make sure that the raised toilet seat is safely fastened to the WC and that the seat doesn't move sideways.

Warning:

The lateral fasteners are manufactured for a simple and effortless use.

Therefore it is important not to keep screwing brackets after they are already fixed on the WC.

Before every use check that the raised toilet seat is firmly fastened to the WC.

Don't exceed the maximum load (200 Kg.).

Disabled people should use the product under the supervision of an adult.

Cleaning instructions:

The raised toilet seat can be cleaned with lukewarm water and neutral soap.

It can also be sterilized in autoclave, without the fixing brackets, for 1 minute at a temperature of 85°C.

Maximum load: 200 Kg.

Size: 40.7x36.6 cm (height 10 cm)

Hole size: 26 x 20 cm