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Eigenschaften

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Instructions for use

Properties

Omnistrip is a sterile skin closure strip made of skin-coloured, random-laid non-woven polyamide material and coated with a hypo-allergenic polyacrylate adhesive. Omnistrip skin closure strips are intended for atraumatic primary closure of minor wounds and surgical incisions (healing by first intention). Combined with sutures or staples, the strips approximate wound edges and minimize skin trauma during scar formation once sutures or staples have been removed.

Method of application

1. Clean the incision site thoroughly and ensure that the wound edges are dry. Note: Do not apply adhesive accelerators to the skin prior to the application of Omnistrip.
2. Peel off backing paper and carefully press one end of Omnistrip to the skin, at the mid point of the incision or wound and at right angles to the edge of the incision. Pull wound edges together and press the other half of the strip firmly into place.
3. Close the entire incision with more strips, leaving an approx. 3 mm-gap between each strip allowing the incision to drain. Make sure that the skin is dry before applying the next strip. Criss-cross application is recommended only on joints or other parts of the body where the edges of the incision are subject to stretching.
4. When removing Omnistrip, pull each end towards the wound centerline and lift up carefully. Do not pull across the wound.

Contraindications

Omnistrip is intended for closure of the epidermis only. Do not apply to other types of tissue or to wounds healing by second intention.

General instructions

Contents sterile unless peel packs are damaged. Store in cool and dry conditions. Single use only. Do not resterilize.

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