# AquaProtect®

**AquaProtect** is the effective water protection for arms and legs which enables you even to swim and bath. It is tough, reusable for many times, and easy-to-clean.

#### Safe application of AquaProtect :

In case of a rough surface pull over a stocking or similar in order to avoid splitting.

Cover open wounds.

AquaProtect is extremely elastic – just slip it on and pull as high as possible. Pull back slightly and make seal lie flat to the skin of the muscle area.

If you feel the seal is constrictive and/or it rolls up at the end, reduce tension by stretching several times. Ready for a shower.

For bathing and swimming immerse carefully the major part of the AquaProtect. Remove excess air inside by pulling a corner and re-establish seal.

#### Please Note:

Although with non-slip sole, always move carefully in wet areas.

Don't extend bathing time excessively (airsealed, sweat).

The right model depends on the size of arm or leg where the seal will be applied. For watertight sealing avoid elbows and knees. If in doubt, choose full arm or full leg type.

**Cleaning:** With hot water (up to 95°C/200°F) and/or soft detergent.

**Desinfection:** With surface or spray disinfectants.

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