


# Wall Cabinet Manual


## GUIDE D'UTILISATION DE L'ARMOIRE MURALE

Instruction pour le changement des piles

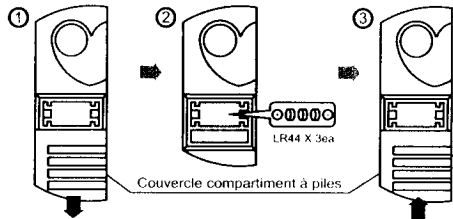

- ① Ouvrir le compartiment à piles
  - ② Insérer les 3 piles LR44 comme indiqué sur la photo  
Assurez-vous que le sens de polarité des piles soit respecté
  - ③ Fermer le compartiment à piles
- △ Attention : Le compartiment à piles doit être fermé à l'arrêt de l'appareil.  
Le port d'alarme des piles doit être fermé à l'arrêt de l'appareil.

- Les piles de l'alarme ont une autonomie de 30 minutes en utilisation continue (si la porte de l'armoire reste ouverte)  
- Alimentation par 3 piles bouton type LR44 X 3EA
- Pour mettre l'alarme en service, baisser le bouton sur le boîtier de l'alarme puis fermer la porte.


**CU Medical Systems, Inc.**  www.cu911.com

- |  |   |
|--|---|
| <ul style="list-style-type: none"> <li>• Head office</li> </ul>  | <ul style="list-style-type: none"> <li>• Europe</li> </ul>  |
| Dongwha Medical Instrument Complex<br>1647-1 Dongwha-ro, Munmak-eup, Wonju-si,<br>Gangwon-do, 220-801 Republic of Korea<br>TEL : +82 31 578 5725 / FAX : +82 31 578 5729 | Branch Office of CU Medical Systems, Inc. in Germany<br>Kuester Strasse 6, 30519 Hannover, Germany<br>TEL : +49 511 365 4353 / FAX : +49 511 848 6054 |

NF-0A12 - Armoire murale pour défibrillateur


## NOTICE

Battery Replacement Method

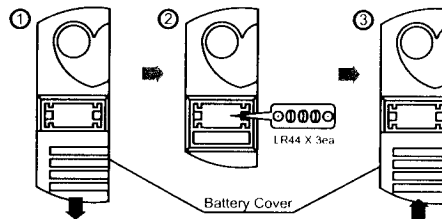

- ① Open the battery compartment cover.
  - ② Insert the LR44 battery as shown in the picture.  
Ensure that the batteries are inserted with correct polarity.
  - ③ Close the battery compartment cover.
- △ Attention: The battery compartment must be closed when the device is not in use.


- The battery for the audible alarm lasts for about 30 minutes if the door remains open.  
- Battery Type : LR44 X 3EA
- Turn on the alarm button on the low right.

**CU Medical Systems, Inc.**  www.cu911.com

- |  |   |
|--|---|
| <ul style="list-style-type: none"> <li>• Head office</li> </ul>  | <ul style="list-style-type: none"> <li>• Europe</li> </ul>  |
| Dongwha Medical Instrument Complex<br>1647-1 Dongwha-ro, Munmak-eup, Wonju-si,<br>Gangwon-do, 220-801 Republic of Korea<br>TEL : +82 31 578 5725 / FAX : +82 31 578 5729 | Branch Office of CU Medical Systems, Inc. in Germany<br>Kuester Strasse 6, 30519 Hannover, Germany<br>TEL : +49 511 365 4353 / FAX : +49 511 848 6054 |

## 주의사항

경광등 배터리 교체방법

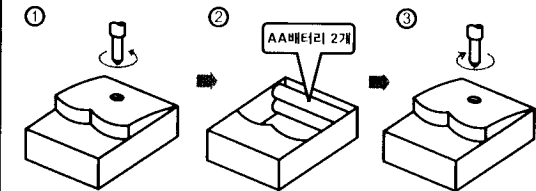

- ① (+) 드라이버로 경광등의 커버를 연다.
- ② AA 배터리 2개를 교체한다.
- ③ (+) 드라이버로 경광등의 커버를 닫는다.

- 보관함의 문이 열려져 있을 경우 알람 및 경광등에 사용되는 배터리 연속 사용 시간은 30분입니다.  
- 배터리 종류 : AA Battery X 2EA
- 경광등 및 알람은 캐비닛의 위쪽에 부착되어 있습니다.

**(주)씨유메디칼시스템**  www.cu911.com

- |  |  |
|--|--|
| <ul style="list-style-type: none"> <li>• 본사 / 공장</li> </ul>                              | <ul style="list-style-type: none"> <li>• 의왕사무소</li> </ul>  |
| 강원도 원주시 문막읍 동화리 1647-1<br>동화의료기기산업단지 (220-801)<br>전화 : 033-747-7657<br>팩스 : 033-747-7659 | 경기도 의왕시 정계동 991-4번지<br>창계프라자 5층<br>전화 : 031-421-9700<br>팩스 : 031-421-9911<br>A/S 문의 : 080-910-0911 |

French

English

Korean

**CU Medical Systems, Inc.**