GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy
Italia: tel. 199 400 401 - fax 199 400 403
Export: tel. +39 02 953854209/221/225 - fax +39 02 95380056
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

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1. Simple Structure Figure

Self-Driving Version with 24" Rear Wheels

This wheelchair version is particularly suitable for the patient transfer.

1 Handle	2 Side plate
3 Spoke rear wheels	4 Pneumatic tires (for Oxford model) PU tires (for Oxford Plus model)
5 Hand rim	6 Quick-Released axles
7 Front wheels	8 Front forks
9 Footrests	10 Leg strap
11 Seat and Back upholstery	12 Cross braces
13 Foot plates	14 Anti-Tipper (only for Oxford Plus model)

2. General Safety Instructions

2.1 Please read the instructions for use first! Before using the product, you should be familiar with the handling, function and use of the product.

You may compromise your safety if you do not observe these instructions. However, it is impossible to protect against and avoid all unforeseeable circumstances and situations.

- 2.2** Do not ascend or descend stairs without the assistance of attendants. If devices such as ramps or elevators are available, they should be used. If wheelchair-friendly access is missing, two attendants must carry the wheelchair over the obstacle.
- 2.3** If possible, the footplates should be flipped up before getting into or out of the wheelchair.
- 2.4** Risk of accident when driving without experience. This can lead to falls and other dangerous situations. Therefore, familiarize yourself with the new wheelchair by practicing on even, straightforward terrain first. Learn how the wheelchair reacts when the center of gravity shifts; for example on slopes or inclines or when clearing obstacles such as steps and curbs. This should be done only with the assistance of another person. Using an anti-tipper is strongly recommended for inexperienced wheelchair users.
- 2.5** All brakes acting on the tires do not serve as service brake but are only designed as parking brake (wheel lock). The wheel locks must not be used as driving brakes for slowing down the wheelchair, as in extreme cases, the abrupt stopping of the wheelchair can lead to falls.
- 2.6** Risk of damage caused by overload. The maximum load capacity is 120 kg.
- 2.7** The drive wheels must be replaced in case of tire damage (tread becomes worn down to such an extent that there is only 5 mm of radius left on the outer edges of the tire) or damage on the rim. The 12" drive wheels of the transfer version is replaced in the same way.

3. Delivery and Preparation for Use

(Please Note: The photos here may show differences to the wheelchair you actually received).

Your wheelchair is usually delivered fully assembled, folded, and with the rear wheels mounted.

A few simple steps are sufficient to prepare it for use:

- 1.** While standing next to the wheelchair (Figure 1), unfold the wheelchair using flat hands (Figure 2). Ensure that the telescoping bars (Figure 1-A) are completely collapsed and that the seat upholstery is flattened out.

fig. 1

fig. 2

2. If applicable, insert the footrests into the receivers from the side (Figure 3). Turn the footrests inward until locking to the right place. Finally, fold down the footplates (Figure 4).

fig. 3

fig. 4

4. Mounted Anti-Tipper (only for Oxford Plus model)

The anti-tipper prevents the wheelchair from tipping too far backward. The anti-tipper must be rotated into the active position on one or both sides before use. This is accomplished by pressing the ball ratchets (Item A) and making the adjustment. The anti-tipper must audibly lock in place, before it is able to bear loads.

For untrained wheelchair users, the use of an anti-tipper is strongly recommended.

5 Transportation

Grab the middle of seat upholstery at the front and the back and pull it upwards, the wheelchair will be folded (Figure 5).

For more manageable transportation, e.g. in cars, the footrests can be entirely removed in just a few easy steps. To move the footrest, press the release lever to the inside (Figure 6). The footrest with footplate can then be swung outward by 90° and be removed by pulling it upward.

fig. 5

fig. 6

6. Armrest usage

The side panels can be flipped-up to the rear for transfer user (Figure 7). To raise the side panel, manipulate the release lever (Figure 8) behind the side panel tube. The height of the armrest is not adjustable.

fig. 7

fig. 8

7. Footrests usage

The height of standard footrests can be changed with tools (Figure 9). To release the screws and then adjust the height to right place for user, finally tighten the screws.

fig. 9

fig. 10

8. Replacing the Seat and Back Upholstery

Can be removed if necessary by loosening the cross screws (Figure 10).

9. Cleaning and Disinfection

- 9.1** Clean the upholstery with warm water and a washing-up liquid. Remove spots with a sponge or a soft brush. Rinse with clear water and let the components dry.
- 9.2** Do not use any aggressive cleansing agents, solvents, or hard brushes etc.
- 9.3** No wet-cleaning. Components are not machine washable. For disinfection, water-based agents (e.g. Sagrotan original concentrate) should be used. The manufacturer's instructions for use must be observed.
- 9.4** Before disinfecting, the pads and handles must be cleaned. Plastic parts, frame parts as well as chassis and wheels can be cleaned with a mild detergent. Dry thoroughly afterwards.

10. Re-use

Information on Re-Use

The product is suitable for re-use. Similar to second-hand machines or cars, products that are being re-used are subject to increased strain. Features and functions must not change in a way that could endanger patients or other persons within the product's life cycle.

Based on market observations and the current state of technology, the manufacturer has calculated that the product can be used for a period of **4 years**, provided that it is used properly and that the service and maintenance instructions are observed. Periods during which the product is stored at the dealer or with the third-party payer are not included in this period. It should be clearly pointed out, however, that the product is reliable far beyond this defined period of time, provided that it is cared for and maintained appropriately.

In cases of re-use, the affected product must first be thoroughly cleaned and disinfected. Subsequently, the product must be examined by an authorised specialist to check the condition, wear and possible damage. Any worn and damaged components as well as components which do not fit or are not suitable for the user must be replaced.

11. Specifications and Parameters

Specifications	
Style	Self-propelled
Surface finishing	Powder coating
Frame	Steel and foldable
Armrest	Flip-up desk length armrest
Seat and backrest	Nylon upholstery
Cross bars	Seat width 43cm with single (for Oxford model) Seat width 46cm with single Seat width 51cm with double
Footrest	Swing away footrest with leg strap
Rear wheel	24" Spoke rear wheel Pneumatic tire (for Oxford model) PU tire (for Oxford Plus model) & quick-release axles
Front wheel	8"x2"
Handrim	Steel with chromeplate
Front fork	Steel with black powder coating
Footplate	Plastic
Anti-tipper	Not available for Oxford model With anti-tipper (only for Oxford Plus model)
Brake	With hand brake
Measurements	
Total length	1020 mm
Total width	630 mm - 680 mm
Total height	920 mm
Folding width	260 mm
Seat width	460 mm - 510 mm
Seat depth	400 mm
Backrest height	410 mm
Back of seat to floor	480 mm
Front of seat to floor	500 mm
Seat to armrest	275 mm
Net weight	18 kg
Gross weight	20 kg
Weight capacity	120 kg
Carton size	820x260x850 mm

Disposal: *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment. For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.*

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product. This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA. During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included. All components subject to wear are not included in the warranty. The repair or replacement performed during the warranty period shall not extend the warranty. The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use. GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc. The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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